GentleCath™
that's a relief

Air

Please read both instruction
leaflets before using this device.

English

(en)

Hydrophilic Intermittent Urinary Catheter

Indications

- This intermittent catheter is indicated for self-catheterisation and routine transient drainage of the bladder. The catheter is inserted through the urethra.
- A healthcare professional should assess the need and suitability for intermittent self-catheterisation and its contraindications.

Contraindications

- Do not use in conjunction with irrigation or instillation.

Cautions, precautions & observations

- Check the catheter is the correct size as indicated on the packaging
- Do not use if the packaging has been previously opened or damaged.
- Do not touch the catheter tip. Use the no-touch sleeve to minimize contact with the rest of the catheter.
- If required, connect a urine bag to the funnel end of the catheter before use
- Use the catheter immediately upon extraction from the case.
- The catheter is lubricated and ready-to-use once removed from the case. The catheter should never be re-lubricated.
- Always slide the catheter slowly and smoothly when inserting and removing. Never force the catheter.
- When removing the catheter, stop if more urine starts to flow. Continue removing once the flow has stopped again.
- If there is no urine flow or other complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge) please consult a healthcare professional.
- After incorrect insertion, catheters must be discarded and not re-used. A new catheter should be selected.
- Used catheters should always be discarded.

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Glossary:

Urethra: Passage which urine travels through from the bladder to the outside of the body.

Hydrophilic: Smooth and slippery surface activated by water.

Instillation: Introduction of a liquid in to the bladder via the catheter which is allowed to remain for a specific length of time before being drained

Irrigation: a procedure in which sterile fluid is administered at a set rate through a catheter with the aim to keep the urine free from clots.

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©2018 ConvaTec Inc.
™ indicates a trade mark of