




WE CAN

Help Improve Lives

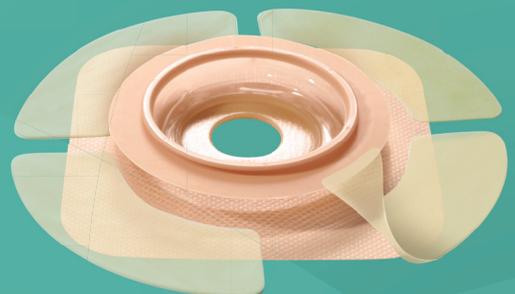

ease™ Strips



WE CAN

Help Improve Lives

ConvaTec's unique technologies - helping people throughout their ostomy journey



Moldable

Unique to ConvaTec, Moldable provides a full continuum of care option for all stoma sizes.

96%

of new ostomates maintained normal skin integrity¹

Proven longer wear time¹

Accordion

Post-operative option designed to prevent pressure or trauma.

2x

the lifting height of any other lifting flange in the market²

Available in Flat, Convex, Cut to Fit and Moldable

Convexity

Growing rates of obesity post-surgery are key contributors to the growth of Convex skin barriers.

45%

of consumers use a convex product long term³

Only ConvaTec offers convex products in cut-to-fit, pre-cut and moldable technology



WE CAN

Help People Feel More Secure



ease™ Strips

ease™ strips help keep your skin barrier in place.

5 Easy Reasons to Use ease Strips



Secure - Helps keep your skin barrier in place



Flexible and thin - Designed to move with you



Skin friendly - Latex free



Easy application - Curved to be used with any ostomy system



Water resistant - Designed to be used for both swimming and bathing

Application Tips


ease™ Strips can help with a more secure feeling.
Follow these tips for the best results.

Step 1 Preparation




Begin application by warming the strip briefly between your hands to aid adhesion. Clean the skin where the strip will be placed with lukewarm water. Pat skin dry.

Step 2 Application

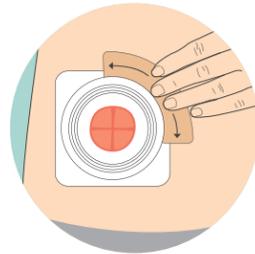

Remove half of the backing paper and apply the strip to the skin, overlapping the edge of the barrier. The strip should overlap the edges of the skin barrier at least half an inch.

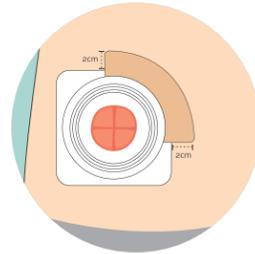

Remove the remaining backing and smooth down for 30 seconds to aid in adhesion.

Step 3 Removal



Support the skin with your hand and gently remove the strip with your other hand by peeling horizontally until removed from the skin. An adhesive remover may be used to assist.

Code	Description / Dimensions	Qty
 422163	ease™ Thin curve 3cm x 9cm (1 ¹ / ₅ in x 3 ¹ / ₂ in)	80 pcs

WE CAN

Help Elevate the Experience
of Living with an Ostomy



With me+™, we surround your patients with the education, support and resources they need to feel stronger, more confident and ready for what's ahead. Our program is designed and customized to guide the right support at the right time at all stages of life.



me+ can offer that peace of mind that there is always someone to lean on.



Care

Products, supplies and advice



Support

Dedicated WOC nurses and product specialists



Answers

Online resources, education and guided recovery tools



Style

Apparel and lifestyle solutions



Community

Peer to peer support

Enrolling your patients is simple.

Contact your me+ concierge at 1-877-585-0470 or visit mepluscare.com



WE CAN

Help Improve Lives

ConvaTec's unique technologies -
helping people throughout their ostomy journey

Moldable Technology™ **NATURA™** **ESTEEM™**  *Flex Convex*



Enrolling your patients is simple. Contact your me+™ concierge
at 1-877-585-0470 or visit mepluscare.com