



ESTEEM[®]


WITH MOLDABLE SKIN BARRIERS



1 Moldable disc application




Remove moldable disc from the pouch.

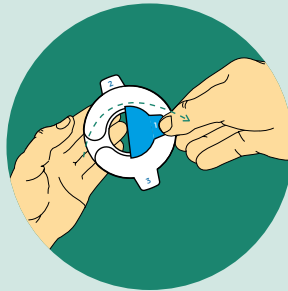


Remove liner marked **#1**. This is the molding side.




Roll and mold the disc to the size of the stoma. **Do not stretch, pull or cut.** Pinch into place once desired size is reached.

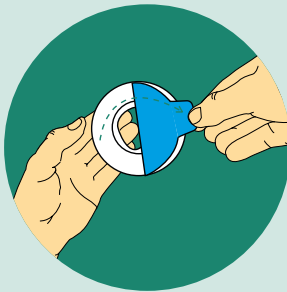

Remove liner marked **#2**. This side will go on the skin.

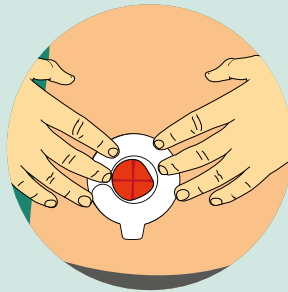


Place the disc on the body. Push the roll of adhesive toward the stoma to ensure a close fit.

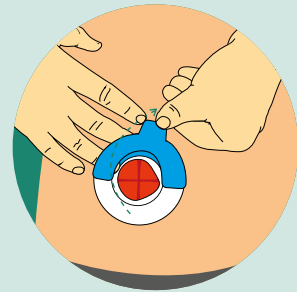


Remove liner marked **#3**.

2 Applying pouch to the moldable disc

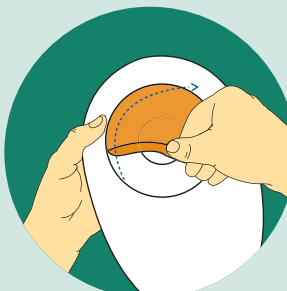


Remove pouch release liner.

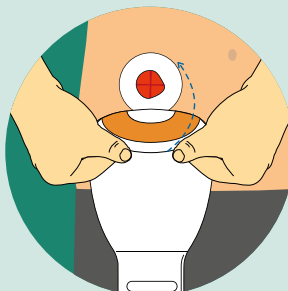


Place pouch on the moldable disc. Smooth and hold the skin barrier in place for 30 seconds.

3


Closing the pouch

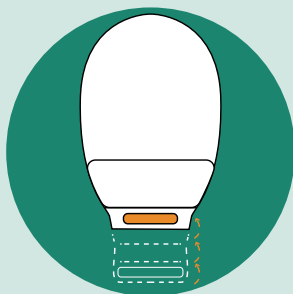

Fold the end up towards you until the interlocking closures line up.

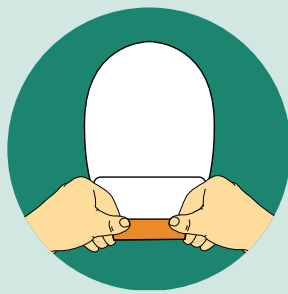

Press and pinch around the closure to secure.

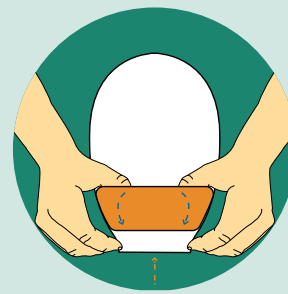


Turn the Lock-it-Pocket™ inside-out to tuck tail inside.

4


Emptying and cleaning the pouch



To empty pouch, tilt the tail up towards your body. Peel open the closure and unravel tail. Unlock pouch closure and open the tail by pushing in both ends of the outlet end-strips with fingers; drain pouch.

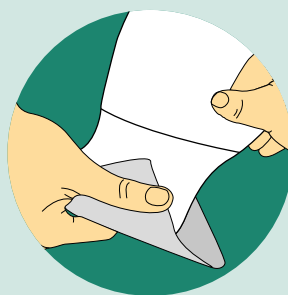


Tips for cleaning the pouch: Support the pouch with one hand and wipe tail of pouch in firm downward motion with toilet tissue in other hand.

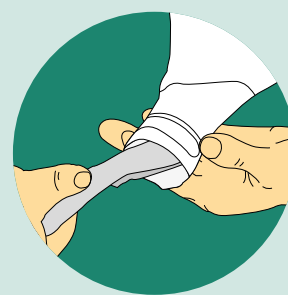

Clean in the inside surface of tail with toilet tissue or baby wipe.

5

Removing the pouch

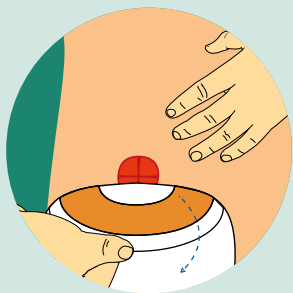

When it's time to change the pouch, the disc and pouch can be removed as one.



For more information, please call our Customer Relations Center (Registered Nurses on staff) at **1-800-465-6302**, Monday through Friday, 8:00 AM to 6:00 PM (EST), or visit our Web Site at www.convatec.ca

