

# Stoma – Coin Comparison Measurements

note: picture sizes are approximate



Dime: 18 mm or 3/4 in



Half dollar: 31 mm so almost 1 1/4 in



Penny: 20 mm or just under 7/8 in



Silver dollar: 38mm or 1 1/2 in (not the Susan B Anthony)



Nickel: 22 mm or 7/8 in



Quarter: 25 mm or 1 in

Moldable Skin Barrier Sizing		
IF YOUR STOMA SIZE IS	USE THIS MOLDABLE SKIN BARRIER	WITH THIS FLANGE SIZE
1/2" to 7/8"	Small	1 3/4" (45mm)
7/8" to 1 1/4"	Medium	1 3/4" (45mm)
1 1/4" to 1 3/4"	Large	2 1/4" (57mm)
Available in Flat only: 1 1/4" x 1 3/4" (wide) to 1 3/4" x 2 1/8" (long)	X-Large (with OVAL starter hole)	2 3/4" (70mm)

Cut-To-Fit Skin Barrier Sizing	
IF YOUR STOMA SIZE IS NO WIDER THAN	CHOOSE A FLANGE SIZE
1/2"	1 1/4"
5/8"	1 1/4"
3/4"	1 1/4"
7/8"	1 1/2"
1"	1 1/2"
1 1/8"	1 3/4"
1 1/4"	1 3/4"
1 3/8"	2 1/4"
1 1/2" to 1 3/4"	2 1/4"
1 7/8" to 2 1/2"	2 3/4"
2 5/8" to 3 1/2"	4"



If you have other questions about measuring a stoma, contact the **me+ Team** at 1-800-422-8811, M-F 8:30a - 7p ET.