


# UnoMeter™ Safeti™ Plus System with Change Bag

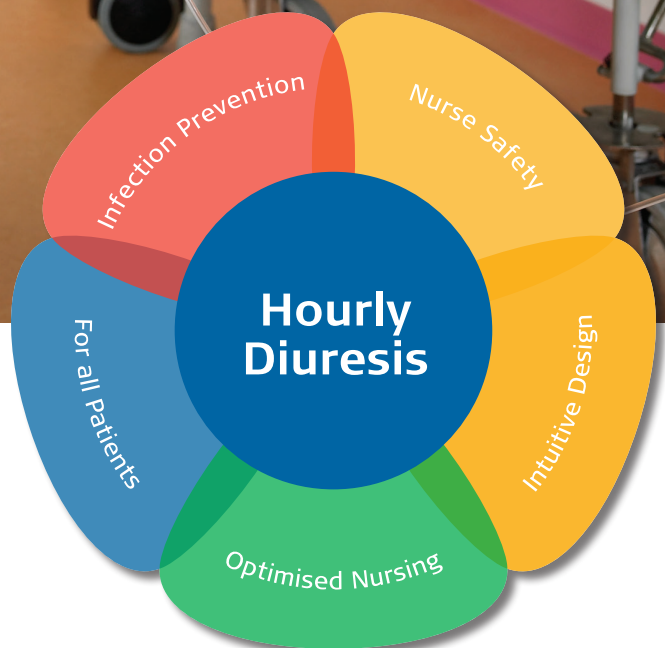

# UnoMeter™ Safeti™ Plus System with Change Bag

## Hourly Diuresis

### Improved efficiency and safety for better care

The UnoMeter™ Safeti™ Plus System Change Bag is an addition to the innovative UnoMeter™ Safeti™ Plus System range of products.

Our Circle of Hourly Diuresis shows the many features and benefits of the

UnoMeter™ Safeti™ Plus System for patients and caregivers.

This new device complements the UnoMeter™ Safeti™ Plus System, offering the same level of protections as the non-return valves (NRV) and

the KombiKon™ needle-free sample port. The introduction of the flexible Change Bag system gives staff an additional option while ensuring measurement of hourly diuresis is safe and easy.

## Infection Prevention

### Protecting patients at risk


The UnoMeter™ Safeti™ Plus System Change Bag helps to ensure safety for the patient while improving efficiency for the nurse.

The Change Bag can easily be detached from the chamber to be replaced, and the NRV in the bag ensures no urine spillage and the EasiMT™ function ensures a

controlled emptying of the bag after use – both features to reduce the risk of infection from urine contamination.

UnoMeter™ Safeti™ Plus System Change Bag also supports infection prevention with the KombiKon™ needle-free sample port, 2 NRV's and a drip chamber which is

standard within the UnoMeter™ Safeti™ Plus System range.

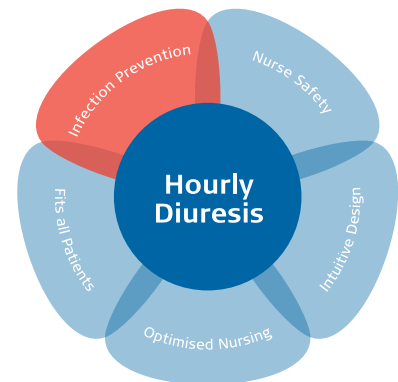

No problem to attach/detach the change bag from the chamber.



EasiMT™ function is simple to use.




Controlled emptying of change bag after use.\*

\*Follow universal precautions per hospital procedures for disposal of urine.

## Nurse Safety

The KombiKon™ needle-free sample port protects against needle stick injuries and reduces risk of infection for the health care provider. In

clinical testing, the sample port maintained the barrier for bacterial contamination for up to 10 perforations.<sup>1</sup>

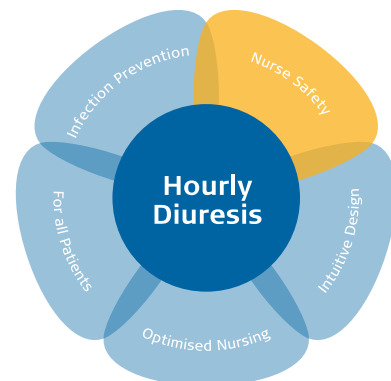

### KombiKon™

#### - needle-free support

- No risk to the nurse of needle stick injuries.
- Universal port can be used with most standard syringes.
- Easy to clean sample port.



KombiKon™ needle-free sample port reduces the risk of contamination<sup>1</sup>

### Controlled emptying of change bag


In the intensive care unit, where accurate hourly monitoring of urine output is important, the ability to empty the Change bag compliments the features of the UnoMeter™ Safeti™ Plus System.

This Change Bag system is consistent with the safety provided

by the NRV in the bag. Designed to prevent accidental detachment, it takes only a moment to untwist from the chamber.\*

The EasiMT™ system as a simple way to empty the bag reduces the risk of infection from urine spillage.


Urine flows rapidly from the chamber to the bag. Bags are available in 1.5L and 3.5L sizes.

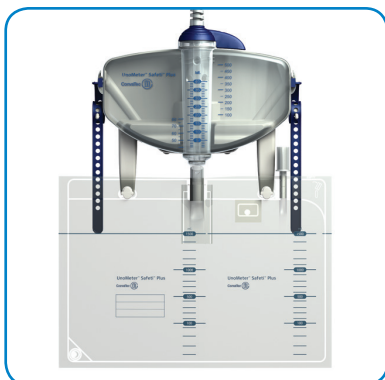

UnoMeter™ Safeti™ Plus System  
Change Bag 1.5L.



Tap ensures correct attachment.

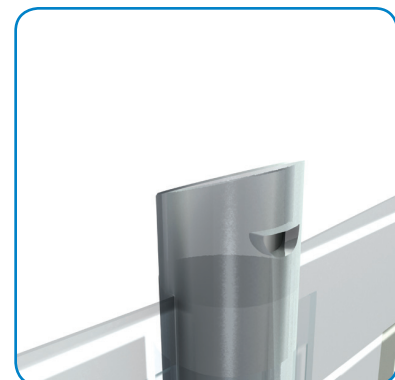

Tap fits in bag to prevent accidental detachment.

<sup>1</sup> Performance of KombiKon needle free sample port. July 2004 Data on file, Unomedical a/s, a ConvaTec company.

\*As demonstrated in vitro

\*\*See package insert for complete Directions for Use

### ***Making a Difference in People's Lives.***

ConvaTec is a global company committed to serving healthcare professionals, hospitals and patients worldwide. In the ConvaTec Continence and Critical Care division, we specialize in the development, manufacturing and distribution of single-use devices to hospitals and healthcare sectors around the world.

We are proud of our strong tradition of innovation and dedication to providing solutions. It is this legacy of ingenuity and invention that allows us to find answers to the challenges health care professionals and ordinary people are confronted with in their daily lives.

Description	Reference	Box Qty.
<b>UnoMeter™ Safeti™ Plus System Change Bag</b>		
1.5L change bag, straps & round hooks, 150 cm, EasiMT™	158100710190	10
1.5L change bag, straps & round hooks, 110 cm, EasiMT™	158101110190	10
1.5L change bag, straps, 150 cm, EasiMT™	158100610190	10
1.5L change bag, straps, 110 cm, EasiMT™	158101010190	10
<b>Single Packed Change Bags</b>		
1.5L Sterile single packed	30 810 1S-90	100
3.5L Sterile single packed	47 810 1S-90	100