


Protect your face with DuoDERM® Extra Thin* dressings when you have PPE-irritation


In the fight against COVID-19 you'll be wearing a variety of personal protective equipment (PPE), including facial masks and visors.

Wearing these, and changing them frequently for extended shifts, may cause pressure injury around the temporal bones, bridge of nose, cheeks and ears. DuoDERM® Extra Thin Dressings can be used in conjunction with medical devices where there is evidence of pressure damage inclusive of medical device and PPE related injuries.

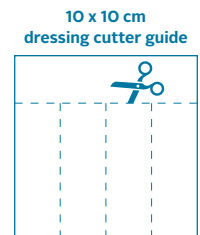

Quick and easy application

1. Wash your hands and cleanse your face – including behind your ears
2. Choose the right sized DuoDERM® Extra Thin dressings for your **forehead, nose and cheeks** or cut the dressing to fit.
3. Warm the dressing between your hands
4. Peel the 'top' white release paper off and gently apply the centre part of the dressing
5. Remove the 'bottom' white release paper while rolling the dressing into place, without stretching
6. Remove the clear white release paper on the other side
7. Carry out this process on your **forehead, nose and cheeks**, molding the dressing into place for 30-60 seconds.

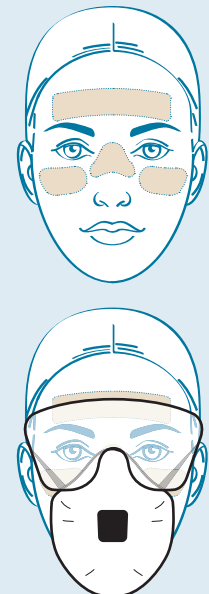

8. It is the HCP's responsibility to **ensure PPE is fit tested** after the application of DuoDERM® Extra Thin Dressings underneath PPE eg N95 or similar face masks or glasses.

Painless removal

1. Press down on the skin with one hand and carefully lift an edge of the dressing with your other hand.
2. Stretch the dressing downwards away from the skin to break the adhesive seal and remove. If necessary, use an adhesive remover, such as Sensi-Care® Adhesive Remover Wipes, to aid dressing removal.

Testing of PPE efficacy has not been undertaken in relation to ConvaTec dressings.

It is the HCP's responsibility to ensure PPE is fit tested after the application of ConvaTec dressings, such as DuoDERM® Extra Thin Dressings, underneath PPE eg N95 or similar masks or glasses.

Please consult Instructions for Use for product indications for use, contraindications, precautions/observations and directions for use.

Please report any adverse events to your local ConvaTec office or representative.

*DuoDERM® Extra Thin is also known as Granuflex® Extra Thin or VariHesive® Extra Thin.

All trademarks are the property of their respective owners. ©2020 ConvaTec Inc.

AP-021312-MM