



# ESTEEM™ Flex Convex

## HOW TO USE THE UROSTOMY POUCH SOFT TAP



### 1 CLOSING THE POUCH



Step 1: Make sure the tap is securely closed.

### 2 DRAINING THE POUCH



Step 1: Tilt tap up and towards you.



Step 2: Hold base of tap and open tap with the other hand.



Step 3: Direct tap into toilet to drain the urine.

### 3 USING THE CONNECTOR



Step 1: Gently insert the ridged part of the end of the tap into the wider section of the adaptor with one hand while holding the base.



Step 2: To remove, pull out Connector with one hand while holding base of tap with the other hand.

**For more information, call us on - 0800 282 254**  
Please refer to the package insert for complete instructions for use.

TM ® indicates trademarks of ConvaTec Inc.  
© 2016 ConvaTec Inc. AP-017083-MM

