

Product Description

Exudate Management – For moderately to heavily exuding malodorous wounds

CarboFLEX® dressing is a five-layer dressing specifically designed to address the management problems associated with malodorous wounds.

CarboFLEX® dressing provides effective exudate control and odour adsorption while being soft and conformable^{1,2}.

Application*

Carefully cleanse the wound with saline and dry surrounding skin. Choose a dressing size that is large enough to overlap the wound edge by at least 3cm. For shallow wounds, CarboFLEX® dressing can be used as a primary dressing. For deeper wounds, CarboFLEX® dressing may be used as a secondary dressing over a suitable cavity filler.

Place the fibrous (non-shiny) surface of the dressing directly onto the wound or over the cavity filler. The absorbent wound contact layer will form a soft gel when in contact with wound exudate. For wounds with very heavy levels of exudate, use an appropriate absorptive dressing such as AQUACEL® dressing as a primary dressing (either ribbon for cavities or a sheet for shallow wounds) and CarboFLEX® dressing as a secondary dressing.

CarboFLEX® dressing can be secured in place with tape or other appropriate material.

*Please refer to pack insert for full instructions, prior to use.

Specifications

Size	Dressings Per Box	Product Code
10 cm x 10 cm	10	403202
8 cm x 15 cm (oval)	5	403203
15 cm x 20 cm	5	403204

RPBS Codes

Size	RPBS Code	Product Code
10 cm x 10 cm	4742N	403202
15 cm x 20 cm	4743P	403204

ARTG Number: 158689

References

1. Griffiths B, Jacques E, Jones S, Davies B, Bowler B, Bishop S. Determination of malodour reduction performance in various charcoal containing dressings. Data on file, ConvaTec.
2. Wainisley R, Waring M. An investigation into the fluid handling characteristics of the wound contact layers of several odour absorbing dressings. Data on file, ConvaTec.
3. Williams, C. Role of CarboFLEX in the Nursing Management of Wound Odor. British Journal of Nursing 2001 vol 10 no. 2.



Clinical Benefits

- Supports moist wound healing environment
- Effectively controls wound malodour¹
- Absorbs and controls exudate²
- Non-adherent allowing less traumatic dressing changes
- Soft and conformable
- Aesthetically pleasing to patients³
- May help relieve feelings of social isolation and restore lost confidence³

Indications

CarboFLEX® dressing is indicated for the management of malodorous wounds.

- Chronic wounds: leg ulcers, pressure ulcers, diabetic ulcers and fungating lesions
- Acute wounds: lacerations and post-surgical wounds

CarboFLEX® dressing may be used as a primary dressing for shallow wounds or with deeper wounds as a secondary dressing over a wound filler.

Call your local ConvaTec Territory Manager or ConvaTec Support Centre for further information:

Australia: 1800 335 276 or New Zealand: 0800 441 763

www.convatec.com

ConvaTec (Australia) Pty Limited. ABN 70 131 232 570. Unipark Monash, Building 2, Ground Floor, 195 Wellington Road, Clayton VIC 3170 Australia. PO Box 63, Mulgrave, Vic 3170. Phone: (03) 9239 2700 Facsimile: (03) 9239 2743. ConvaTec (New Zealand) Limited. AK2135265 Level 4, 369 Queen Street, Auckland 1010 New Zealand. PO Box 62663, Kalmia Street, Auckland 1544 New Zealand. Phone: (09) 306 8833 Facsimile: (09) 306 8831.

®/™Indicates trademark of ConvaTec Inc. © 2009 ConvaTec Inc. AP-006688-AU. March 2009.

Our world is what we make of it™

