





NATURA[®]


WITH MOLDABLE SKIN BARRIERS

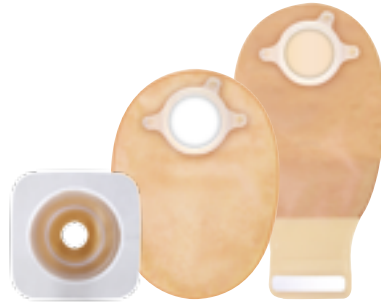


1 Getting the right fit

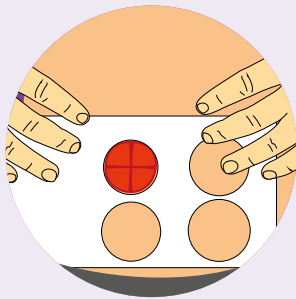

Measure the stoma and select the appropriate size Moldable skin barrier.

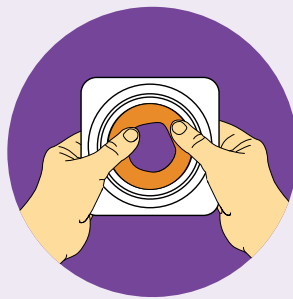


Mold the barrier with your fingers, then roll and shape the disc to the size of your stoma.
Do not stretch, pull or cut.

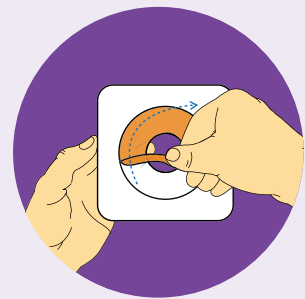


Peel off clear release liner from the reverse of the barrier. Position the circular opening over your stoma.


2 Skin Barrier Application

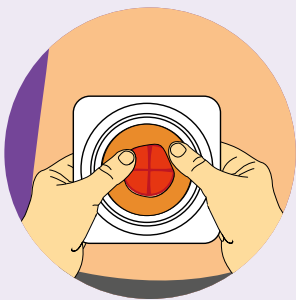

Press the skin barrier into place and if necessary re-adjust the rolled adhesive to create a snug fit. Smooth and hold the skin barrier in place for 30 seconds.

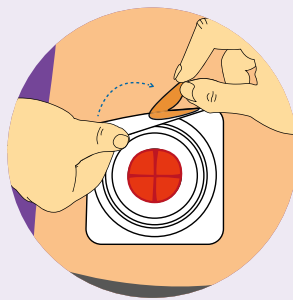


Peel off the white release paper from the tape border.

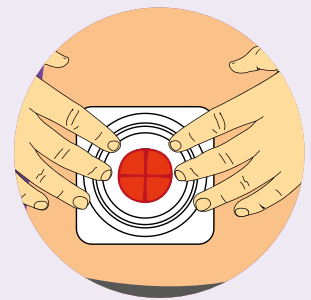


Press gently against skin, smoothing out any wrinkles.


3 Closing and applying the pouch

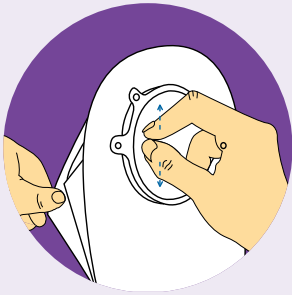


Separate the side of the pouch to allow some air to enter.

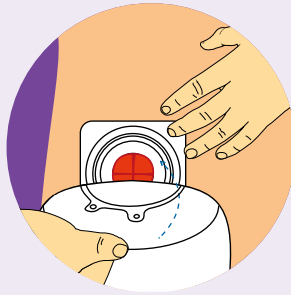

Position pouch over the skin barrier. Gently press the flanges together starting at the bottom until they “snap” in place and feel secure. **Always double check the coupling is completely closed, if not repeat the steps.**

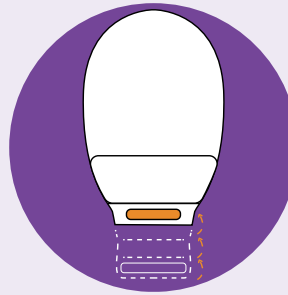


Fold the end up towards you until the interlocking closures line up. Press and pinch around the closure to secure.

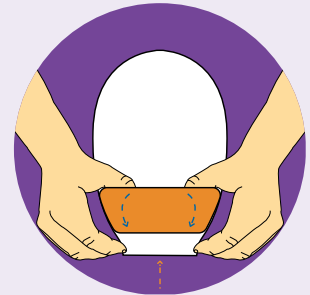


Turn the Lock-it-Pocket™ inside-out to tuck tail inside.


4 Emptying and cleaning the pouch

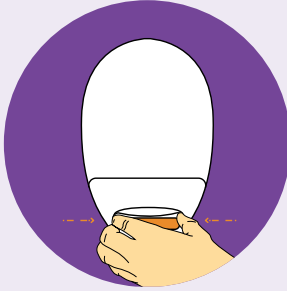

To empty pouch, tilt the tail up towards your body. Peel open the closure and unravel tail. Unlock pouch closure and open the tail by pushing in both ends of the outlet end-strips with fingers; drain pouch.

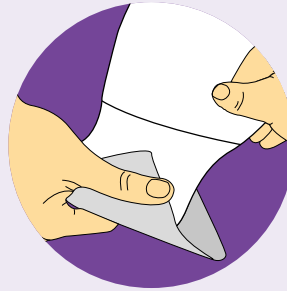


Tips for cleaning the pouch: Support the pouch with one hand and wipe tail of pouch in firm downward motion with toilet tissue in other hand.

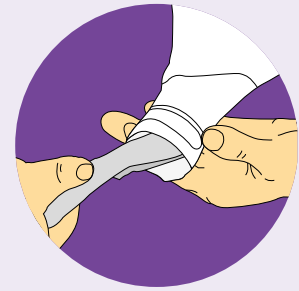


Clean in the inside surface of tail with toilet tissue or baby wipe.

5 Removing the pouch

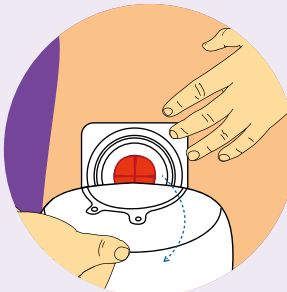

To remove the pouch from the skin barrier firmly pull downward on the pouch tab and lift the pouch off.

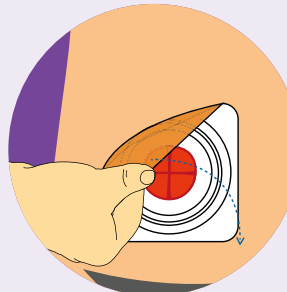

Gently pull the skin barrier down, away from the skin. While supporting the adjacent skin with your other hand.