


## ESENTA™ STING FREE SKIN BARRIER SPRAY

EASY TO USE: FOLLOW THE BELOW STEPS




### GETTING STARTED

Shake well before use. Clean and dry the skin.

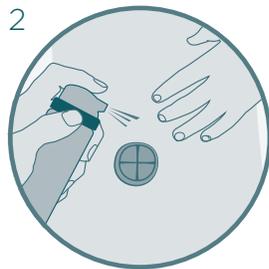

### EASY TO APPLY

Hold approximately 10 cm or 4 inches from the area to be treated and apply a uniform coating in a sweeping motion.

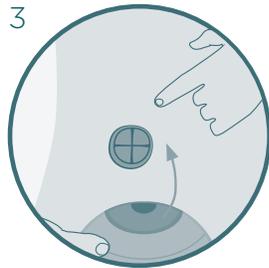

### CHECK IF THE SKIN IS DRY

Ensure the area is completely dry before applying adhesive products such as ostomy appliances or dressings.




### REAPPLY

ESENTA™ Skin Barrier should be replaced at every dressing or appliance change to maintain complete protection. ESENTA™ Barrier will not cause a build up of barrier film on the skin surface.




## ESENTA™ STING FREE SKIN BARRIER WIPES

EASY TO USE: FOLLOW THE BELOW STEPS

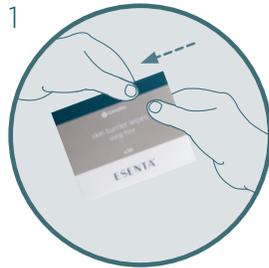


### GETTING STARTED

Clean and dry the skin. Tear sachet to open and unfold wipe.

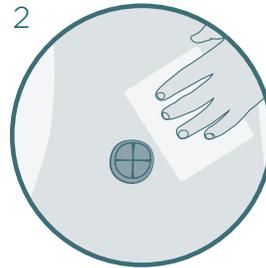

### EASY TO APPLY

Gently wipe the area to be protected and wait until dry.

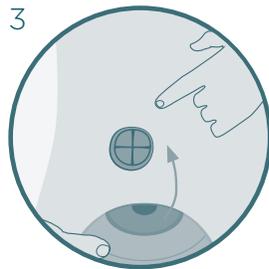

### CHECK IF THE SKIN IS DRY

Ensure the area is completely dry before applying adhesive products such as ostomy appliances or dressings.



### REAPPLY

ESENTA™ Skin Barrier should be replaced at every dressing or appliance change to maintain complete protection. Reapplication is recommended every 12-72 hours, dependent upon skin surface conditions. ESENTA™ Barrier will not cause a build-up of barrier film on the skin surface.



Learn more at [convatec.com](https://www.convatec.com) or contact us at 1-800-422-8811 or [cic@convatec.com](mailto:cic@convatec.com).



See package insert for complete instructions for use.

