



ESENTA™ Sting Free Adhesive Remover Spray

Easy to use: follow the below steps

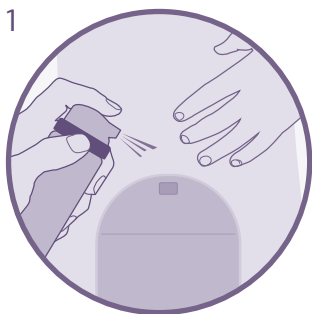


Getting started

Spray around the edge of the adhesive and wait a few seconds. You can spray at any angle.

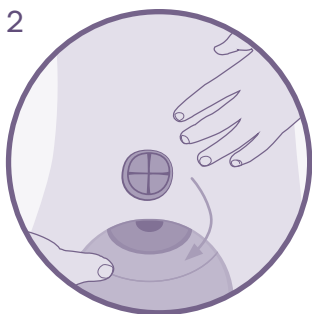

Easy removal

Gently peel adhesive away from the skin using additional ESENTA™ adhesive remover spray to ease removal, if necessary.

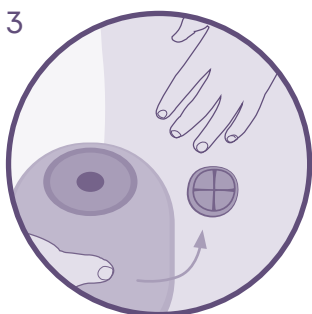

Dries in seconds

Once dried ESENTA™ adhesive remover will not affect adhesion of further dressings or appliances.

See package insert for complete instructions for use.

ESENTA™



convatec
— forever caring —


ESENTA™ Sting Free Adhesive Remover Wipes

Easy to use: follow the below steps




Getting started

Tear sachet to open and unfold wipe.

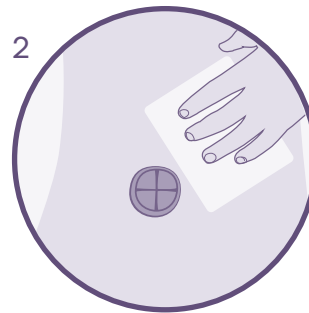

Easy removal

Rub the wipe over adhesive residues to remove.

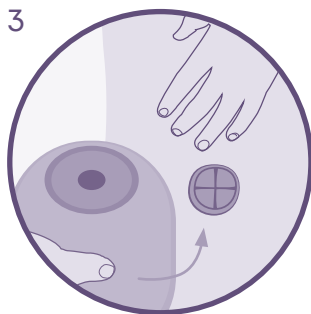

Dries in seconds

Once dried ESENTA™ adhesive remover will not affect adhesion of further dressings or appliances.



For more information, please call our Customer Relations Center (Registered Nurses on staff) at 1-800-465-6302, Monday through Friday, 8:00 AM to 6:00 PM (EST), or visit our Web Site at convatec.ca

See package insert for complete instructions for use.

©2023 Convatec Inc. ®/™ all the trademarks are the property of Convatec group companies.
AP-54646-CAN-ENG
V1826EN