




How to use ConvaTec Little Ones® Pediatric pouching system with Adhesive Coupling Technology™

Begin by cleansing the skin around your child's stoma with a mild non-oily soap and warm water. If you prefer to use commercial wet wipes on the skin around the stoma, choose a brand that does not contain oils or lotions, since they make it difficult for the pouching system to stick. The skin should be free of any solvent or greasy substances. Pat dry thoroughly.

Skin Barrier Preparation*

1. Measure the stoma size from the base of the stoma with the measuring guide provided. Trace the stoma outline you have selected onto the release paper on the skin barrier. The small starter hole should be in the center of your tracing. (Gloves may not be necessary; consult with your healthcare professional).
2. Cut the center hole the same size as the stoma. Cut along the tracing with curved, short-blade scissors.
3. Remove the release paper on the skin barrier.

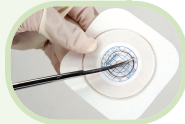

Application of Skin Barrier*

4. Gently place the skin barrier in place around the stoma holding the skin barrier for 30 seconds to ensure a snug fit around the stoma.
5. If you are using a flexible skin barrier (taped border), you can remove the paper on one side of the skin barrier and smooth onto the skin. Repeat with the other side, making sure the tape is smooth and wrinkle free.



Attachment of Pouch to Skin Barrier*

6. If you are using a drainable pouch, prior to applying the pouch it is recommended that you close the InvisiClose® clipless tail closure system by folding upwards and pressing the interlocking closures together until you feel them lock into place. A “click” indicates a secure closure.
7. Make sure the inner surfaces of the pouch are separated and that the pouch contains a small amount of air.
8. Remove the release paper from the ring on the pouch.
9. Fold the top half of the wafer towards yourself so that you can see the application around the stoma and then roll your fingers over the upper portion to secure the pouch. Always look beneath the top of the pouch at the adhesive to ensure the pouch to skin barrier adhesion is secure.



Pouch Repositioning and Removal*

To reposition the pouch, peel the pouch from the clear square “landing zone” by holding the tab on the pouch ring with one hand and the end of the landing zone with the other hand. Then, peel the pouch away in a rolling motion and reposition as needed.

To remove the old pouch and skin barrier, have a wet paper towel or soft wash cloth ready to clean the skin. Hold the edge of the pouch with one hand and push down gently on the skin with your other hand, to release the skin from the pouch and the skin barrier. Go slowly and gently, removing the skin barrier and pouch together.

Discard as necessary.



*See package insert for complete instructions for use.



SUPPORT

For more information on this product and others, call our Customer Relations Center (Registered Nurses on staff) at 1 800 465-6302, Monday through Friday, 8:00 AM to 6:00 PM (EST), or visit our Web Site at www.convatec.ca



Remember to empty your child's pouch when it is one-third full. The use of soaps or cleaners in the pouch is not recommended. If desired, rinse only with tepid water.

Convatec
Little Ones®