






# RLS GLOBAL

## ChloraSolv® INSTRUCTIONS FOR USE

<b>EN</b>	<b>Instructions for use</b>	<b>4</b>
<b>DE</b>	<b>Gebrauchsanweisungen</b>	<b>6</b>
<b>ES</b>	<b>Instrucciones de uso</b>	<b>8</b>
<b>FR</b>	<b>Mode d'emploi</b>	<b>10</b>
<b>IT</b>	<b>Istruzioni per l'uso</b>	<b>12</b>
<b>SV</b>	<b>Bruksanvisning</b>	<b>14</b>
<b>NO</b>	<b>Bruksanvisning</b>	<b>16</b>
<b>DA</b>	<b>Brugsanvisning</b>	<b>18</b>
<b>NL</b>	<b>Instructies voor gebruik</b>	<b>20</b>
<b>FI</b>	<b>Käyttöohje</b>	<b>22</b>
<b>Symbols</b>		<b>24-27</b>

**CE** 0477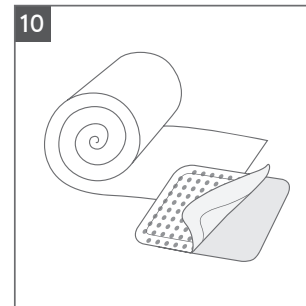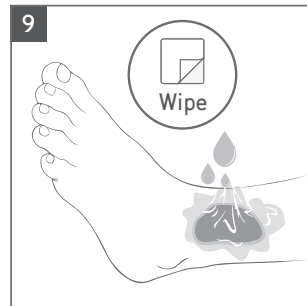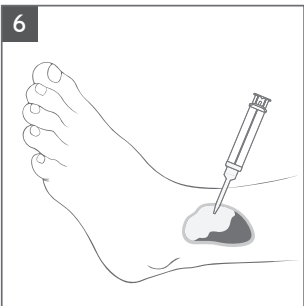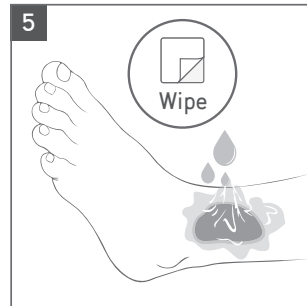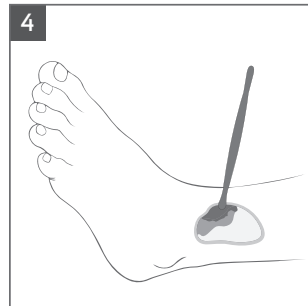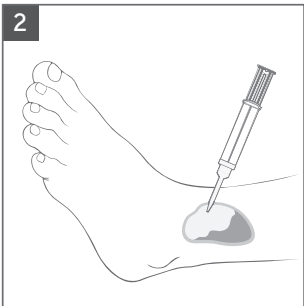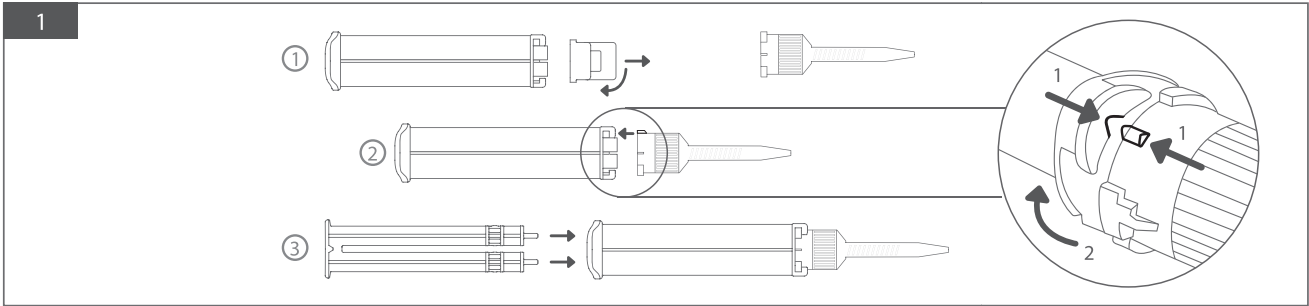

## EN – ChloraSolv®

### INSTRUCTIONS FOR USE

#### ChloraSolv® WOUND DEBRIDEMENT GEL

ChloraSolv® shall be used under the supervision of a qualified health care professional.

#### PRODUCT DESCRIPTION

ChloraSolv® is a wound bed preparation gel intended for cleansing and debridement/ de-sloughing of the wound.

The gel provides a moist alkaline pH environment that helps loosen up and dissolve necrotic tissue while delivering sodium hypochlorite. Sodium hypochlorite has shown *in vitro* anti-microbial activity and reduction of bacteria (*Staphylococcus aureus* and *Pseudomonas aeruginosa*) in biofilms *in vitro*. The use of this product does not negate the need for the use of topical/ systemic anti-infective therapy, as required.

ChloraSolv® makes it easy to mechanically remove devitalized tissue, pus and dirt with a blunt instrument and/or washing and wiping off the wound bed.

ChloraSolv® consists of two solutions provided in a double syringe with a cap closure. The solutions are mixed to form a gel at the time of use. Each package consists of a double syringe, a plunger and a mixer application tip. The mixing of the solutions forms an opaque gel, with slight odour of chlorine. The mixed gel is applied to the wound with the mixer application tip. After application, a blunt instrument is used to help in the mechanical removal of debris and foreign material from the application site.

#### INDICATIONS FOR USE

ChloraSolv® is intended to be used in adult patients with chronic leg ulcers and diabetic foot ulcers in need of debridement.

ChloraSolv® is intended for single patient single use.

### INSTRUCTIONS FOR USE

Up to 2 products can be used per patient and treatment

- 1 Take the double syringe out of the aluminium pouch. Hold the syringe with the opening upwards and remove the cap. Keep the syringe upright and mount the brown mixer element. Apply the plunger in the two barrels.
- 2 Press the plunger downwards and apply a thin layer of the mixed gel directly to the wound bed. The gel should cover the wound completely when applied.
- 3 Leave the gel on the wound for 2–5 minutes.
- 4 Remove loosened necrotic tissue, using a gentle scraping action with a blunt instrument.
- 5 Rinse the wound area with water or isotonic saline solution and wipe dry.
- 6 Finish the procedure with a second application of the gel directly onto the wound area to completely clean the wound bed. The product should cover the wound area completely when applied.
- 7 Leave the gel on the wound for 2–5 minutes.
- 8 Remove loosened necrotic tissue, using a gentle scraping action with a blunt instrument.
- 9 Rinse the wound area with water or isotonic saline solution and wipe dry.
- 10 Protect the wound with a bandage or dressing appropriate for the state of the wound.

Repeat the procedure 1–2 times/week until no more necrotic tissue is present in the wound.

If required, the treatment procedure may be continued for a maximum of 24 weeks.

### INGREDIENTS

**Gel:** opaque viscous solution consisting of water, carboxymethyl cellulose, sodium chloride, titanium dioxide and amino acids (glutamic acid, leucine, lysine) with alkaline pH.

**Sodium hypochlorite 0.9 %:** clear water-based liquid with alkaline pH.

### CONTRAINDICATIONS

ChloraSolv® has not been clinically studied on the following patient populations and are therefore contraindicated:

- Patients under the age of 18
- Pregnant and breastfeeding women
- Renal insufficiency defined as serum creatinine >200 µmol/L
- Macroangiopathy expected to demand vascular intervention or percutaneous angioplasty performed less than 3 months before start of treatment; toe pressure <30mm Hg
- Kidney or pancreas transplanted
- Patient under cortisone treatment >60mg/day
- Patient treated with chemotherapy or any other immune-inhibiting drugs 12 months before start of treatment
- Other identified on-going pathologies in the area of wound, such as cancer

### PRECAUTIONS

ChloraSolv® shall not be used if the patient is suspected to be allergic to any of the ingredients.

ChloraSolv® shall not be used together with chlorhexidine.

In case of contact with the eyes, rinse carefully with plenty of water for several minutes.

The product is single use. If re-used there is risk for cross-contamination.

Do not use if the aluminum pouch or the syringe is damaged.

Any serious incident that has occurred in relation to ChloraSolv® should be reported to RLS Global and the competent authority in your country.

### HANDLING, STORAGE AND DISPOSAL

Keep the product refrigerated (2–8°C) in its original packaging.

Can stain clothing.

Disposal should be handled in accordance to local regulations and instructions.

### SHELF-LIFE

Use prior to expiration date noted on the package.

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RLS Global AB