

# How to Use

## Sillesse™ Sting Free Skin Barrier Spray



### Getting Started:

Shake well before use. Clean and thoroughly dry the skin.



### Spray:

Hold approximately 4" from the area to be protected and apply an even coating in a sweeping motion.



### Let Dry:

Make sure Sillesse™ Skin Barrier Spray is completely dry before applying adhesive dressing or appliance. Sillesse™ Skin Barrier Spray can be used at every dressing/appliance change.

**TIP:** Try Niltac™ Sting Free Adhesive Releaser Spray or Remover Wipes to help clean the skin!

Product Code	Product Description	Product Size
420790	Sillesse™ Sting Free Skin Barrier Spray (50ml)	One 50ml Spray Can

# How to Use

## Sillesse™ Sting Free Skin Barrier Wipes



### Getting Started:

Clean and thoroughly dry the skin.



### Open:

Tear sachet to open and unfold wipe.



### Wipe:

Gently wipe the area to be protected and wait a few seconds until dry. You are ready to apply your system.

**TIP:** Try Niltac™ Sting Free Adhesive Releaser Spray or Remover Wipes to help clean the skin!

Product Code	Product Description	Product Size
420789	Sillesse™ Sting Free Skin Barrier Wipes (30 wipes)	Box of 30 wipes



For more information, please call our Customer Relations Center (Registered Nurses on staff) at **1-800-465-6302**, Monday through Friday, 8:00 AM to 6:00 PM (EST), or visit our Website at [www.convatec.ca](http://www.convatec.ca)