

ConvaTec Ostomy Care Accessories...



Discover The ConvaTec Complete Care Team

And discover the many ways our products can enhance quality of life!

■ AllKare® Adhesive Remover Wipes



■ AllKare® Protective Barrier Wipes



■ Eakin Cohesive®
Moldable Seal

ConvaTec Ostomy Care Accessories



■ Stomahesive®
Paste



■ Stomahesive®
Protective Powder

This special group of products is designed to work together to:

- Create a great foundation for the Ostomy Appliance
- Protect Skin
- Improve Security
- Extend Wear Time

Making A Difference in People's Lives



Everyone Deserves the Best

Grandchildren, family reunions, precious time spent with the ones we love... Ostomy patients want the most out of life. Being proactive to maintain skin health and pouch security can make all the difference.

Something as little as using the Eakin Cohesive® Seal to feel more secure, or an Allkare® Adhesive Remover Wipe to properly clean the skin are easy and practical approaches to taking control of life.

**See how ConvaTec Accessories
can make a difference.**

Eakin Cohesive® Seals

Our most **versatile skin protection** tool!

- Can be molded and customized to improve fit.
- Creates a level and leak resistant base.
- Fills uneven skin surfaces, including waves and ripples.
- Also absorbs moisture and forms a gel for long-lasting protection between skin and the ostomy system.



Stomahesive® Paste

Minimizes the risk of **leakage** and **protects the skin** by filling in uneven surfaces!



- A better fit means longer wear time.
- Use **Stomahesive®** Paste to create a custom seal around the stoma.
- Fills in voids and skin folds.
- Protects the skin to minimize irritation.
- Simply apply **Stomahesive®** Paste to skin or wafer prior to wafer application.





AllKare® Adhesive Remover Wipe

Use every time to promote **healthy skin!**

- Removing an Ostomy appliance can be hard on the skin.
- Use **AllKare®** Adhesive Remover Wipes to aid in the removal of skin barriers, wafers and tapes.
- Helps to completely and gently remove adhesives from the skin, allowing each pouch application to start with clean, residue-free skin.
- Contains emollients that leave skin feeling soft and supple.
- ISO – alcohol-free.



AllKare® Protective Barrier Wipe

Fast, easy to use protection!

- Apply to the skin before the wafer to protect the skin around the stoma.
- Forms a thin, invisible, non-water soluble barrier between the skin and the wafer.
- The **AllKare®** barrier lets a wafer or **Eakin Cohesive®** seal stick better.
- For a protective AND absorbent barrier, use **AllKare®** Protective Barrier wipes with **Stomahesive®** Powder.




Stomahesive® Protective Powder

When irritated skin needs
extra **protection!**

- Moisture can be harmful, **Stomahesive®** Protective Powder absorbs moisture from irritated skin to promote healing.
- Use on weepy areas to create a dry surface.
- Use every day to keep skin dry and healthy.
- For extra protection, **Stomahesive®** Protective Powder can be blotted with **AllKare®** Protective Barrier Wipe to create a protective and absorbent barrier.



Accessories

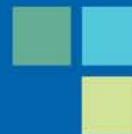

ITEM	CODE	DESCRIPTION	COUNT	HCPCS
AllKare® Protective Barrier	037439	Skin barrier wipe	50 count	A5120
	037444	Skin barrier wipe	100 count	A5120
AllKare® Adhesive Remover Wipe	037436	Adhesive remover wipe	50 count	A4365
	037443	Adhesive remover wipe	100 count	A4365
Stomahesive® Paste	183910	Paste sealer, 2 oz tube	1 tube	A4406
Stomahesive® Protective Powder	025510	Protective powder, 1oz bottle	1 bottle	A4371
Stomahesive® Skin Barrier	021712	4"x 4" skin barrier wafer	5 count	A4362
	021715	8"x 8" skin barrier wafer	3 count	A5122
	021901	4"x 4" skin barrier wafer - STERILE	5 count	A4362
Stomahesive® Strips	025542	skin barrier strips	15 count	A4362
Eakin Cohesive® Seals	839001	Large 4" Seal	10 count	A4385
	839002	Small 2" Seal	20 count	A4385
Eakin Cohesive® Skin Barrier	839003	Large 4" x 8" skin barrier	5 count	A4385
	839004	Small 4" x 4" skin barrier	5 count	A4385
DuoLock® Curved Tail Closure	175652	Curved tail closure	10 count	A4363
ConvaTec Tail Closure	175635	Straight tail closure	10 count	A4363
Ostomy Appliance Belt	175507	Appliance belt adjustable up to 42"	1 count	A4367
ConvaTec Night Drainage Container Set	027060	Night drainage container, tubing, cover, adapter	1 count	A5102
ConvaTec Night Drainage Container Tubing	027062	Night drainage tubing	1 count	A4331



Making a Difference in People's Lives

To learn more, call **1-800-422-8811**

www.convatec.com

AllKare® & Stomahesive® are registered trademarks of E.R. Squibb and Sons, L.L.C.

Eakin Cohesive® is a registered trademark of T.G. Eakin Limited.

©2006 E.R. Squibb & Sons, L.L.C. June 2006 US-06-1258 Printed in U.S.A.

