YpsoPump®
Inset

YpsoPump® Inset

Infusion Set

Freedom. Confidence. With Ypsomed.

YPSOMED
SELF CARE SOLUTIONS

en	Do not re-use
cs	Pro jednorázové použití
da	Må ikke genbruges
de	Nicht wiederverwenden
es	No reutilizar
fi	Älä käyttää uudelleen
fr	Ne pas réutiliser
hr	Nije za višekratnu uporabu
hu	Újrafelhasználni tilos
it	Non riutilizzare
nl	Niet opnieuw gebruiken
no	Må ikke brukes om igjen
pl	Nie używać ponownie
pt	Não reaproveitar
sk	Nepoužívajte opakovane
sl	Samo za enkratno uporabo
sv	Får ej återanvändas
zh	不可重新使用
ar	لا يُعاد استعماله
he	לא לשימוש חוזר

en	Use-by (Expiry date) YYYY-MM-DD
cs	Použijte do (datum použitelnosti) RRRR.MM.DD
da	Anvendes inden (udløbsdato) ÅÅÅÅ-MM-DD
de	Verwendbar bis (Verfallsdatum) JJJJ-MM-TT
es	Fecha de caducidad AAAA-MM-DD
fi	Viimeinen käyttöpäivä VVVV-KK-PP
fr	Utiliser jusqu'au (date de péremption) AAAA-MM-JJ
hr	Upotrijebiti do (rok valjanosti) GGGG-MM-DD
hu	Felhasználható (lejárati idő) ÉÉÉÉ-HH-NN
it	Utilizzare entro (data di scadenza) AAAA-MM-GG
nl	Te gebruiken tot (vervaldatum) JJJJ-MM-DD
no	Bruk innen (utløpsdato) ÅÅÅÅ-MM-DD
pl	Należy wykorzystać do (data ważności) RRRR-MM-DD
pt	Utilizar até (Prazo de validade) AAAA-MM-DD
sk	Dátum spotreby (dátum expirácie) RRRR-MM-DD
sl	Uporabiti do (rok uporabe) LLLL-MM-DD
sv	Utgångsdatum (ÅÅÅÅ-MM-DD)
zh	使用截止日期 (过期日期) 年-月-日
ar	يُستخدم قبل (تاريخ انتهاء الصلاحية) يوم/شهر/سنة
he	לשימוש עד (תאריך תפוגה) YYYY-MM-DD

en	Sterilized using ethylene oxide
cs	Sterilizováno pomocí ethylenoxidu
da	Steriliseret med ethylenoxid
de	Mit Ethylenoxid sterilisiert
es	Esterilizado con óxido de etileno
fi	Steriloitu etyleenioksidilla
fr	Stérilisé à l'oxyde d'éthylène
hr	Sterilizirano etilen oksidom
hu	Etilénoxiddal sterilizálva
it	Sterilizzato con ossido di etilene
nl	Gesteriliseerd met ethyleenoxide
no	Sterilisert med etylenoksid
pl	Sterylizowany przy użyciu tlenku etylenu
pt	Esterilizado com óxido de etileno
sk	Sterilizované etylenoxidom
sl	Sterilizirano z etilen oksidom
sv	Steriliserad med etylenoxid
zh	已使用环氧乙烷灭菌
ar	عُقم باستخدام أكسيد الإيثيلين
he	עוקר באמצעות אתילן אוקסיד

en	Single sterile barrier system
cs	Systém jednovrstvé sterilní bariéry
da	Enkelt sterilt barriersystem
de	Einfaches Sterilbarriersystem
es	Sistema de barrera esteril simple
fi	Yksittäinen steriili estojärjestelmä
fr	Système de barrière stérile simple
hr	Sustav s jednostrukom sterilnom barijerom
hu	Egyszeres steril gátrendszer
it	Sistema di barriera sterile singola
nl	Enkel steriel barrièresysteem
no	System med én steril barriere
pl	System pojedynczej bariery sterylnej
pt	Sistema de barreira esteril simples
sk	Jednoduchý sterilný bariérový systém
sl	Enojni sterilni pregradni sistem
sv	Enkelt sterilt barriärsystem
zh	单次无菌屏障系统
ar	نظام حاجز معقم واحد
he	מערכת מחסום מעוקר יחיד

en	Reference/model number
cs	Referenční číslo/číslo modelu
da	Reference-/modelnummer
de	Referenz-/Artikelnummer
es	Referencia/número de modelo
fi	Tuotenumero
fr	Référence catalogue
hr	Referentni broj / broj modela
hu	Referencia/modellszám
it	Numero rif./Numero modello
nl	Referentie-/modelnummer
no	Referanse-/modellnummer
pl	Numer ref./numer modelu
pt	Referência/número do modelo
sk	Referenčné číslo/číslo modelu
sl	Referenčna številka/številka modela
sv	Referens-/modellnummer
zh	参考 / 型号
ar	المرجع/رقم الموديل
he	סימוכין/מספר דגם

en	Global Trade Item Number
cs	Globální číslo obchodní položky
da	Global Trade Item Number
de	Globale Handelsartikelnummer
es	Número Mundial de Artículo Comercial
fi	Global Trade -erän numero
fr	Code article international
hr	Globalni broj trgovine
hu	Globális kereskedelmi szám
it	Global Trade Item Number
nl	Global Trade Item Number
no	Global Trade Item Number
pl	Globalny numer jednostki handlowej
pt	Número global de item comercial
sk	Číslo globálneho obchodného čísla
sl	Globalna trgovinska številka izdelka
sv	Global Trade Item Number
zh	全球贸易项目编号
ar	معرف رقم وحدة المنتج العالمي
he	מספר פריט מסחרי גלובלי

en	Lot number
cs	Šarže
da	Lotnummer
de	Chargennummer
es	Número de lote
fi	Eränumero
fr	Numéro de lot
hr	Broj serije
hu	Sorozatszám
it	Numero lotto
nl	Batchnummer
no	Partinummer
pl	Numer serii
pt	Número do lote
sk	Číslo šarže
sl	Serijska številka
sv	Lotnummer
zh	批号
ar	رقم الدفعة
he	מספר אצווה

en	Manufacturer
cs	Výrobce
da	Producent
de	Hersteller
es	Fabricante
fi	Valmistaja
fr	Fabricant
hr	Proizvodač
hu	Gyártó
it	Produttore
nl	Producent
no	Produsent
pl	Producent
pt	Fabricante
sk	Výrobca
sl	Proizvajalec
sv	Tillverkare
zh	制造商
ar	الشركة المصنعة
he	יצרן

en	Date of Manufacture YYYY-MM-DD
cs	Datum výroby RRRR.MM.DD
da	Fremstillingsdato ÅÅÅÅ-MM-DD
de	Herstellungsdatum JJJJ-MM-TT
es	Fecha de fabricación AAAA-MM-DD
fi	Valmistuspäivämäärä VVVV-KK-PP
fr	Date de fabrication AAAA-MM-JJ
hr	Datum proizvodnje GGGG-MM-DD
hu	Gyártási idő: ÉÉÉÉ-HH-NN
it	Data di fabbricazione AAAA-MM-GG
nl	Productiedatum JJJJ-MM-DD
no	Produksjonsdato ÅÅÅÅ-MM-DD
pl	Data produkcji RRRR-MM-DD
pt	Data de fabrico AAAA-MM-DD
sk	Dátum výroby RRRR-MM-DD
sl	Datum izdelave LLLL-MM-DD
sv	Tillverkningsdatum (ÅÅÅÅ-MM-DD)
zh	制造日期年-月-日
ar	تاريخ التصنيع يوم/شهر/سنة
he	תאריך הייצור YYYY-MM-DD

en	Distributor
cs	Distributor
da	Distributør
de	Distributor
es	Distribuidor
fi	Jälleenmyyjä
fr	Distributeur
hr	Distributer
hu	Forgalmazó
it	Distributore
nl	Distributeur
no	Distribuer
pl	Dystrybutor
pt	Distribuidor
sk	Distribútor
sl	Distributer
sv	Distributör
zh	分销商
ar	الموزع
he	מפיץ

en	Consult instructions for use
cs	Prostudujte si návod k použití
da	Se venligst brugervejledningen
de	Gebrauchsanweisung beachten
es	Consulte las instrucciones de uso
fi	Katso käyttöohjeita
fr	Consulter le mode d'emploi
hr	Pogledajte upute za uporabu
hu	Lásd a használati utasítást
it	Consultare le istruzioni per l'uso
nl	Raadpleeg gebruiksaanwijzing
no	Følg bruksanvisningen
pl	Przestrzegać instrukcji obsługi
pt	Consultar as instruções de utilização
sk	Pozrite si návod na používanie
sl	Upoštevajte navodila za uporabo
sv	Följ bruksanvisningen
zh	参见使用说明
ar	اتبع إرشادات الاستعمال
he	יש לעיין בהוראות השימוש

en	Caution: see instructions for use
cs	Upozornění: přečtěte si návod k použití.
da	Forsigtig: Se brugsanvisningen
de	Achtung: Gebrauchsanweisung beachten
es	Atención: ver las instrucciones de uso
fi	Huomio: katso käyttöohje
fr	Attention: Consulter les instructions d'utilisation
hr	Pozor: pogledati upute za uporabu
hu	Figyelem! Olvassa el a használati utasítást
it	Attenzione: consultare le istruzioni per l'uso
nl	Let op: raadpleeg de gebruiksaanwijzing
no	Forsiktig: Se bruksanvisningen
pl	Uwaga: należy zapoznać się z instrukcją obsługi
pt	Cuidado: consultar as instruções de utilização
sk	Pozor: pozri návod na použitie
sl	Pozor: glejte navodila za uporabo
sv	Se upp! Läs bruksanvisningen
zh	注意: 参阅使用说明书
ar	تنبيه: انظر تعليمات الاستخدام
he	זהירות: עיין בהוראות השימוש

en	Medical device
cs	Zdravotnický prostředek
da	Medicinsk udstyr
de	Medizinprodukt
es	Producto sanitario
fi	Lääkinnällinen laite
fr	Dispositif médical
hr	Medicinski proizvod
hu	Gyógyászati eszköz
it	Dispositivo medico
nl	Medisch hulpmiddel
no	Medisinsk utstyr
pl	Urządzenie medyczne
pt	Dispositivo médico
sk	Zdravotnícka pomôcka
sl	Medicinski pripomoček
sv	Medicinteknisk produkt
zh	医疗器械
ar	جهاز طبي
he	מכשור רפואי

en	Non-pyrogenic
cs	Nepyrogní
da	Non-pyrogen
de	Pyrogenfrei
es	Apirógeno
fi	Pyrogeeniton
fr	Apyrogène
hr	Nepirogeno
hu	Nem pirogén
it	Apirogeno
nl	Niet-pyrogeen
no	Ikke-pyrogen
pl	Niepirogenny
pt	Não-pirógeno
sk	Nepyrognéne
sl	Apirogeno
sv	Pyrogenfri
zh	无热原
ar	لا يؤدي للإصابة بالحمى
he	לא פירוגני

en	Keep dry
cs	Uchovávejte v suchu
da	Opbevares tørt
de	Trocken aufbewahren
es	Mantener seco
fi	Pidettävä kuivana
fr	Conserver au sec
hr	Čuvati na suhom mjestu
hu	Tartsa szárazon
it	Conservare in un luogo asciutto
nl	Doorg bewaren
no	Oppbevares tørt
pl	Chronić przed wilgocią
pt	Conservar seco
sk	Uchovávať v suchu
sl	Hraniti na suhem
sv	Förvaras torrt
zh	保持干燥
ar	احفظ المنتج جافاً
he	שמור יבש

en	Do not use if package is damaged
cs	Nepoužívejte, pokud je obal poškozený
da	Må ikke anvendes, hvis emballagen er beskadiget
de	Nicht verwenden, wenn Verpackung beschädigt ist
es	No utilizar si el embalaje está dañado
fi	Ei saa käyttää, jos pakkaus on vaurioitunut
fr	Ne pas utiliser si l'emballage est détérioré
hr	Ne koristiti ako je pakiranje oštećeno
hu	Ne használja, ha a csomagolás megsérült.
it	Non utilizzare se la confezione è danneggiata
nl	Niet gebruiken als verpakking beschadigd is
no	Skal ikke brukes hvis emballasjen er skadet
pl	Nie używać, jeśli opakowanie jest uszkodzone
pt	Não utilizar se a embalagem estiver danificada
sk	Nepoužite, ak je obal poškodený.
sl	Ne uporabljajte, če je ovojnina poškodovana
sv	Använd inte produkten om förpackningen är skadad
zh	若包装破损不得使用
ar	لا تستخدم المنتج في حالة تلف التغليف
he	אין להשתמש אם האריזה ניזוקה

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English

Indications for use:

The YpsoPump® Inset infusion set is indicated for the subcutaneous infusion of insulin administered by the external infusion pump YpsoPump®. The infusion set is indicated for single use.

Intended Use

The infusion set is indicated for subcutaneous infusion of insulin in the treatment of diabetes. The infusion set has been tested to be compatible with insulin approved for subcutaneous infusion.

Description

YpsoPump® Inset is an infusion device which combines an infusion set with an aid for infusion (serter). As the infusion set is an integrated part of the sserter, YpsoPump® Inset is delivered assembled and ready for use as a single, sterile unit.

Contraindications

YpsoPump® Inset is neither intended nor indicated for use in intravenous infusion of blood or blood products.

WARNINGS

- Carefully examine the instructions for use before inserting YpsoPump® Inset. Failure to follow instructions may result in pain or injury.
- YpsoPump® Inset is only sterile and non-pyrogenic if the insertion device is undamaged and unopened. Do not use if the insertion device is already open or has been damaged. Ensure sterility by checking if the sterile paper and tamper-proof seal are undamaged.
- When using YpsoPump® Inset for the first time, do so in the presence of a healthcare provider. Since different individuals have varying amounts of subcutaneous layer, ensure that an appropriate cannula length is chosen to fit your needs.
- Inaccurate medication delivery, site irritation or infection may result from improper insertion or maintenance of your infusion site.
- Do not re-insert the introducer needle into the infusion set. This could cause a tearing or damage of the soft cannula and may lead to unpredictable medication flow.
- Reuse of the infusion set may cause site irritation or infection and may lead to inaccurate medication delivery.

- Be sure that the needle guard is removed before insertion is made.
- If the soft cannula bends during insertion, apply a new YpsoPump® Inset immediately at a new site.

PRECAUTIONS

- Never try to fill or free a clogged tubing while the infusion set is inserted. This may result in unpredictable medication flow.
- When unwinding, release the tubing with caution as a hard pull of the tubing can result in damage to the infusion device. Ensure that the infusion set is properly in place in the sserter when the tubing is fully released – see picture 8.
- Do not leave air in the infusion set. Make sure to fill the tubing completely. Consult pump manufacturer's instructions for use.
- YpsoPump® Inset is a single use device and should be disposed of immediately after use. Do not clean or re-sterilize.

- Do not in any way put disinfectants, perfumes, deodorants, cosmetics or other products containing alcohol or disinfectants in contact with the connector or the tubing. This may impact the integrity of the infusion set.
- Replace the infusion set if the adhesive tape becomes loose or is displaced from its original site. Since the cannula is soft, it will not cause any pain if it becomes dislodged, and this may take place without notice. The soft cannula must always be completely inserted to receive the full amount of medication.
- Replace the infusion set every two to three days, or per your healthcare provider's instructions.
- If your insertion site becomes irritated or infected, replace the infusion set and use a new site until the first site has healed.
- Wash your hands before disconnecting and reconnecting the infusion set. Consult your healthcare provider on how to compensate for missed medication when disconnected.
- Never point a loaded insertion device towards any body part, where insertion is not desired.

RECOMMENDATIONS

- Check your blood glucose level one to three hours after insertion of the YpsoPump® Inset. Measure your blood glucose on a regular basis. Discuss this with your healthcare provider.
- Do not change your infusion set just prior to bedtime, unless blood glucose can be checked one to three hours after insertion.
- If your blood glucose level becomes unexplainably high or an occlusion alarm occurs, check for clogs and leaks. If in doubt, change your infusion set since the soft cannula could be dislodged or partially clogged. Discuss a plan to bring down your blood glucose level with your healthcare provider should any of these problems arise. Test your blood glucose level to make sure the problem is corrected.
- Carefully monitor your blood glucose level when disconnected and after reconnecting.
- If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

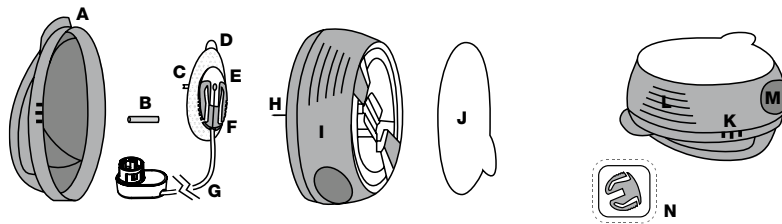
Storage and Disposal

- Store infusion sets in a cool, dry location at room temperature. Do not store infusion sets in direct sunlight, or in high humidity.
- Store and handle insulin according to the manufacturer's instructions.
- Put the lid back on the insertion device and dispose in a proper sharps container according to local laws, to avoid risk of needle stick.
- Dispose of a used infusion set according to local regulations for biohazardous waste.

English

Preparation and Insertion

Read all instructions carefully before using the YpsoPump® Inset infusion set. For information about pump therapy, possible errors, and potential risks and their causes, consult your pump manufacturer's instructions for use before connecting YpsoPump® Inset. When using YpsoPump® Inset follow proper hygiene procedures. Please see the corresponding illustrations on page 88–90.

Contents

- A** Lid
- B** Needle guard
- C** Soft cannula
- D** Adhesive tape
- E** Cannula housing
- [F]** Connector needle
- [G]** Tubing
- [H]** Introducer needle
- [I]** Insertion device
- [J]** Sterile paper

Important note!

- [K]** Three raised dots – press here when lifting the lid off (see Figure 6)
- [L]** Lined indentations – press here when loading YpsoPump® Inset (see Figure 10)
- [M]** Round Indentations – press here simultaneously when releasing YpsoPump® Inset (see Figure 12)
- [N]** Disconnect cover (see Figure B)

INSTRUCTIONS

Unfold the cover of this booklet to view images that correspond to the steps below.

- 1** Wash your hands before inserting YpsoPump® Inset.
- 2** Suggested insertion sites. Choose an insertion site as recommended by your healthcare provider. Do not use an area right next to your previous insertion site.
- 3** Clean the infusion site with a disinfectant as directed by your healthcare provider. Be sure the area is air dry before inserting YpsoPump® Inset.
- 4** Pull the red tape to remove the protective seal.
- 5** Remove the sterile paper.
- 6** Press the three raised dots on each side of the lid firmly with one hand and lift up the lid with the other hand.

CAUTION: Be careful not to bend or touch the YpsoPump® Inset introducer needle when preparing the inserter and infusion set.

Important note! If the infusion set is not securely placed in the insertion device with the needle pointing straight ahead prior to insertion, pain or minor injury may occur. To avoid this, please ensure that the following steps are taken gently so the infusion set is not loosened/removed by accident.

- 7 Unwinding the tubing:** Carefully twist the beginning of the tubing out of the slot. Now unwind the tubing from the insertion device by pulling it carefully upwards.

CAUTION: Do not touch the YpsoPump® Inset introducer needle when unwinding the tubing. Be careful not to pull too hard when the tubing is fully released. You could accidentally remove the infusion set from the introducer needle.

- 8 Before insertion:** Attach the adapter of the infusion set straight onto the cartridge compartment of the YpsoPump®. Push the adapter gently down to pierce the septum of the insulin cartridge. Turn the adapter in a clockwise direction until you feel it slide into lock position. Check visually that the adapter is attached firmly and aligned with the pump case. After connection, make sure air bubbles are removed by priming the tubing assembly according to the priming instructions of the YpsoPump®.

Priming volume for the tubing with adapter (U-100 insulin, volumes are approximative):

46 cm (18 inch):	10 units (0.10 ml)
60 cm (24 inch):	12 units (0.12 ml)
80 cm (31 inch):	14 units (0.14 ml)
110 cm (43 inch):	18 units (0.18 ml)

CAUTION: When priming the infusion set, hold YpsoPump® Inset with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.

- 9** Pull up gently to remove the adhesive backing paper.

- 10 Prepare the inserter:** Place fingers on lined indentations. Press lined indentations on each side. Pull spring until you hear a “click”.

- 11** Carefully remove the needle guard by gently twisting and then pulling it. Check that the soft cannula does not exceed the introducer needle.

CAUTION: Before positioning YpsoPump® Inset over the cleaned infusion site, place the tubing in the slot on the side of the inserter to ensure that the tubing is not caught underneath the infusion set during insertion.

- 12 Insert YpsoPump® Inset:** Position YpsoPump® Inset over the insertion site. Simultaneously press the round indentations on both sides of the insertion device to insert YpsoPump® Inset.

- 13** Push gently on the center of the inserter to secure the adhesive on the skin.

- 14** Remove the inserter and introducer needle by grasping the center and pulling it gently back. Gently massage the adhesive onto the skin. Fill the empty space in the cannula with the following amount of insulin:

U-100 insulin:

6 mm:	0.1 units (0.001 ml)
9 mm:	0.2 units (0.002 ml)

- 15** Push the lid back in place until you hear a “click.”

Disconnect

- A** YpsoPump® Inset allows you to temporarily disconnect your pump without having to change the infusion set. Hold the adhesive in place by placing a finger just in front of the cannula housing, and gently squeeze the sides of the connector. Pull the connector needle straight out from the cannula housing.
- B** Insert the disconnect cover into the cannula housing until you hear a “click”.

Reconnect

- C** If necessary, prime the infusion set until medication dispenses from the needle.
CAUTION: When priming the infusion set, hold YpsoPump® Inset with the needle pointing down.
- D** Remove the disconnect cover from the cannula housing by pressing the grips. Place a finger on the adhesive tape just in front of the cannula housing.
- E** Push the connector needle straight in until you hear a “click”.

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IT Ypsomed Italia S.r.l.
SV Ypsomed AB
UK Ypsomed Ltd.
Ypsomed AG