

OCTOBER 2020

# STOMA

## Product Guide

Shaping Lives with Complete  
Stoma Products, Care and Support



ConvaTec


Stoma Care



# Together, We Can Help SHAPE LIVES

Innovation begins by listening  
and working together.

ConvaTec is a global medical products and technologies company focused on therapies for the management of chronic conditions. Our portfolio of stoma products, me+™ patient support programme and collaboration with health care professionals help to improve the lives of the people we touch.

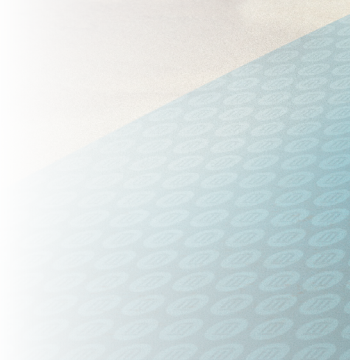







# QUICK REFERENCE GUIDE

## CONVATEC POPULAR PRODUCTS

FLEX CONVEX

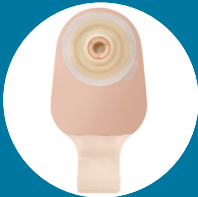

SOFT CONVEX

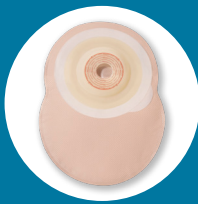

MOULDBABLE



## MY FAVOURITES

### Product Description

.....

.....

.....

.....

### Product Code

.....

.....

.....

.....



# TABLE OF CONTENTS

me+™ SUPPORT .....	1
Amcare™ Group.....	3
<b>GUIDE TO STOMA SOLUTIONS</b>	
• A Guide to ConvaTec Hydrocolloids .....	5
• ConvaTec Mouldable Technology™ .....	11
• The Characteristics of Convexity .....	15
<b>ONE-PIECE SOLUTION.....</b>	<b>7</b>
• Esteem™+ Flat.....	7
• Esteem™+ Flat Mouldable .....	13
• Esteem™+ Convex .....	15
<b>TWO-PIECE SOLUTIONS .....</b>	<b>23</b>
• Natura® & Natura®+ .....	25
• Esteem Synergy® & Esteem Synergy®+.....	35
• Stoma Caps .....	40
<b>PAEDIATRIC SOLUTIONS .....</b>	<b>41</b>
<b>ACCESSORIES .....</b>	<b>47</b>
<b>REFERENCES .....</b>	<b>53</b>



## THE **me+**<sup>TM</sup> PROGRAMME SUPPORTS PATIENTS

The **me+**<sup>TM</sup> programme is a FREE comprehensive support programme designed to complement clinical care by providing patients with support, inspiration, insights and ideas throughout their entire stoma journey.

### **me+**<sup>TM</sup> programme members can:

- ✓ Request free stoma samples
- ✓ Speak to our Specialist Stoma Nurse Advisor
- ✓ Speak to one of our qualified counsellors
- ✓ Connect to our **me+** community
- ✓ Receive ongoing lifestyle education, support and much more



*"I think **me+**<sup>TM</sup> is absolutely fantastic. Darren, who spoke to me was so caring and thoughtful. **me+** helped me with all sorts of hints and tips on hydration with a stoma (which I so desperately need), and the team are there to support you through everything."*

JENNIE, BLACKPOOL

\* Model portrayal for illustrative purposes only.



Our goal is to *partner with you and your patients to deliver best-in-class care* throughout the patient journey:

✓ **Pre-Surgery / Surgery**

- Access online resources at [meplus.convatec.co.uk](http://meplus.convatec.co.uk)
- Request me+ patient Welcome Kit and materials
- Enroll patients on the me+ programme
- Learn about the me+ recovery phased patient-centered movement and recovery programme developed to guide and aid patients recovery after stoma surgery

✓ **Weeks Immediately After Surgery**

- me+ Welcome email and support materials
- me+ recovery Welcome Kit, including Phase 1 and 2 exercise guide and diary
- Speak to our nurses and product specialist team

✓ **First Year and Beyond**

- Explore online resources and tools
- Get inspired by the me+ community
- Continue me+ recovery exercise programme into Phase 3
- Continue me+ recovery series exercises\*

---

**TO ENROLL YOUR PATIENTS on the me+ programme**  
call us on 0800 467 866 or visit [meplus.convatec.co.uk](http://meplus.convatec.co.uk)

---



\*The information in the series is not intended to provide medical advice. Patients and users should speak with their health care professional before engaging in physical activity following stoma surgery. In the event they experience pain or discomfort, they should stop the activity immediately and contact their health care professional.



Free Home  
Delivery\*



Register your patients at:  
[www.amcaregroup.co.uk/register](http://www.amcaregroup.co.uk/register)

Call your Amcare Specialist:  
**0800 88 50 50**





**Prescription  
Management**



**Product  
Customisation**



**Advice &  
Support**

## RELIABLE DELIVERY SERVICE WITH NATIONAL COVERAGE



Free prescription and home  
delivery service. Simple, reliable  
and discreet.

**Call or click  
to renew a prescription**



**56%**

of Amcare™ Group  
prescriptions are dispensed  
electronically which can  
avoid unnecessary  
trips to the GP



**90%**

of all calls are answered within  
our agreed service levels

**18  
seconds**

Average answer speed of  
our dedicated, professional  
team of customer care  
advisors readily available to  
discuss order requirements



**99%**

of deliveries are sent within  
**48 hours** of receipt of the  
prescription



**ConvaTec**

Stoma Care

# A Guide to ConvaTec HYDROCOLLOIDS

## STOMAHESIVE®

Designed for frequent removal whilst maintaining skin integrity, our unique Stomahesive® technology is a hydrocolloid adhesive that gently adheres to the skin, protecting it from the damaging effects of stoma effluent.

We also offer a **modified Stomahesive®** option, which features a touch of mineral oil that makes the baseplate more flexible during wear, and removal easier on the skin.

## DURAHESIVE®

Designed for a longer wear time, our unique Durahesive® technology is a hydrocolloid adhesive that swells up upon hydration to protect the stoma. This special effect called turtlenecking, helps create a more secure seal without harming the skin.

## ESTEEM™+ *Soft & Flex* CONVEX

Adhesive designed with a high initial tack, to aid in the initial attachment to the skin. The hydrocolloid barrier features a thin tapered edge to help improve flexibility and comfort, and a thicker center to help absorption around the stoma.<sup>1</sup>

## BASEPLATE COLLARS

*are available in two forms:*





### Designed for patients

- ✓ with all output types
- ✓ who have sensitive skin
- ✓ with frequent changes

### Designed for patients

- ✓ with semi-liquid/liquid output, including urine
- ✓ extended wear time
- ✓ helps maintain skin integrity

### Designed for patients

- ✓ with both solid and liquid output

**Microporous Collar:** Collar designed with good tack

**Hydrocolloid Collar:** Designed for those with sensitivity to other tape collars

# ONE-PIECE

## Solutions

### ESTEEM™+ BRAND MAKES UP CONVATEC'S ONE-PIECE SOLUTION *FLAT & CONVEX*

## ESTEEM™

The Esteem™ + brand is an all-in-one solution combining the baseplate and pouch in a single unit. Designed to be easy and convenient, it's one simple step to attach or remove. Available in multiple adhesive options, growing convexity options, and with ConvaTec's latest pouch technologies.

## ESTEEM™ + FLAT

*One Piece Solution with Flat Baseplates.*



 ESTEEM™+ POUCH

### Cut-To-Fit | Closed

- Modified Stomahesive®
- Quantity - 30

**Filter** (*Filter covers included*)

Cutting Range	Pouch Size	Opaque with Viewing Window	Opaque	Transparent
20-70mm	Small	421816		
20-70mm	Standard	421686	416701	416700
20-70mm	Large	421825		421831

## Pre-Cut | Closed | Small

Filter (Filter covers included)

Stoma Size	Opaque with Viewing Window
25mm	421817
30mm	421818
35mm	421819
40mm	421820

## Pre-Cut | Closed | Standard

Filter (Filter covers included)

Stoma Size	Opaque with Viewing Window	Opaque	Transparent
25mm	421687	416704	416703
30mm	421688	416707	416706
35mm		416710	416709
40mm	421822	416713	416712
50mm	421823	416715	
60mm	421824		

## Pre-Cut | Closed | Large

Filter (Filter covers included)

Stoma Size	Opaque with Viewing Window	Transparent
25mm	421826	421832
30mm	421827	421833
35mm	421828	421834
40mm	421829	421835
50mm	421830	421836





## Cut-To-Fit | Drainable | InvisiClose® Closure | Lock-it Pocket™ | Standard

- Modified Stomahesive®
- Quantity - 10

Filter (Filter covers included)

Cutting Range	Opaque with Viewing Window	Opaque	Transparent	Transparent No Filter
20-70mm	421858	416719	416718	416721

## Pre-Cut | Drainable | InvisiClose® Closure | Lock-it Pocket™ | Standard

Filter (Filter covers included)

Stoma Size	Opaque with Viewing Window	Opaque	Transparent
25mm	421859	416725	416724
30mm	421860	416729	416728
35mm	421861		
40mm	421862	416737	



## Cut-To-Fit | Drainable | Clipped Closure | Standard

- Modified Stomahesive®
- Pouch clip (2 per box)
- Quantity - 10

Filter (Filter covers included)

Stoma Size	Size	Opaque	Transparent
20-70mm	Small	421837	421847
20-70mm	Standard	421838	421848

## Pre-Cut | Drainable | Clipped Closure | Standard

Filter (Filter covers included)

Stoma Size	Opaque
25mm	421839
30mm	421840
35mm	421841
40mm	421842





Niltac™ fast acting spray makes bag changes so much more comfortable especially if you have sensitive skin.

JO, SWANSEA



FOR POUCH  
REMOVAL SUPPORT,  
SEE PAGE 48



ConvaTec

Stoma Care

# ConvaTec

# MOULDABLE TECHNOLOGY™

ConvaTec Mouldable Technology™ baseplates are designed to hug the stoma opening for a more custom and secure fit.

## Designed for patients


- ✓ who have large, round, oval or irregular shaped stomas
- ✓ with dexterity issues who find cutting challenging

## CLINICALLY PROVEN

to prevent and improve peristomal skin complications<sup>2</sup>

## Features and Benefits


- ✓ Reduced application steps – fewer steps than traditional cut-to-fit baseplates
- ✓ No cutting required
- ✓ Rebounding Memory Technology™ – flexible materials gently expand and contract with the stoma
- ✓ Turtlenecking – the adhesive gently swells when in contact with output & stool to help protect the skin and secure the seal

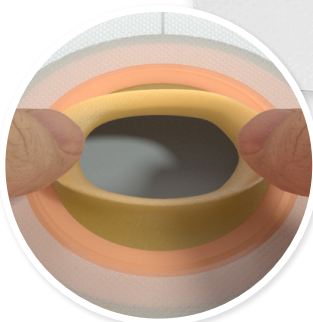

Roll, mould, and hold to the stoma's shape and size



Turtlenecks around the stoma to protect against leaks

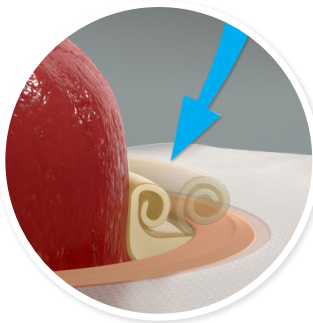

Rebounding Memory Technology™ - rolls back to achieve a custom fit

# CONVATEC MOULDABLE TECHNOLOGY™

One Piece Solution with Mouldable Baseplates.



My stoma has prolapsed in the past and it can change in size quite a lot. The mouldable adhesive moves with my stoma and does not cause any restrictions, or leave gaps where soreness could start to occur. I can go without worrying and eat without panicking, its been life changing.

CHARLIE, CORNWALL

95% of people who started on ConvaTec Mouldable Technology™ kept their skin healthy²



✓ ESTEEM™+ POUCH

## Mouldable | Closed | Standard

- Modified Stomahesive® & Durahesive®
- Quantity - 30

Filter (Filter covers included)

Stoma Size	Opaque with Viewing Window
20-30mm	413509
30-40mm	413510
40-50mm	413511



## DRAINABLE POUCH CLOSURE



### InvisiClose® Closure

Drainable pouch closure which provides an easy closure option that does not require a separate clip for added security.



### Lock-it Pocket™

Folds inside out to hide away the secured drainage closure – providing the flexibility of a drainable pouch, with the feel and comfort of a closed pouch.



### ✓ ESTEEM™+ POUCH

#### Mouldable | Drainable | InvisiClose® Closure | Lock-it Pocket™ | Standard

- Stomahesive® & Durahesive®
- Quantity - 10

**Filter** (*Filter covers included*)

Stoma Size	Opaque with Viewing Window	Transparent
20-30mm	413521	413515
30-40mm	413522	413516
40-50mm	413523	413517



# The Characteristics of CONVEXITY



## RIGIDITY

### Soft Convex

Typically used on any abdomen type.

### Flex Convex

Typically used on skin folds and creases.

### Firm/Rigid Convex

Typically used on soft or flaccid abdomen.


## DEPTH

Height of the convex plateau.

✓ Shallow: 4mm

✓ Deep: 7mm

✓ Rigid: 5mm

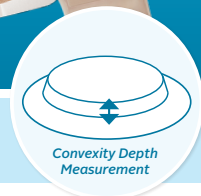

## PLATEAU WIDTH


Distance across the widest flat part of the outward convex curve.

### Why Plateau Width Matters





Successful convexity should support the stoma and peristomal skin to achieve a good seal around the stoma.<sup>3</sup>

✓ **Convexity Support:** A narrow plateau may be used to create support directly around stoma.

✓ **Plateau Width:** A wider plateau width may be used to flatten out skin creases away from the stoma.

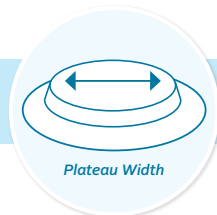

### Plateau Width Guidance for Esteem™+ Flex and Soft Convex

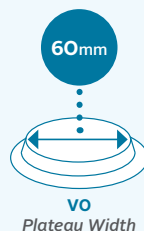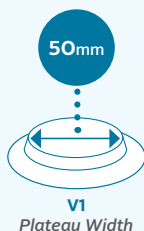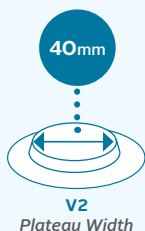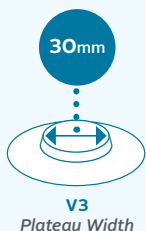

# ESTEEM™ + CONVEX

One Piece Solution with Convex Baseplates.



✓ ESTEEM™+ SOFT CONVEX POUCH

## Cut-to-fit | Closed | Soft Convex | Standard

- Convex Baseplate Depth - 4mm
- Quantity - 30

Filter (Filter covers included)

Cutting Range	Plateau Size	Opaque with Viewing Window
20-47mm	V1	422360
15-40mm	V2	422361
10-28mm	V3	422362

## Pre-Cut | Closed | Soft Convex

Filter (Filter covers included)

Stoma Size	Plateau Size	Opaque with Viewing Window
25mm	V1	422527
30mm	V1	422528
35mm	V1	422529
40mm	V1	422530
45mm	V1	422531
25mm	V2	422533
30mm	V2	422534
35mm	V2	422535
25mm	V3	422538







## ESTEEM™+ SOFT CONVEX POUCH

### Cut-to-fit | Drainable | Soft Convex | Standard

- Convex Baseplate Depth - 4mm
- Pouch Clip (1 clip per box)
- Quantity - 10

**Filter** *(Filter covers included)*

Cutting Range	Plateau Size	Opaque with Viewing Window
20-47mm	V1	422363
15-40mm	V2	422364
10-28mm	V3	422365

### Pre-Cut | Closed | Soft Convex | Standard

**Filter** *(Filter covers included)*

Stoma Size	Plateau Size	Opaque with Viewing Window
25mm	V1	422539
30mm	V1	422540
35mm	V1	422541
40mm	V1	422542
45mm	V1	422543
25mm	V2	422545
30mm	V2	422546
35mm	V2	422547
25mm	V3	422550



## ✓ ESTEEM™+ SOFT CONVEX POUCH

### Cut-to-fit | Urostomy | Soft Convex | Standard

- Convex Baseplate Depth - 4mm
- Esteem™+ Soft Convex Connector (2 per box)
- Non-return valve
- Quantity - 10

Cutting Range	Plateau Size	Opaque with Viewing Window
20-47mm	V1	422551
15-40mm	V2	422557
10-28mm	V3	422562

### Pre-Cut | Closed | Soft Convex | Standard

Stoma Size	Plateau Size	Opaque with Viewing Window
25mm	V1	422552
30mm	V1	422553
35mm	V1	422554
40mm	V1	422555
45mm	V1	422556
25mm	V2	422559
30mm	V2	422560
35mm	V2	422561
20mm	V3	422564
25mm	V3	422565





## Cut-to-fit | Closed | Flex Convex | Standard

- Convex Baseplate Depth - 7mm
- Quantity - 30

**Filter** (Filter covers included)

Cutting Range	Plateau Size	Opaque with Viewing Window
20-57mm	V0	422158
20-43mm	V1	S421612
20-35mm	V2	421613
20-25mm	V3	421614



## Cut-to-fit | Drainable | Flex Convex | Standard

- Convex Baseplate Depth - 7mm
- Pouch Clip (1 clip per box)
- Quantity - 10

**Filter** (Filter covers included)

Cut-to-fit	Drainable	Convex
20-57mm	V0	422159
20-43mm	V1	421615
20-35mm	V2	421619
20-25mm	V3	421622

## Pre-Cut | Drainable | Flex Convex | Standard

**Filter** (Filter covers included)

Stoma Size	Plateau Size	Opaque with Viewing Window
25mm	V1	421616
30mm	V1	421617
35mm	V1	421618
25mm	V2	421620
30mm	V2	421621



## ✓ ESTEEM™+ FLEX CONVEX POUCH

### Cut-to-fit | Urostomy | Flex Convex | Standard

- Convex Baseplate Depth - 7mm
- Esteem™+ Flex Convex Connector (2 per box)
- Non-return valve
- Quantity - 10

Cutting Range	Plateau Size	Opaque with Viewing Window
10-43mm	V1	421623
10-35mm	V2	421626
10-25mm	V3	421628

### Pre-cut | Urostomy | Flex Convex | Standard

Stoma Size	Plateau Size	Opaque with Viewing Window
15mm	V1	421624
20mm	V1	421625
15mm	V2	421627
15mm	V3	421629







 ESTEEM™+ FIRM CONVEX POUCH

**Pre-Cut | Drainable | InvisiClose® Closure |  
Lock-it Pocket | Firm Convex | Standard**

- Durahesive®
- Microporous Collar
- Quantity - 10

**Filter** (*Filter covers included*)

Stoma Size	Transparent	Opaque
19mm		416739
22mm	416740	416741
25mm	416742	416743
28mm	416744	416745
32mm	416746	416747
35mm	416748	416749
38mm		416751



# A NETWORK OF SUPPORT AND ADVICE FOR THOSE LIVING WITH A STOMA



## MEMBER BENEFITS


A free RADAR key and a 'Can't Wait Card' gives wearers access to over 10,000 accessible toilets throughout the UK.



FOR PATIENT  
SIGN UP SEE  
PAGES 1 & 2

# TWO-PIECE



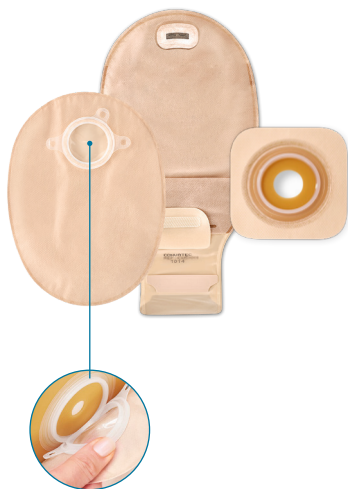


## *Solutions*




**NATURA®** 

The Natura® brand (inclusive of Natura®, and Natura®+) are baseplates and pouches that connect together with Click Coupling Technology. The baseplate and pouches connect together with a plastic ring called a flange in a simple snap-off, snap-on design.

### Colour Key & Sizing Guide

COLOUR MATCH						
FLANGE SIZE	32mm	38mm	45mm	57mm	70mm	100mm

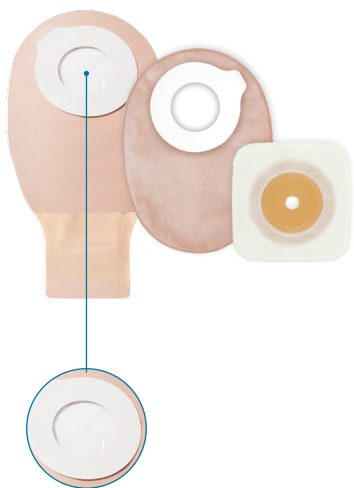


There are two brands that make up ConvaTec's Two-Piece Stoma Solutions: Natura<sup>®</sup>+ & Natura<sup>®</sup> and Esteem Synergy<sup>®</sup>+ and Esteem Synergy<sup>®</sup>.



## ESTEEM SYNERGY<sup>®</sup>+

The Esteem Synergy<sup>®</sup>+ brand are baseplates and pouches that connect together with Adhesive Coupling Technology™. This is a flat, flexible adhesive foam ring on the pouch that sticks to a flat 'landing zone' on the baseplate.

### Colour Key & Sizing Guide

COLOUR MATCH			
COUPLING SIZE	35mm	48mm	61mm



When ordering a Two-Piece Solution, ***make sure you select the same coupling size for the baseplate and pouch.***

Use the coloured dots in the table to help.





# NATURA® FLAT BASEPLATE

Compatible with Natura®+ and Natura® pouches



NATURA® BASEPLATE

## Mouldable | Accordion

- Durahesive®
- Quantity - 5

	Coupling Size	Stoma Size	Microporous Collar TAN
●	57mm	13-22mm	421036
●	57mm	22-33mm	421037
●	70mm	33-45mm	421038



NATURA® BASEPLATE

## Mouldable | Flexible | Hydrocolloid

- Stomahesive® & Durahesive®
- Quantity - 10

	Coupling Size	Stoma Size	Microporous Hydrocolloid Collar WHITE
●	45mm	13-22mm	S7550
●	45mm	22-33mm	S7551
●	57mm	33-45mm	S7552
●	70mm	45-56mm	S7553



✓ NATURA® BASEPLATE

### Cut-to-fit | Accordion

- Durahesive®
- Quantity - 5

	Coupling Size	Cutting Range	Microporous Collar WHITE
●	45mm	13-21mm	421453
●	57mm	13-33mm	421457
●	70mm	13-45mm	421461




✓ NATURA® BASEPLATE

### Cut-to-fit | Flexible

- Stomahesive®
- Quantity - 10

	Coupling Size	Cutting Range	Microporous Collar WHITE
●	32mm	13-19mm	S7238
●	38mm	13-25mm	S7239
●	45mm	13-32mm	S7240
●	57mm	13-45mm	S7241
●	70mm	13-57mm	S7242

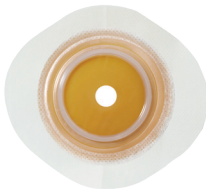

✓ NATURA® BASEPLATE

### Cut-to-fit

- Stomahesive®
- Quantity - 5

	Coupling Size	Cutting Range	Microporous Collar WHITE
●	32mm	13-19mm	S7244
●	38mm	13-25mm	S7245
●	45mm	13-32mm	S7246
●	57mm	13-45mm	S7247
●	70mm	13-57mm	S7248





## Cut-to-fit | Flexible | Hydrocolloid

- Stomahesive®
- Quantity - 10

	Coupling Size	Cutting Range	Microporous Hydrocolloid Collar WHITE
●	32mm	13-19mm	S7201
●	38mm	13-25mm	S7202
●	45mm	13-32mm	S7203
●	57mm	13-45mm	S7204
●	70mm	13-57mm	S7205



## Cut-to-fit | Flexible | Hydrocolloid

- Stomahesive®
- Quantity - 10

	Coupling Size	Cutting Range	Full Hydrocolloid TAN
●	32mm	13-19mm	S7340
●	38mm	13-25mm	S7341
●	45mm	13-32mm	S7342
●	57mm	13-45mm	S7343
●	70mm	13-57mm	S7344




## Cut-to-fit | Firm | Hydrocolloid

- Stomahesive®
- Quantity - 10
- \*Quantity - 5

	Coupling Size	Cutting Range	Full Hydrocolloid TAN
●	32mm	13-19mm	S7294
●	38mm	13-25mm	S7295
●	45mm	13-32mm	S7296
●	57mm	13-45mm	S7297
●	70mm	13-57mm	S7298
●	100mm	13-88mm	S7299B*

# NATURA® CONVEX BASEPLATES

Compatible with Natura®+ and Natura® pouches

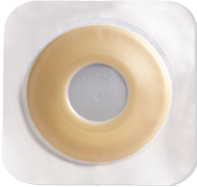

✓ NATURA® BASEPLATE

## Mouldable | Flexible | Firm Convex

- Durahesive®
- Convex Depth - 5mm
- Quantity - 10

	Coupling Size	Stoma Size	Microporous Collar WHITE
●	45mm	13-22mm	S7304
●	45mm	22-33mm	S7305
●	57mm	33-45mm	S7306



✓ NATURA® BASEPLATE

## Cut-to-fit | Accordion | Firm Convex | Hydrocolloid

- Durahesive®
- Convex Depth - 5mm
- Quantity - 5

	Coupling Size	Cutting Range	Microporous Collar TAN
●	45mm	13-22mm	421633
●	57mm	13-35mm	421635
●	70mm	13-48mm	421637





# Pre-cut | Firm Convex

- Durahesive®
- Convex Depth - 5mm
- Quantity - 5

	Coupling Size	Stoma Size	Microporous Collar WHITE
●	45mm	13mm	S7325
●	45mm	16mm	S7326
●	45mm	19mm	S7327
●	45mm	22mm	S7328
●	45mm	25mm	S7329
●	45mm	28mm	S7330
●	45mm	32mm	S7331
●	45mm	35mm	S7332
●	57mm	38mm	S7335
●	57mm	41mm	S7336
●	57mm	45mm	S7337
●	57mm	50mm	S7338





Sillesse barrier wipes have removed an annoying itch around the edge of my baseplate. My skin is much healthier as a result.

RACHEL, KENT



FOR SKIN  
PREPARATION,  
SEE PAGE 49



ConvaTec

Stoma Care

# NATURA® + POUCHES

Compatible with Natura® Baseplates



✓ NATURA® + POUCH

## Closed | Small

- Quantity - 30

Filter (Filter covers included)

	Coupling Size	Opaque with Viewing Window
●	32mm	421794
●	38mm	421795
●	45mm	421796
●	57mm	421797

## Closed | Standard

- Quantity - 30

Filter (Filter covers included)

	Coupling Size	Opaque with Viewing Window	Opaque	Transparent
●	32mm	421676	416400	
●	38mm	421678	416403	
●	45mm	421680	416406	421803
●	57mm	421682	416409	421804
●	70mm	421798	416412	421805

## Closed | Large

- Quantity - 30

Filter (Filter covers included)

	Coupling Size	Opaque with Viewing Window
●	38mm	421799
●	45mm	421800
●	57mm	421801
●	70mm	421802



✓ NATURA® + POUCH

## Drainable | InvisiClose® Closure | Lock-it Pocket™ | Standard

- Quantity - 10

**Filter** (Filter covers included)

	Coupling Size	Opaque with Viewing Window	Opaque	Transparent	Transparent No Filter
●	38mm	421746	416415		
●	45mm	421747	416417	416416	416418
●	57mm	421748	416420	416419	416421
●	70mm	421749	416423	416422	
●	100mm				416472



✓ NATURA® + POUCH

## Urostomy | Soft Tap | Standard

- Universal Connector (2 per box)
- Non-return valve
- Quantity - 10

	Coupling Size	Transparent
●	32mm	413435
●	38mm	413436
●	45mm	413437
●	57mm	413438
●	70mm	413439



# NATURA® HIGH OUTPUT POUCH

Compatible with Natura® Baseplates



## Drainable | High-Output | Large

- Non-return valve
- Outlet with spout and cap for high output drainage
- Night Drainage Connector (1 per box)
- Replacement Filters (5 per box)
- Quantity - 5

**Filter** *(Filter covers included)*

	Coupling Size	Opaque	Transparent
●	45mm	S7467	420695
●	57mm	S7468	420696
●	70mm	S7469	420697

Notes

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

Product Description

Product Code

.....	.....
.....	.....
.....	.....
.....	.....
.....	.....
.....	.....
.....	.....
.....	.....
.....	.....



# ESTEEM SYNERGY® FLAT BASEPLATES

Compatible with Esteem Synergy®+ and Esteem Synergy® Pouches



## ✓ ESTEEM SYNERGY® BASEPLATE

### Mouldable | Flexible

- Stomahesive® & Durahesive®
- Quantity - 10

	Coupling Size	Stoma Size	Microporous Collar WHITE
●	48mm	13-22mm	411657
●	48mm	22-33mm	411658
●	61mm	33-45mm	411659



## ✓ ESTEEM SYNERGY® BASEPLATE

### Cut-to-fit | Firm | Hydrocolloid

- Stomahesive®
- Quantity - 10

	Coupling Size	Cutting Range	Full Hydrocolloid TAN
●	35mm	13-35mm	S1130
●	48mm	13-48mm	S1131
●	61mm	13-61mm	S1132





### ✓ ESTEEM SYNERGY® BASEPLATE

#### Cut-to-fit | Flexible

- Stomahesive®
- Quantity - 10

	Coupling Size	Cutting Range	Microporous Collar WHITE
●	35mm	13-35mm	S1100
●	48mm	13-48mm	S1101
●	61mm	13-61mm	S1102



### ✓ ESTEEM SYNERGY® BASEPLATE

#### Pre-cut | Flexible

- Stomahesive®
- Quantity - 10

	Coupling Size	Stoma Size	Microporous Collar WHITE
●	35mm	25mm	S1144
●	35mm	32mm	S1146
●	35mm	35mm	S1147



### ✓ ESTEEM SYNERGY® BASEPLATE

#### Cut-to-fit | Flexible | Hydrocolloid


- Stomahesive®
- Quantity - 10

	Coupling Size	Cutting Range	Microporous Hydrocolloid Collar WHITE
●	35mm	13-35mm	S1190
●	48mm	13-48mm	S1191
●	61mm	13-61mm	S1192



# ESTEEM SYNERGY® CONVEX BASEPLATE

Compatible with Esteem Synergy®+ and Esteem Synergy® Pouches

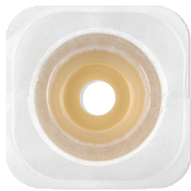

 ESTEEM SYNERGY® BASEPLATE

## Mouldable | Flexible | Firm Convex

- Durahesive®
- Convex Depth - 5mm
- Quantity - 10

	Coupling Size	Stoma Size	Microporous Collar WHITE
●	35mm	13-35mm	S1180
●	48mm	13-48mm	S1181
●	61mm	13-61mm	S1182

# ESTEEM SYNERGY®+ POUCHES

Compatible with Esteem Synergy®+ and Esteem Synergy® Pouches



✓ ESTEEM SYNERGY®+ POUCH

## Closed | Small

- Quantity - 30

Filter (Filter covers included)

	Coupling Size	Opaque
●	35mm	416777
●	48mm	416781

## Closed | Standard

Filter (Filter covers included)

	Coupling Size	Opaque	Transparent
●	35mm	416775	416774
●	48mm	416779	416778
●	61mm	416783	416782



✓ ESTEEM SYNERGY®+ POUCH

## Drainable | InvisiClose® Closure | Locket-it Pocket™ | Standard

- Quantity - 10

Filter (Filter covers included)

	Coupling Size	Opaque	Transparent	Transparent No Filter
●	35mm	416796	416795	
●	48mm	416799	416798	416900
●	61mm	416902	416901	416903










 ESTEEM SYNERGY® POUCH

## Urostomy | Accuseal® Tap | Small

- Non-return valve
- Accuseal® Connector (2 per box)
- Quantity - 10

	Coupling Size	Transparent
	35mm	S1075
	48mm	S1076

## Urostomy | Accuseal® Tap | Standard

	Coupling Size	Opaque	Transparent
	35mm	S1080	S1085
	48mm	S1081	S1086
	61mm	S1082	S1087

# STOMA CAPS

*Ideal for those who choose to irrigate.*

## ONE PIECE SOLUTION



COLODRESS® STOMA CAP

### Cut-To-Fit | Stoma Cap

- Stomahesive®
- Quantity - 30

Stoma Size	Size	Opaque
19-50mm	Small	S821

## TWO PIECE SOLUTION

*Compatible with Natura® Baseplates*



NATURA® STOMA CAP

### Pre-Cut | Stoma Cap

- Stomahesive®
- Quantity - 25

	Coupling Size	Opaque
●	45mm	S7251
●	57mm	S7252



# PAEDIATRIC

## Solutions

**LITTLE ONES™** is a paediatric ONE-Piece and TWO-Piece solution.





Little ONES™ Extra Small is designed for children weighing less than 4.5kg whilst LITTLE ONES™ is suggested for children weighing over 4.5kg.



### Colour Key & Sizing Guide



When ordering a Paediatric Two-Piece Solution, ***make sure you select the same coupling size for the baseplate and pouch.*** Use the coloured dots in the table to help.

COLOUR				
CUTTING RANGE / COUPLING SIZE	0-23mm	4.8-31mm	32mm*	45mm*

*\*Indicates Baseplate size measurement*



# LITTLE ONES™ EXTRA SMALL ONE-PIECE



LITTLE ONES™ EXTRA SMALL POUCH

## Cut-to-Fit | Drainable | InvisiClose® Closure

- Stomahesive®
- No Starter Hole
- Pouch Length - 6 inches
- Quantity - 10

### Stoma Size

### Transparent



0-23mm

S5090

# LITTLE ONES™ ONE-PIECE



LITTLE ONES™ ILEODRESS®

## Cut-to-Fit | Drainable | Clipped Closure

- Stomahesive®
- Pouch Length - 6 inches
- Includes Straight Pouch Clip (1 per box)
- Quantity - 15

### Cutting Range

### Transparent

8-50mm

S880



# LITTLE ONES™ EXTRA SMALL TWO-PIECE



LITTLE ONES™ EXTRA SMALL ESTEEM SYNERGY® BASEPLATE

## Cut-to-Fit

- Stomahesive®
- Adhesive Coupling Technology™
- No Starter Hole
- Quantity - 5

	Stoma Size	Without Collar TAN
●	0-23mm	S1171



LITTLE ONES™ EXTRA SMALL ESTEEM SYNERGY® POUCH

## Cut-to-Fit

- Adhesive Coupling Technology™
- Pouch Length - 6 inches
- Quantity - 10

	Stoma Size	Transparent
●	0-23mm	S1271

# LITTLE ONES™ TWO-PIECE

## NATURA® BASEPLATE



LITTLE ONES™

### Cut-to-Fit | Flexible

- Stomahesive®
- Quantity - 5

	Coupling Size	Cutting Range	Microporous Collar WHITE
●	32mm	12-32mm	S7811
●	45mm	12-45mm	S7812

## ESTEEM SYNERGY® BASEPLATE



LITTLE ONES™

### Cut-to-Fit

- Stomahesive®
- Adhesive Coupling Technology™
- Quantity - 5

	Cutting Range	Microporous Collar WHITE	Microporous Flexible Hydrocolloid Collar WHITE
●	4.8-31mm	S1172	S1173



# NATURA® POUCHES



LITTLE ONES™ NATURA® POUCH

## Closed

- Pouch Length - 5 inches
- Quantity - 20

	Coupling Size	Opaque
●	45mm	S7892



LITTLE ONES™ NATURA® POUCH

## Drainable | Clipped Closure

- Stomahesive®
- Pouch Length - 6 inches
- Soft Wire Tie (10 per box)
- Quantity - 10

	Coupling Size	Transparent
●	32mm	S7880
●	45mm	S7881



LITTLE ONES™ NATURA® POUCH

## Urostomy | Fold-up-Tap

- Pouch Length - 6 inches
- Universal Paediatric size Connector (1 per box)
- Quantity - 10

	Coupling Size	Transparent
●	32mm	S7850

# ESTEEM SYNERGY® POUCHES



LITTLE ONES™ ESTEEM SYNERGY® POUCH

## Drainable | InvisiClose® Closure

- Adhesive Coupling Technology™
- Pouch Length - 8 inches
- Quantity - 10

**Filter (15 Filter covers included)**

	Stoma Size	Opaque	Transparent
●	4.8-31 mm	S1273F	S1275F

## Notes

# ACCESSORIES

## Choosing the right accessories

Whether you are looking for better skin protection or added security to your stoma solution, with the right combination of accessories, you can feel even more comfortable and at ease.



**Pouch  
Removal**



**Skin  
Preparation**



**Pouching  
Support**



**Added  
Security**





# POUCH REMOVAL

*Solutions that help promote healthy skin by protecting it from irritation and damage caused by pouch removal.*



NILTAC™

## Sting Free Medical Adhesive Remover Spray

- Gentle adhesive remover without damaging skin
- Helps minimise painful skin stripping during pouch removal
- Silicone based
- Alcohol free

**Product Code**

**Spray Bottle Size**

TR101

50ml



NILTAC™

## Sting Free Medical Adhesive Remover Wipe

- Helps minimise painful skin stripping during pouch removal
- Designed to remove adhesive residue from the skin
- Silicone based
- Alcohol free

**Product Code**

**Box of**

TR102

30





# SKIN PREPARATION

*Solutions that help promote healthy skin by reducing irritation and damage.*



SILESSE™

## Sting Free Skin Barrier Spray

- Designed to be used at every pouch change
- Creates a comfortable, breathable barrier between skin & baseplate
- Silicone based
- Non-clogging spray

**Product Code**

**Spray Bottle Size**

TR104

50ml



SILESSE™

## Sting Free Skin Barrier Wipes

- Designed to be used at every pouch change
- Creates a comfortable, breathable barrier between skin & baseplate
- Silicone based

**Product Code**

**Box of**

TR103

30



## SENSi-CARE™

## Sting Free Skin Barrier (Sterile Foam Applicator)

- Designed to create a protective layer
- Helps reduce skin stripping
- Silicone based

Product Code	Applicators Size	Individually Packaged Applicators per Box
420793	1ml	5
420795	3ml	5



## ORAHESIVE®

## Protective Powder

- Helps protect irritated skin by absorbing excess moisture

Product Code	Size
S106	25g



## ORABASE®

## Protective Paste

- Protective paste which adheres to moist areas

Product Code	Size
S103	30g





# POUCHING SUPPORT

*Products that help create the best fit around the stoma and prevent leakage for better pouch performance.*



DUODERM®

## Extra Thin Dressing

- Designed to help protect the skin from contact with body fluids

Product Code	Size	Box of
S161	100mm x 100mm	10



STOMAHESIVE®

## Skin Barrier Wafer

- Designed to help protect the skin from contact with bodily fluids
- Large cutting surface to accommodate large and swollen stomas
- Sterile Stomahesive® skin barrier can be used post-operatively

Product Code	Size	Box of
S100	100mm x 100mm	5
S101	200mm x 200mm	3



STOMAHESIVE®

## Seal

- Seals provided added security and comfort.
- Acts as a filler on uneven skin surfaces to help create a flatter area for the baseplate to adhere to
- Designed to absorb moisture and prevent it coming in contact with, and causing harm to the skin

Product Code	Size	Box of
413503	Standard Seal (48mm Width, 4mm Depth)	10
413504	Standard Thin Seal (48mm Width, 3mm Depth)	10



## ADDED SECURITY

*An extra layer of protection to keep your stoma solution in place.*



### DIAMONDS™

#### Gelling and Odour Control Sachets

- Super absorbent gelling sachets with unique ActiveOne™ odour control prevents leaks, odours and noise
- Provides a flatter discreet pouch

#### Product Code

#### Box of

TR105

100

STR105 (Sample)

5



### STOMAHESIVE®

#### Paste

- Helps to protect exposed skin between the base of the stoma and the opening of the baseplate
- May be used to fill crevices, skin folds and scars to help create a flat surface to aid pouch application
- Contains alcohol

#### Product Code

#### Size

S105

60g



### URIHESIVE™

#### Strips

- Helps to create a secure seal between the stoma and the appliance, to protect the skin against stoma output
- Can be cut, torn and shaped into a variety of shapes to help prevent leakage around the stoma by acting as a filler
- Designed to absorb moisture from the stoma, creating a snug fit around the stoma.

#### Product Code

#### Box of

S120

15 strips





CONVATEC®

## Stoma Belt

- Used to help keep your stoma solution in place

Product Code	Box of
S210	1



CONVATEC®

## Curved Clip

- Helps secure the end of ConvaTec Drainable pouch

Product Code	Box of
S202	10



CONVATEC®

## Straight Clip

- Helps secure the end of ConvaTec Drainable pouch

Product Code	Box of
S207	10

## REFERENCES

1. ConvaTec Internal in vitro lab testing. OCT 0112.
2. Maria Teresa Szewczyk, MD, PhD; Grazyna Majewska, RN, ETN; Mary V. Cabral, MS, FNP-BC, CWOCN-AP; and Karin Hölzel-Piontek, RN; The Effects of Using a Moldable Skin Barrier on Peristomal Skin Condition in Persons with an Ostomy: Results of a Prospective, Observational, Multinational Study, Ostomy Wound Management 2014;60(12):16–26.
3. Turnbull, GB. The convexity controversy. Ostomy Wound Manage. 2003;49(1): 16-[www.o-wm.com/article/1203](http://www.o-wm.com/article/1203). Accessed 29 Feb, 2016.
4. Trio Products Test Report TR- 012. September 2010. Data on file. ConvaTec, Inc
5. Steele N. Skin care protection for urostomates. Journal of Community Nursing. 2009; 23 (6): 16-17,







Our goal is to *partner with you and your patients to deliver best-in-class care* throughout the patient journey

---

**TO ENROLL YOUR PATIENTS  
in the me+ programme**

call us on 0800 467 866  
or visit [meplus.convatec.co.uk](https://meplus.convatec.co.uk)

---



**ConvaTec**

Stoma Care

For general product information visit [www.convatec.co.uk/stoma-care](https://www.convatec.co.uk/stoma-care)