INDICATIONS FOR USE

The thalaset™ infusion set is indicated for subcutaneous infusion of medication administered by an external pump. The infusion set is indicated for single use.

PRODUCT DESCRIPTION

thalaset™ is a 90-degree, stainless steel needle infusion set with a standard luer lock connection.

CONTRAINDICATIONS

thalaset™ is neither intended nor indicated for intravenous infusion (I.V.) of medication, including blood and blood products.

WARNINGS

■ Carefully examine the instructions for use before inserting the thalaset™. Failure to follow instructions may result in pain or injury.

■ The thalaset™ infusion set is only sterile and non-pyrogenic if the packaging is undamaged

and unopened. Do not use it if the packaging is already open or is damaged.

- Inaccurate medication delivery, site irritation or infection may result from improper insertion and maintenance of infusion site.
- When using thalaset™ for the first time, do so in the presence of a healthcare provider.
- Do not reuse the infusion set. Reuse of the infusion set may cause site irritation or infection and may lead to inaccurate medication delivery.
- Carefully remove the needle guard before inserting the infusion set.
- Do not bend the needle prior to insertion, and do not use the infusion set if the needle is bent or has been damaged.

PRECAUTIONS

- Never prime the infusion set or attempt to free a clogged tubing while the infusion set is inserted. You may accidentally inject too much medication.
- Check the infusion site frequently to ensure that the

steel needle remains firmly in place. Replace with a new infusion set at a new site, if the infusion set is not in place.

- thalaset™ is a single use item which must be disposed of after usage. Do not clean or re-sterilize.
- Choose insertion sites as recommended by your healthcare provider.
- Do not in any way put disinfectants, perfumes, deodorants or other products containing disinfectants in contact with the luer lock connector and the tubing, as this may impact the integrity of the infusion set.
- Replace with a new infusion set if the adhesive tape becomes loose.
- Change the infusion set every one to two days, or as recommended by your healthcare provider. Refer to the prescribing information (PI) of the medication being infused as to the recommended replacement schedule for thalaset™.
- Upon removal, carefully loosen the edges of the adhesive tape, and remove the infusion set slowly by

pulling straight out. Make sure that the needle is still attached to the adhesive and pulled completely out of the skin. If the needle is missing, immediately contact your healthcare provider for assistance.

- If the infusion site becomes irritated or infected, replace with a new infusion set and use a new site until the first site has healed.
- If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

STORAGE & DISPOSAL

- Store infusion sets in a cool, dry location at room temperature. Do not store infusion sets in direct sunlight, or in high humidity.
- Dispose of the infusion set in a proper sharps container according to local laws, to avoid risk of needle stick.

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