



EXUDATE



INFECTION



BIOFILM

Wound healing
has always had villains.

Now it has a hero.

AQUACEL[®] **Ag+**
Dressings

No dressing does more.[†]

Available on Drug Tariff from 1st June 2014

No dressing does more.[†]

3

Three wound healing villains:
exudate, infection and biofilm.



2

Two powerful technologies.

NEW Ag+ Technology

Revolutionary technology **destroys biofilm**
and **kills infection-causing bacteria**.^{*1-3}



Hydrofiber® Technology

Proven technology that **absorbs and retains**
excess exudate to help create an ideal
healing environment.^{*4-8}



1

One wound healing hero.

AQUACEL® **Ag+**
Dressings

Currently available in **AQUACEL® Ag+ Extra™** and
AQUwACEL® Ag+ Ribbon dressings.

*As demonstrated *in vitro*

†Demonstrated ability to manage excess exudate, infection and biofilm.

Biofilm delays wound healing.

Biofilm is common.

Biofilm is formed when colonies of bacteria secrete a slime layer to protect themselves.⁹

It is involved in approximately 80% of all healthcare infections.¹⁰ The plaque on your teeth, urinary tract infections and eye infections are all linked to biofilm.¹¹⁻¹³

Even though you can't always see it, **the majority of chronic wounds contain biofilm**¹⁴ – and it's a key cause of delayed wound healing¹⁵ and a precursor to infection.¹⁶

Biofilm is stubborn.


Biofilm is **difficult to completely remove**¹⁷ – even with debridement – and it **reforms quickly**.¹⁸ Biofilm tolerates:

- Antimicrobials such as PHMB^{**19}, honey²⁰, iodine^{21,22} and silver²³
- Antibiotics²⁴
- Attempts by the body to clean the wound bed²⁵ and close the wound¹⁹



Suspected biofilm

Microscopic view
of biofilm

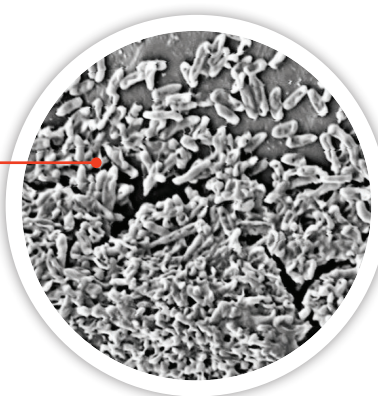

BIOFILM

^{**} polyhexamethylene biguanide

All pictures used with permission of their respective owners.

Two powerful technologies working together to overcome the key barriers to wound healing.



Ag+ Technology is a unique, silver-containing anti-biofilm formulation that:²⁶


- **DISRUPTS** and breaks down biofilm slime to expose bacteria^{*1-3}
- **KILLS** a broad spectrum of bacteria, including antibiotic resistant superbugs, with its reservoir of silver^{*2,3,27}
- **PREVENTS** biofilm reformation^{*2,3}

Proven in the lab

In an *in vitro* biofilm model, **AQUACEL® Ag+ Extra** dressing demonstrated superior ability to **destroy biofilm and prevent reformation.**²⁸⁻³⁰

- **AQUACEL® Ag+ Extra™** (n=5)
- **AQUACEL® Ag Extra™** (n=5)
- **Acticoat 7** (n=5)
- **Reinoculation**


Pseudomonas aeruginosa

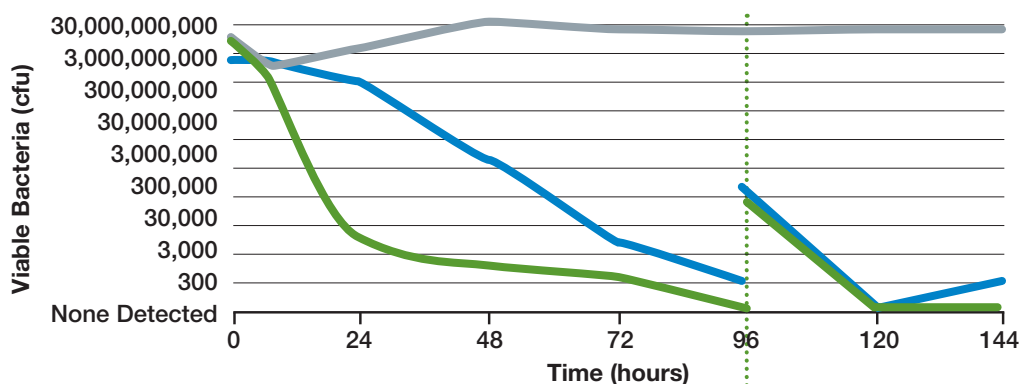

Disrupts Biofilm / Kills Bacteria

Prevents Biofilm Reformation

MRSA

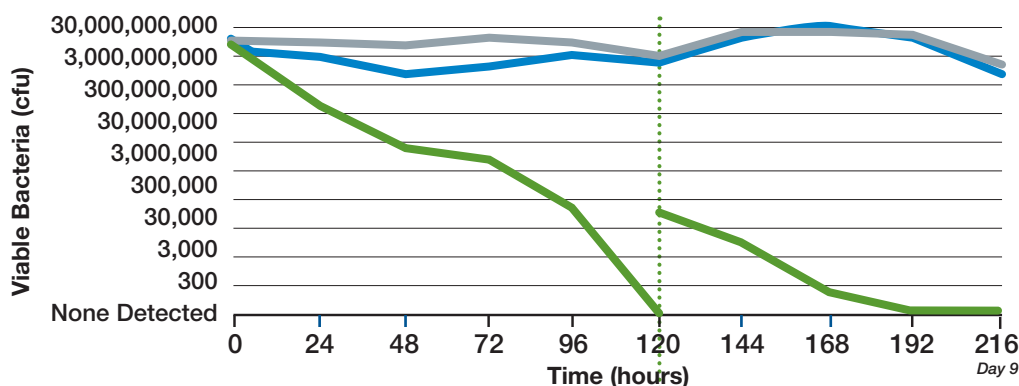

Disrupts Biofilm / Kills Bacteria

Prevents Biofilm Reformation

In this *in vitro* model, mature biofilm was grown on a gauze substrate and confirmed by microscopy. Gauze-biofilm substrates were then transferred to agar plates to create a simulated wound biofilm model; dressings were applied to the biofilm surface, hydrated and covered with an appropriate secondary dressing. Following incubation, the killing effect of the dressing on biofilm-embedded bacteria was assessed at several time points over a maximum of 120 hours. Biofilm reformation was also assessed by inoculating fresh bacteria onto the gauze substrate beneath the dressing, followed by assessment of biofilm presence or absence over a maximum of 96 hours.

^{*}As demonstrated *in vitro*

synergistically to manage


Hydrofiber® Technology helps create an ideal environment for healing – and for Ag+ Technology to work.



- **LOCKS IN** excess exudate, bacteria and biofilm to help minimise cross infection and prevent maceration^{*4-7,31,32}
- **MICRO-CONTOURS** to the wound bed, maintaining optimal moisture balance and eliminating dead spaces where bacteria and biofilm can grow^{*33-35}
- **RESPONDS** to wound conditions by forming a cohesive gel, while helping to minimise pain associated with dressing changes^{*36-38}

Extra absorbency means **longer wear time**^{*39-41}

Extra strength means **easier removal**^{*39}

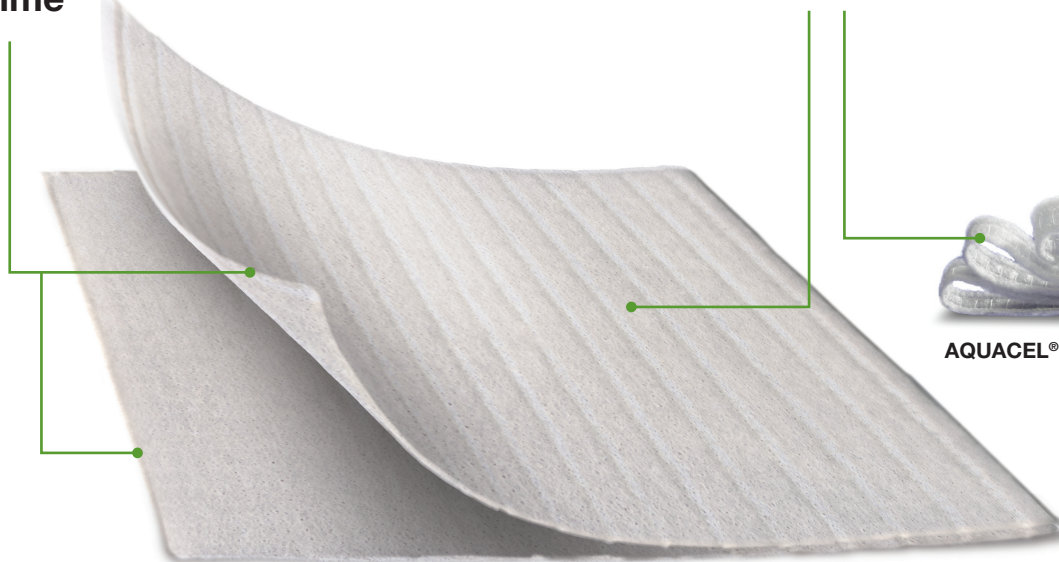

AQUACEL® Ag+ Extra™ dressing



AQUACEL® Ag+ Ribbon dressing

AQUACEL® Ag+ Dressings

AQUACEL[®] Ag+ Dressings – the family


Proven in scientifically controlled wounds

In an adapted *in vivo* biofilm model¹⁹, **Ag+ Technology** in combination with **Hydrofiber[®] Technology** demonstrated:*

- ▶ A significantly greater reduction of **biofilm** vs. a PHMB gauze.⁴¹

95%


greater reduction by day 6 (p<0.05)

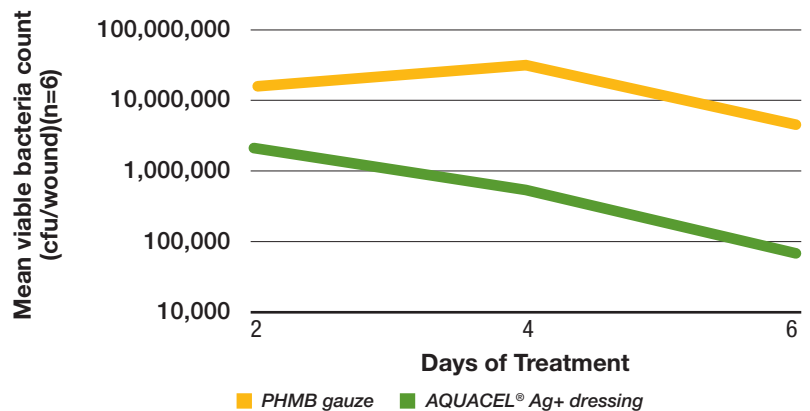

- ▶ A significantly greater rate of **epithelialisation** and **granulation** vs. a PHMB gauze.⁴¹


48%

more granulation tissue by day 6 (p<0.05)

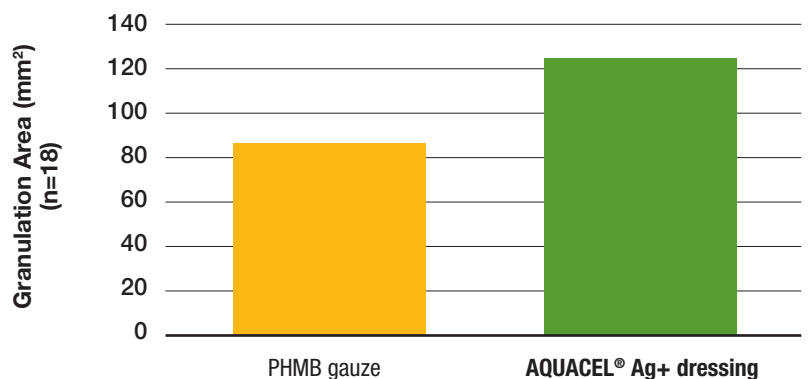

24%

more epithelial tissue by day 6 (p<0.05)

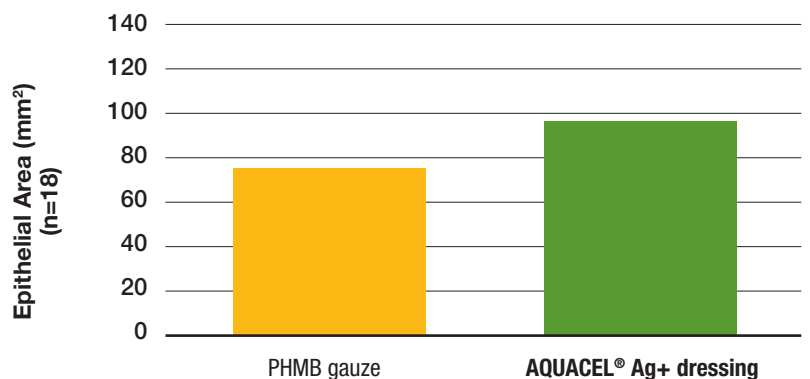

*AQUACEL[®] Ag+ dressing was used in this study

Family of wound healing heroes.

Demonstrated wound healing in the clinic⁴²



In a prospective, multi-Centre, non-comparative study on 42 chronic venous leg ulcer patients with at-risk or infected wounds[^] where biofilm is highly likely, **Ag+ Technology** in combination with **Hydrofiber® Technology** demonstrated:

54%

reduction in ulcer area
for **all** wounds

70%


reduction in ulcer area
for **infected** wounds

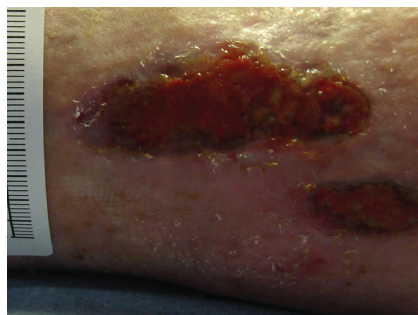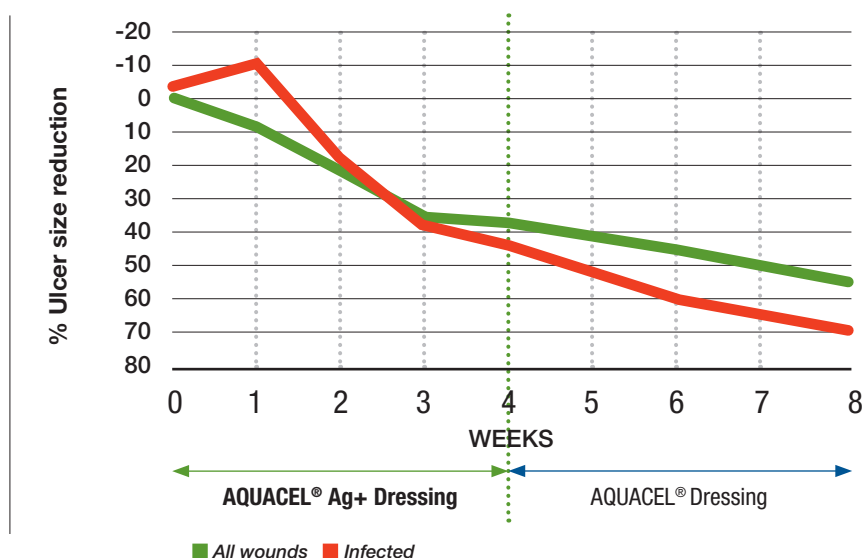

Day 1



Day 28

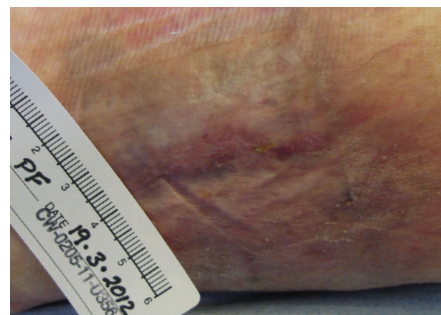


Day 49 - healed



Day 1



Day 22

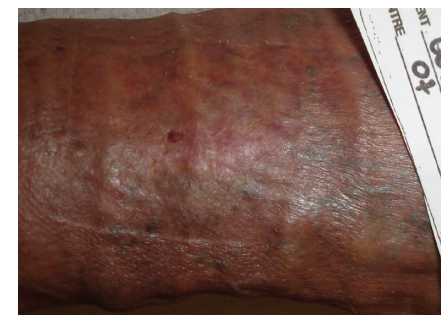

Day 56 - healed

[^]10 infected (exhibiting all 5 signs of clinical infection) and 32 at-risk wounds (exhibiting at least 3 of the 5 classical clinical signs of infection)

*AQUACEL® Ag+ dressing was used in this study

All pictures used with permission of their respective owners.

AQUACEL® Ag+ Dressings *No dressing does more†.*

Add **AQUACEL® Ag+ Dressings** to your protocol of care for chronic and acute wounds that are **infected** or **at risk of infection**.



Perfect partners: **AQUACEL® Ag+ Dressings** and **AQUACEL® Foam**

Dressing Size	Dressings per box	Product Code	NHS Code	PIP Code	Dressing Size	Dressings per box	Product Code	NHS Code	PIP Code
AQUACEL® Ag+ Extra™					AQUACEL® Foam Adhesive				
5 cm x 5 cm	10	413566	ELY514	386-2703	8 cm x 8 cm	10	420804	ELY428	378-1820
10 cm x 10 cm	10	413567	ELY515	386-2695	10 cm x 10 cm	10	420680	ELY417	370-2784
15 cm x 15 cm	5	413568	ELY516	386-2711	12.5 cm x 12.5 cm	10	420619	ELY418	370-2792
20 cm x 30 cm	5	413569	ELY517	386-2679	17.5 cm x 17.5 cm	10	420621	ELY419	370-2800
4 cm x 10 cm	10	413581	ELY520	386-0350	21 cm x 21 cm	5	420623	ELY420	370-2818
4 cm x 20 cm	10	413598	ELY521	386-0368	25 cm x 30 cm	5	420624	ELY421	370-7361
4 cm x 30 cm	10	413599	ELY522	386-2687	19.8 cm x 14 cm Heel	5	420625	ELY422	370-7486
AQUACEL® Ag+ Ribbon Dressing					20 cm x 16.9 cm Sacral	5	420626	ELY423	370-6041
2 cm x 45 cm	5	413571	ELY519	386-2737	AQUACEL® Foam Non-Adhesive				
1 cm x 45 cm	5	413570	ELY518	386-2729	5 cm x 5 cm	10	420631	ELY412	369-9311
					10 cm x 10 cm	10	420633	ELY413	369-9329
					15 cm x 15 cm	5	420635	ELY414	370-2750
					20 cm x 20 cm	5	420636	ELY416	370-2776
					15 cm x 20 cm	5	420637	ELY415	370-2768

1. Physical Disruption of Biofilm by AQUACEL® Ag+ Wound Dressing. Scientific Background Report. WHRI3850 MA232, 2013. Data on file, ConvaTec Inc. 2. Antimicrobial activity and prevention of biofilm reformation by AQUACEL™ Ag+ EXTRA dressing. Scientific Background Report. WHRI3875 MA239, 2013. Data on file, ConvaTec Inc. 3. Antimicrobial activity against CA-MRSA and prevention of biofilm reformation by AQUACEL™ Ag+ EXTRA dressing. Scientific Background Report. WHRI3875 MA239, 2013. Data on file, ConvaTec Inc. 4. Newman GR, Walker M, Hobot JA, Bowler PG, 2006. Visualisation of bacterial sequestration and bacterial activity within hydrating Hydrofiber™ wound dressings. *Biomaterials*; 27: 1129-1139. 5. Walker M, Hobot JA, Newman GR, Bowler PG, 2003. Scanning electron microscopic examination of bacterial immobilization in a carboxymethyl cellulose (AQUACEL™) and alginate dressing. *Biomaterials*; 24: 883-890. 6. Bowler PG, Jones SA, Davies BJ, Coyle E, 1999. Infection control properties of some wound dressings. *J. Wound Care*; 8: 499-502. 7. Walker M, Bowler PG, Cochrane CA, 2007. *In vitro* studies to show sequestration of matrix metalloproteinases by silver-containing wound care products. *Ostomy/Wound Management*. 2007; 53: 18-25. 8. Assessment of the *in vitro* Physical Properties of AQUACEL EXTRA, AQUACEL Ag EXTRA and AQUACEL Ag+ EXTRA dressings. Scientific background report. WHRI3817 TA297, 2013. Data on file, ConvaTec Inc. 9. Bjarnsholt T, 2013. The role of bacterial biofilms in chronic infections. *APMIS*. 121. 1-51. 10. Research on microbial biofilms. National Institute of Dental and Craniofacial Research. <http://grants.nih.gov/grants/guide/pa-files/PA-03-047.html>; Sept. 9, 1997. 11. Marsh PD, Bradshaw DJ, 1995. Dental plaque as a biofilm. *J. Industr. Microbiol*; 15: 169-175. 12. Trautner BW, Darouiche RO, 2004. Role of biofilm in catheter-associated urinary tract infection. *Am J Infect Control*; 32: 177-183. 13. Elder MJ, Stapleton F, Evans E, Dart JK, 1995. Biofilm-related infections in Ophthalmology. *Eye (Lond)* 9: 102-109. 14. James GA, Swogger E, Wolcott R, Pulcini EL, Secor P, Sestrich J, et al. 2008. Biofilms in Chronic Wounds. *Wound Rep Regen*; 16: 37-44. 15. Metcalf D, Bowler P, 2013. Biofilm delays wound healing: A review of the evidence. *Burns & Trauma*. 1: 5-12. 16. Percival SL, Bowler PG, 2004. Biofilms and their potential role in wound healing. *WOUNDS*; 16: 234-240. 17. Wolcott RD, Rumbaugh KP, James G, Schultz G, Phillips P, Yang O, et al. 2010. Biofilm maturity studies indicate sharp debridement opens a time-dependent therapeutic window. *J Wound Care*; 19: 320-328. 18. Wolcott RD, Kennedy JP, Dowd SE, 2009. Regular debridement is the main tool for maintaining a healthy wound bed in most chronic. *J Wound Care*; 18: 54-56. 19. Gurjara AN, Geringer MR, Seth AK, Hong SJ, Smeltzer MS, Galiano RA, et al. 2011. Development of a novel, highly quantitative *in vivo* model for the study of biofilm-impaired cutaneous wound healing. *Wound Rep Reg*. 19: 400-410. 20. Brackman G, De Meyer L, Nelis HJ, Coenye T, 2013. Biofilm inhibitory and eradicating activity of wound care products against *Staphylococcus aureus* and *Staphylococcus epidermidis* biofilms in an *in vitro* chronic wound model. *J Appl Microbiol*; 114: 1833-42. 21. Darouiche RO, Mansouri MD, Gawande PV, Madhyastha S. Antimicrobial and antibiofilm efficacy of triclosan and DispersinB combination. *J Antimicrob Chemother*. 2009 Jul;64(1):88-93. 22. Thorn RM, Greenman J. A novel *in vitro* flat-bed perfusion biofilm model for determining the potential antimicrobial efficacy of topical wound treatments. *J Appl Microbiol*. 2009 Dec 1;107(6):2070-9. 23. Bjarnsholt B, Kirketerp-Moller K, Kristiansen S, Phipps R, Nielsen AK, Jensen Po, et al. 2007. Silver against *Pseudomonas aeruginosa* biofilms. *APMIS* 115: 921-8. 24. Stewart PS, Costerton JW, 2001. Antibiotic resistance of bacteria in biofilms. *Lancet*; 358: 135-138. 25. Thurlow LR, Hanke ML, Fritz T, Angie A, Aldrich A, Williams SH, Engbrechtsen IL, et al. 2011. *Staphylococcus aureus* biofilms prevent macrophage phago-cytosis and attenuate inflammation *in vivo*. *J Immunol*; 186: 6585-96. 26. Composition comprising antimicrobial metal ions and a quaternary cationic surfactant. Scientific Background Report. WO 2012136968 A1, 2012. Data on file, ConvaTec Inc. 27. Bowler PG, Welsby S, Towers V, Booth V, Hogarth A, Rowlands V, Joseph A, et al. 2012. Multidrug-resistant organisms, wounds and topical antimicrobial protection. *Int Wound J*. 9: 387-396. 28. Antimicrobial activity against CA-MRSA and prevention of biofilm reformation by AQUACEL™ Ag+ EXTRA Dressing and Acticoat 7 Dressing. Scientific Background Report. WHRI3876 MA240, 2013. Data on file, ConvaTec Inc. 29. Antimicrobial Activity and Prevention of Biofilm Reformation by AQUACEL® Ag EXTRA Dressing and Silvercel® Non Adherent Dressing. WHRI3877 MA241, 2013. Data on file, ConvaTec Inc. 31. Walker M and Parsons D, 2010. Hydrofiber Technology: its role in exudate management. *Wounds UK*; 6: 31-38. 32. Parsons D, Bowler PG, Myles V, Jones SA, 2005. Silver antimicrobial dressings in wound management: A comparison of antibacterial, physical and chemical characteristics. *WOUNDS*; 17: 222-232. 33. Jones SA, Bowler PG, Walker M, 2005. Antimicrobial activity of silver-containing dressings is influenced by dressing conformability with a wound surface. *WOUNDS*; 17: 263-270. 34. Bowler P, Jones S, Towers V, Booth R, Parsons D, Walker M, 2010. Dressing conformability and silver-containing wound dressings. *Wounds UK*; 6: 14-20. 35. Walker M, Jones S, Parsons D, Booth R, Cochrane C, Bowler P, 2011. Evaluation of low-adherent antimicrobial dressings. *Wounds UK*; 7: 32-45. 36. Barnea Y, Amir A, Leshem D, Zaretski A, Weiss J, Shafir R, et al. 2004. Clinical comparative study of Aquacel and paraffin gauze dressing for split-skin donor site treatment. *Ann Plast Surg*; 53: 132-136. 37. Kogan L, Moldavsky M, Szvalb S, Govrin-Yehudain J, 2004. Comparative study of Aquacel and Silvercel treatment in burns. *Ann Burns Fire Disasters*; 17: 201-207. 38. Brunner U, Eberlein T, 2000. Experiences with hydrofibres in the moist treatment of chronic wounds, in particular of diabetic foot. *VASA*; 29: 253-257. 39. Assessment of the *in vitro* physical properties of AQUACEL Ag, AQUACEL Ag EXTRA and AQUACEL Ag+ Dressings. Scientific Background Report. WHRI3817 TA297, 2013. Data on file, ConvaTec Inc. 40. Harding K, Ivans N, Cains J, An opened randomized comparative study to evaluate the clinical and economic performance of two absorbent dressings in venous leg ulcers. Poster presented at EWMA; May 15-17 2013; Copenhagen, Denmark. 41. Parsons D, Mustoe T, Seth A. A new anti-biofilm Hydrofiber™ dressing: an *in vivo* investigation. Poster presented at Wounds UK; Nov 11-13 2013; Harrogate, UK. 42. Harding K, Ivans N, Cains J, Peters K, Parsons D. A new anti-biofilm dressing – a clinical study. Poster presented at EWMA; May 15-17 2013; Copenhagen, Denmark.

To learn more about AQUACEL® Ag+ dressings or to arrange a visit from your ConvaTec representative, please call **0800 289 738 (UK)** or **1800 946 938 (ROI)**
www.convatec.co.uk

AQUACEL® Ag+
Dressings

†Demonstrated ability to manage excess exudate, infection and biofilm.

AQUACEL, AQUACEL Extra, ConvaTec, the ConvaTec logo, Hydrofiber and the Hydrofiber logo are trademarks of ConvaTec Inc. All other trademarks are property of their respective owners.

©2014 ConvaTec Inc. AP-014236-GB WCON653

