



ConvaTec

How to Use the **InvisiClose®** Clipless Tail Closure

A. “Lock-it pocket”

B. Two Interlocking Closures

C. Two outlet end strips



Step 1:

Holding the white outlet end strips, fold the end of the pouch up and towards you three times until the interlocking closures line up.



Step 2:

Press interlocking closures together across the width of the tail and squeeze until you feel them close.

Optional steps for added discretion and security:



Step 3:

Place your thumbs into the “lock-it pocket” and hold the folded end of the tail between your thumbs and your index fingers.



Step 4:

Turn the “lock it pocket” inside out to hide the tail inside the pocket.

Steps to empty the pouch



Step 1:

Reverse the "lock-it pocket" to reveal the tail.



Step 2:

Unlock the inter-locking closure by lifting at the corner and unraveling the folds 3 times.



Step 3:

To make draining and/or cleaning easier, you can open the tail end by pushing in both ends of the outlet end strips with your thumb and forefinger. The pouch tail may be cleaned with gentle soap and water.



Pouch Options

There are a variety of pouch options available to meet your lifestyle needs.

Closed-End, Mini pouch, and pouches for kids.

For more information or if you have questions, call 0800 282 254
ROI 1800 721 721 or email us on stoma.webcare@convatec.com
Visit our website at www.convatecstomacare.co.uk

See package insert for complete Instructions for Use.
®/™ indicate trademarks of ConvaTec Inc.

© 2011 ConvaTec Inc. March 2010 AP-009500-MM Stock No. PLUS08INV