

How to Use

Niltac™ Sting Free Adhesive Releaser Spray



Getting Started:

Spray Niltac™ Adhesive Releaser around the edge of the adhesive and wait a few seconds.



Easy Removal:

Gently peel adhesive away from the skin using additional Niltac™ Adhesive Releaser spray to ease removal, if necessary.



Dries in Seconds:

Once dried, Niltac™ Adhesive Releaser will not affect adhesion of further dressings or appliances.

TIP: Try Silesse™ Sting Free Skin Barrier spray or wipes to protect skin before applying your appliance.

Product Code	Product Description	Product Size
420787	Niltac™ Sting Free Adhesive Releaser Spray (50ml)	One 50ML Spray Can

How to Use

Niltac™ Sting Free Adhesive Remover Wipes



Getting Started:

Tear sachet to open and unfold wipe.



Easy Removal:

Rub wipe over adhesive residue to remove.



Dries in Seconds:

Once dried, Niltac™ Adhesive Remover will not affect adhesion of further dressings or appliances.

TIP: Try Niltac™ Sting Free Adhesive Releaser Spray for no touch removal of appliances

Product Code	Product Description	Product Size
420788	Niltac™ Sting Free Adhesive Remover Wipe (30 wipes)	Box of 30 wipes



For more information, please call our Customer Relations Center (Registered Nurses on staff) at **1-800-465-6302**, Monday through Friday, 8:00 AM to 6:00 PM (EST), or visit our Website at www.convatec.ca