

FLEXI-TRAK® Anchoring Device



Indications†

For anchoring tubes and catheters to skin, such as:

- Urinary Catheters
- Gastrostomy Tubes
- I.V. Tubes
- Rectal Tubes
- Chest Tubes
- Central Venous Lines
- Pacemaker Wires
- Sump Tubes (Nasogastric Tubes)
- Biliary Drainage Tubes
- Nephrostomy Tubes
- Peritoneal Dialysis Tubes

Contraindications

- Do not use on patients with a known sensitivity to acrylic adhesive or its components¹

Please Note

- Apply only to intact skin
- Can remain in place up to 5 days
- The tabs are resealable to accommodate repositioning of the catheter/tube

¹ Data on file, ConvaTec

FLEXI-TRAK® Anchoring Device

ConvaTec SKU#	Facility SKU#	Size	Qty.
037449		Large, 4" x 1 1/2"	50/box

Instructions for Use†

- Ensure the area is clean, dry, and free from powder and greasy substances
- Remove the plain white backing paper from the device
- Apply the body pad adhesive to the desired location
- Remove printed backing paper from reusable tape on the top side
- Place the catheter/tube along the center of the device
- Mold both sides of the tape to the contour of the catheter/tube. Check that the tape is secure around catheter/tube

Instructions for Removal

- Remove catheter/tube, then apply light pressure on adjacent skin while gently pulling the adhesive from the skin

To learn more, call
1-800-422-8811
M-F, 8:30 AM - 7:00 PM, EST
Email: CIC@ConvaTec.com
www.convatec.com

†Please refer to package insert for complete indications and Instructions for Use.
®/™ indicates trademarks of ConvaTec Inc.