

Product Description

Exudate Management – As a primary or secondary cover dressing for moderately to heavily exuding wounds

CombiDERM® NA dressing is a non-adhesive absorbent dressing that is made up of two main parts; a central absorbent island pad and gel forming poly-acrylate granules.

The super absorbent granules act to absorb, gel and hold exudate away from the skin. This helps to create an ideal moist wound healing environment.

CombiDERM® NA dressing retains exudate even under compression¹ and helps prevent maceration².

Application*

Carefully cleanse the wound with saline and dry surrounding skin. Choose a dressing size that allows the absorbent pad to be slightly larger than the wound area. Remove from the sterile pouch and place the island pad, white side, directly over the wound. The dressing can then be secured, e.g. with compression or other bandaging. CombiDERM® NA dressing may be used as a primary dressing or as a secondary dressing in combination with other appropriate dressings such as, but not limited to, AQUACEL®, AQUACEL® Ag, KALTOSTAT® or DuoDERM® Hydroactive® Gel dressing. To remove, carefully lift the dressing away from the wound.

*Please refer to pack insert for full instructions prior to use.

Specifications

Size	Dressings Per Box	Product Code
7.5 cm x 7.5 cm	10	187774
14 cm x 14 cm	10	187773
15 cm x 25 cm	5	187775

ARTG Number: 158691

References

1. Hudak J, Constantine B, Buglino D, Eberhardt D. An in vitro evaluation of the fluid absorption capabilities under compression dressing. Data on file, ConvaTec. (1998).
2. Langmaid S. Utilizing Hydrofiber® technology in conjunction with an absorbent hydrocolloid dressing to prevent peri-wound maceration and contamination in moderate to heavily exuding pressure ulcers. Data on file, ConvaTec. (2005).
3. Sturmeijer J. The use of CombiDERM® on a mixed aetiology leg ulcer. Data on file, ConvaTec. (1996).
4. Murray M, Flores O, Sinkovic S. Clinical performance of CombiDERM® ACD™ in the management of pressure ulcers in a home care setting. Data on file, ConvaTec. (1996).



Clinical Benefits

- Supports moist wound healing environment
- Non-adhesive dressing allowing for easy application and atraumatic removal^{2,3}
- Poly-acrylate granules absorb and lock in exudate reducing the risk of maceration²
- Non-adherent wound contact layer allowing for atraumatic removal³
- Effective under compression when tested against comparative products*¹

*in vitro.

Indications

CombiDERM® NA dressing is indicated for the management of moderately to heavily exuding wounds.

- Chronic wounds: leg ulcers, diabetic ulcers, pressure ulcers⁴
- Acute wounds: abrasions, lacerations, surgical wounds
- Protection of fragile, sensitive surrounding skin

Call your local ConvaTec Territory Manager or ConvaTec Support Centre for further information:

Australia: 1800 335 276 or New Zealand: 0800 441 763

www.convatec.com

ConvaTec (Australia) Pty Limited. ABN 70 131 232 570. Unipark Monash, Building 2, Ground Floor, 195 Wellington Road, Clayton VIC 3170 Australia. PO Box 63, Mulgrave, Vic 3170. Phone: (03) 9239 2700 Facsimile: (03) 9239 2743. ConvaTec (New Zealand) Limited. AK2135265 Level 4, 369 Queen Street, Auckland 1010 New Zealand. PO Box 62663, Kalmia Street, Auckland 1544 New Zealand. Phone: (09) 306 8833 Facsimile: (09) 306 8831.

®/™Indicates trademark of ConvaTec Inc. © 2009 ConvaTec Inc. AP-006691-AU. March 2009.

Our world is what we make of it™