

DuoDERM[®]

Hydrocolloid Paste

Product Description

Moisture Retentive – Maintain moisture balance

DuoDERM[®] Paste interacts with wound exudate to form a moist wound environment that is supportive of the healing process by aiding autolytic debridement and allowing the non-traumatic removal of the dressing without damaging newly formed tissue¹.

Application*

Carefully cleanse the wound with saline and dry surrounding skin carefully to remove any greasy substances since these will interfere with the adhesion of the dressing.

Swab the piercing spike (on cap) with alcohol.

Remove the cap and puncture the tube's opening by inverting the cap and pressing on the metal membrane. NOTE: A small amount of clear liquid may be evident when the tube is opened. This is normal and is not an indication of product deterioration.

Squeeze the paste into the wound. Smooth, if necessary, using a sterile spatula or a sterile gloved finger. The surface of the paste should not protrude above the level of the surrounding skin (excess paste should be removed). NOTE: Do not allow paste on the surrounding normal skin, otherwise the adherence of the dressing will be impaired.

Select an appropriate moisture retentive dressing examples being Versiva[®] XC[®], DuoDERM[®] or CombiDERM[®] dressing of appropriate size so that it extends at least 3.2 cm onto normal skin. Apply in a rolling motion and hold gently in place to ensure good adhesion. NOTE: Refer to the instructions accompanying the Dressing for further information.

*Please refer to the pack insert for full instructions prior to use.



Clinical Benefits

- Interacts with wound exudate to form a moist wound environment¹
- Aids autolytic debridement¹
- Allows for the non-traumatic removal of the dressing without damaging newly formed tissue¹
- Provides a moisture retentive contact layer between wound bed and secondary dressing¹

Specifications

Size	Tubes Per Box	Product Code
30 grams	1	187930

ARTG Number: 157333

Reference

1. DuoDerm Hydroactive Paste [package insert]. Skillman, NJ: Convatec; 2004.

Indications

DuoDERM[®] paste is intended to be used in association with a secondary dressing such as DuoDERM[®] dressings for the management of exuding dermal ulcers, including full-thickness wounds, such as leg ulcers, pressure ulcers, and diabetic ulcers. The paste aids in the absorption of excessive exudate and extends the life of the secondary dressing.

Call your local ConvaTec Territory Manager or ConvaTec Support Centre for further information:

Australia: 1800 335 276 or New Zealand: 0800 441 763

www.convatec.com

ConvaTec (Australia) Pty Limited. ABN 70 131 232 570. Unipark Monash, Building 2, Ground Floor, 195 Wellington Road, Clayton VIC 3170 Australia. PO Box 63, Mulgrave, Vic 3170. Phone: (03) 9239 2700 Facsimile: (03) 9239 2743. ConvaTec (New Zealand) Limited. AK2135265 Level 4, 369 Queen Street, Auckland 1010 New Zealand. PO Box 62663, Kalmia Street, Auckland 1544 New Zealand. Phone: (09) 306 8833 Facsimile: (09) 306 8831.

®/™Indicates trademark of ConvaTec Inc. © 2009 ConvaTec Inc. AP-006695-AU. March 2009.

Our world is what we make of it™

