



INVISICLOSE®

HOW TO USE THE INVISICLOSE® CLIPLESS TAIL CLOSURE


1 CLOSING THE POUCH

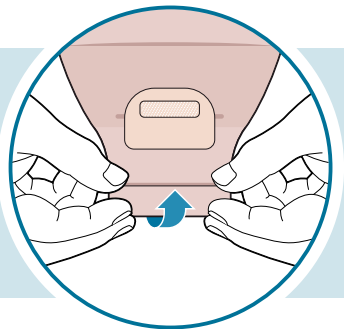

Step 1: Holding the outlet end strips, fold the end up towards you until the interlocking closures line up.

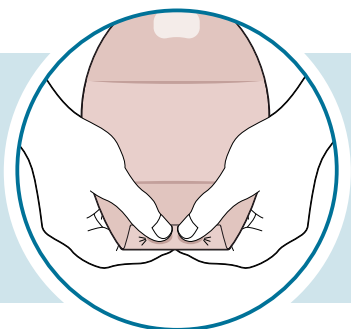



Step 2: Press the interlocking closures together across the width of the tail. Press and pinch across the closure until you feel them close.

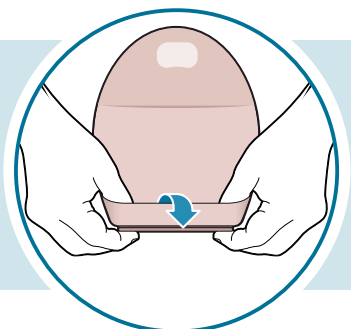

Step 3: Place your thumbs into the Lock-it-Pocket™ and hold the folded end of the tail between your thumbs and your index fingers.

Step 4: Turn the Lock-it-Pocket™ inside out to hide the tail inside the pocket.

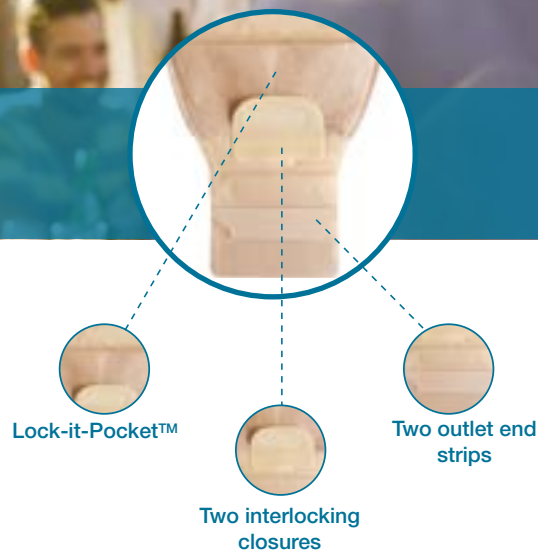

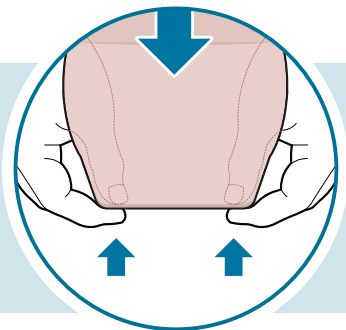


Step 1: Reverse the Lock-it-Pocket™ to reveal the tail.




Step 2: To empty the pouch, tilt the tail up towards your body. Peel open the foam closure and unravel the tail.

Tips for cleaning the tail of the pouch:

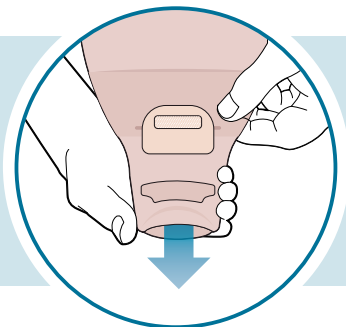

Step 3: Holding the pouch tail with 1 hand, open the tail by pushing in both ends of the outlet end strips with the other hand to drain the pouch.

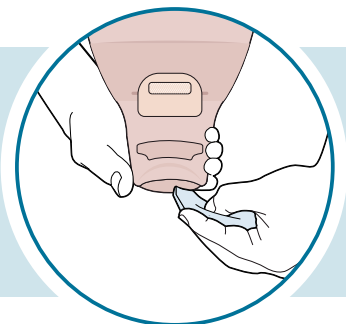

Step 4: Clean the inside surface of the pouch tail with toilet tissue or a wipe. The pouch tail can also be cleaned gently with pH neutral soap and water.



For more information, please call our Customer Relations Center (Registered Nurses on staff) at **1-800-465-6302**, Monday through Friday, 8:00 AM to 6:00 PM (EST), or visit our Web Site at **www.convatec.ca**

Please refer to the package insert for complete instructions for use.