

Product Description

Exudate Management – For moderately to heavily exuding wounds

KALTOSTAT® dressing is an absorbent, calcium sodium alginate (80% Ca and 20% Na) with a high Gularonic acid content. When in contact with wound exudate, calcium ions in the dressing exchange with sodium ions in the exudate causing the dressing to transform from a dry, fibrous state to a firm, moist gel. The dressing provides a moist environment helping to create optimal wound healing conditions and maintains its integrity allowing for easy, atraumatic removal¹.

KALTOSTAT® dressing is also designed to promote haemostasis, providing a matrix to support blood clot formation².

Application*

Carefully cleanse the wound with saline and dry surrounding skin.

KALTOSTAT® dressing should be folded or cut to the shape of the wound and applied dry directly onto the wound. For moderately exuding wounds, Versiva® XC®, DuoDERM® or CombiDERM® dressings, amongst others, can be used as a secondary dressings over KALTOSTAT® dressing. If highly exuding, a secondary absorbent pad should be used for additional absorbency and secured in place.

*Please refer to pack insert for full instructions prior to use.

Specifications

Size	Dressings Per Box	Product Code
5 cm x 5 cm	10	168210
7.5 cm x 12 cm	10	168212
10 cm x 20 cm	10	168214
15 cm x 25 cm	10	168215
2g rope	5	168117

RPBS Codes

Size	RPBS Code	Product Code
5 cm x 5 cm	4699H	168210
7.5 cm x 12 cm	4683L	168212
2g rope	4832H	168117

ARTG Number: 158719

References

1. Harder FP. Wound Treatment in Diabetic Subjects: the Use of the Calcium Sodium Alginate Kaltostat. Postar. Ricklinger Stadtweg 33, Hanover.
2. Sirmanna KS, Todd GB, Madden GJ. A randomised study to compare calcium sodium alginate fibre with two commonly used materials for packing after nasal surgery. Clinical Otolaryngology. 1992, 17; 237-239.



Clinical Benefits

- Supports moist wound healing environment
- Non-adherent to wound surface allowing for easy, non-traumatic removal¹
- Longer wear time than conventional dressings, therefore cost-effective¹
- Haemostatic – helps stop minor bleeding²

Indications

KALTOSTAT® dressing is indicated as a primary dressing for moderate to highly exuding wounds and for wounds with minor bleeding.

- Chronic wounds: leg ulcers, pressure ulcers, diabetic ulcers and fungating lesions
- Acute wounds: donor sites, abrasions, lacerations and post-surgical wounds

Call your local ConvaTec Territory Manager or ConvaTec Support Centre for further information:

Australia: 1800 335 276 or New Zealand: 0800 441 763

www.convatec.com

ConvaTec (Australia) Pty Limited. ABN 70 131 232 570. Unipark Monash, Building 2, Ground Floor, 195 Wellington Road, Clayton VIC 3170 Australia. PO Box 63, Mulgrave, Vic 3170. Phone: (03) 9239 2700 Facsimile: (03) 9239 2743. ConvaTec (New Zealand) Limited. AK2135265 Level 4, 369 Queen Street, Auckland 1010 New Zealand. PO Box 62663, Kalmia Street, Auckland 1544 New Zealand. Phone: (09) 306 8833 Facsimile: (09) 306 8831.

®/™Indicates trademark of ConvaTec Inc. © 2009 ConvaTec Inc. AP-006698-AU. March 2009.

Our world is what we make of it™

