





ESTEEM SYNERGY™


WITH MOULDABLE BASEPLATE

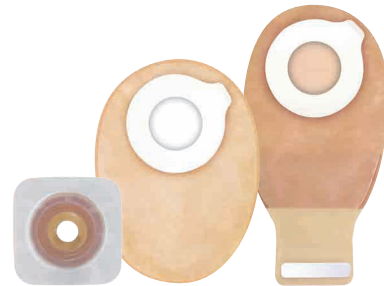


1 Mouldable baseplate application

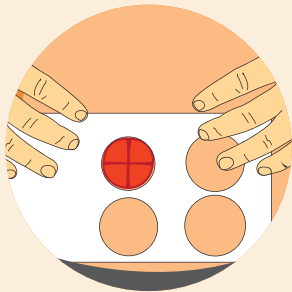


Measure the stoma and select the appropriate size Mouldable baseplate.

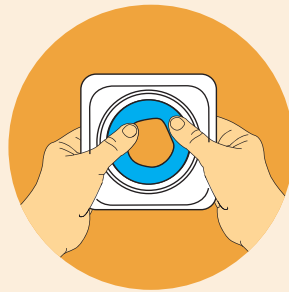


Roll and mould the disc to the size of the stoma. **Do not stretch, pull or cut.** Pinch into place once desired size is reached.

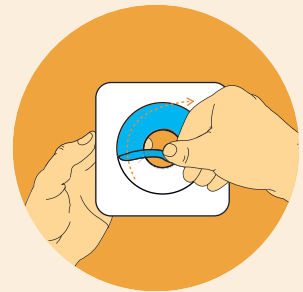


Peel off clear release liner and position the circular opening over your stoma.

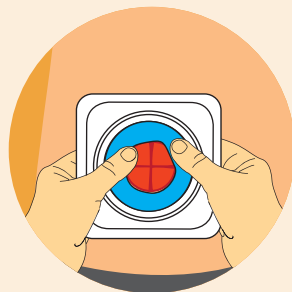

Press the baseplate into place and if necessary re-adjust the rolled Mouldable Technology™ to create a snug fit.

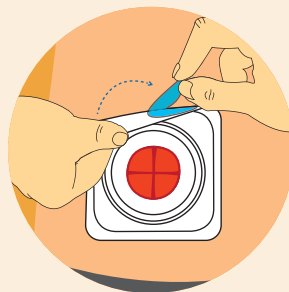

Peel off the white release paper from the tape border. Smooth and hold the baseplate in place for 30 seconds.

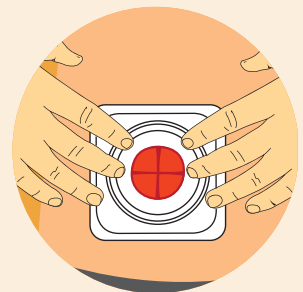

Press gently against skin, smoothing out any wrinkles.

3 Closing the pouch



Fold the end up towards you until the interlocking closures line up. Press and pinch around the closure to secure.



Turn the Lock-it-Pocket™ inside-out to tuck tail inside.


TIP

If using a drainable pouch over a closed pouch, we recommend you close the tail closure before applying your pouch to the baseplate.

2 Applying and wearing the pouch



Separate the side of the pouch to allow some air to enter.

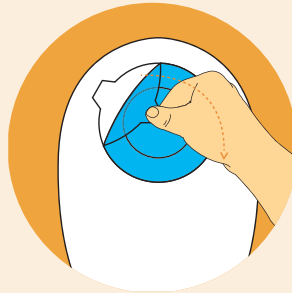


Remove the release paper from the adhesive ring on the pouch.




Softly fold the adhesive ring, apply the lower half to the bottom of the landing zone, then "roll or slide" your fingers over the upper portion to secure the pouch.

4 Emptying and cleaning a drainable pouch

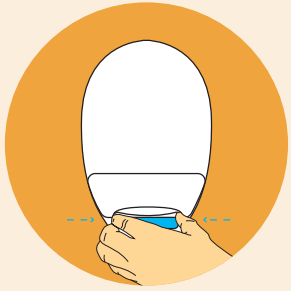

To empty pouch, tilt the tail up towards your body. Peel open the closure and unravel tail. Unlock pouch closure and open the tail by pushing in both ends of the outlet end-strips with fingers; drain pouch.

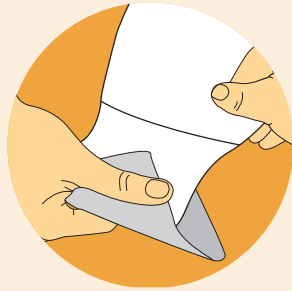


Tips for cleaning the pouch: Support the pouch with one hand and wipe tail of pouch in firm downward motion with toilet tissue in other hand.




Clean the inside surface of tail with toilet tissue or baby wipe.

5 Removing the pouch

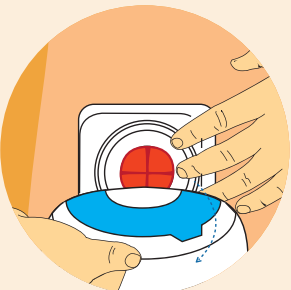

To remove the pouch from the baseplate firmly pull downward on the pouch tab and lift the pouch off. Only remove the pouch if replacing with a new, clean pouch.

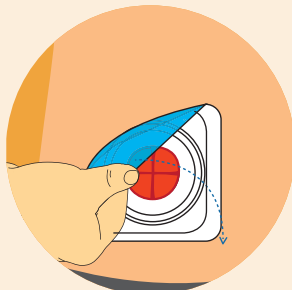

To remove baseplate, gently lift the corner and peel away whilst pressing down on the skin. Only remove the baseplate if replacing with a new, clean baseplate.



TIP
To ensure easy removal of the baseplate use ConvaTec Niltac™ Adhesive Removal where appropriate.

For more information, call us on - 0800 282 254