ENGLISH (en)

Indications for use:

The TruSteel™ infusion set is indicated for subcutaneous infusion of insulin administered by an external pump. The infusion set is indicated for single use.

INTENDED USE

The infusion set is indicated for subcutaneous infusion of insulin in the treatment of diabetes. The infusion set has been tested to be compatible with insulin approved for subcutaneous infusion.

PRODUCT DESCRIPTION

TruSteel™ is a 90-degree, stainless steel needle infusion set with a tubing connection.

CONTRAINDICATIONS

TruSteel™ is neither intended nor indicated for intravenous infusion (IV) of medication, including blood or blood products.

WARNINGS

■ Carefully examine the instructions for use before inserting TruSteel™. Failure to follow instructions may result in pain or injury.

■ TruSteel™ infusion sets with tlock™ tubing connectors should only be used with Tandem cartridges featuring the tlock™ connector.

■ The TruSteel™ infusion set is only sterile and non-pyrogenic if the packaging is undamaged and unopened. Do not use it if the packaging is already opened or is damaged.

■ Inaccurate medication delivery, site irritation or infection may result from improper insertion or maintenance of your infusion site.

■ When using TruSteel™ for the first time, do so in the presence of a healthcare provider.

■ Do not reuse the infusion set. Reuse of the infusion set may cause infection or irritation and may lead to inaccurate medication delivery.

■ Carefully remove the Needle guard before inserting the infusion set.

■ Do not bend the needle prior to insertion, and do not use the infusion set if the needle is bent or has been damaged.

PRECAUTIONS

■ Never prime the infusion set or attempt to free a clogged tubing while the infusion set is inserted. You may accidentally inject too much medication.

■ Check the infusion site frequently to ensure that the steel needle remains firmly in place. Replace with a new infusion set at a new site, if the infusion set is not in place.

■ TruSteel™ is a single-use item which must be disposed of after usage. Do not clean or re-sterilize.

■ Choose insertion sites as recommended by your healthcare provider.

■ Do not in any way put disinfectants, perfumes, deodorants or other products containing disinfectants in contact with the tubing connector and the tubing, as these may impact the integrity of the infusion set.

■ Replace with a new infusion set if the adhesive tape becomes loose.

■ Change the infusion set every one to two days, or as recommended by your healthcare provider.

■ Upon removal, carefully loosen the edges of the adhesive tape, and remove the infusion set slowly by pulling straight out. Make sure that the needle is still attached to the adhesive and pulled completely out of the skin. If the needle is missing, immediately contact your healthcare provider for assistance.

■ If the infusion site becomes infected or inflamed, replace with a new infusion set and use a new site until the first site has healed.

■ Use aseptic techniques when temporarily disconnecting TruSteel™. Consult with your healthcare provider on how to compensate for missed medication when disconnected.

RECOMMENDATIONS

■ Check the blood glucose level one to three hours after introducing TruSteel™ and check the infusion site several times a day. Measure your blood glucose regularly.

■ Do not change infusion set just prior to bedtime, unless blood glucose can be checked one to three hours after insertion.

■ If blood glucose level becomes unexplainably high, or an occlusion alarm occurs, check for clogs or leaks. If in doubt, change the infusion set. Discuss plans for rapid replacement of insulin with your healthcare provider should any of these problems arise. Test blood glucose level to make sure problem is corrected.

■ Carefully monitor blood glucose levels while disconnected and after reconnecting.

■ If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

STORAGE & DISPOSAL

■ Store infusion sets in a cool, dry location at room temperature. Do not store infusion sets in direct sunlight, or in high humidity.

■ Store and handle insulin according to the manufacturer's instructions.

■ Dispose of the infusion set in a proper sharps container according to local laws, to avoid risk of needle stick.

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