



neria™ guard infusion sets

neria™ guard is a soft cannula infusion set with an integrated insertion device for simple insertions by the touch of a button. A retractable needle may eliminate accidental needle sticks and increase insertion technique comfort.

Product Specifications

Cannula:	Soft
Insertion angle:	90 degrees (straight in)
Disconnection:	Yes, at site
Introducer needle gauge:	G27
Soft cannula diameter:	0.68 mm (≈ G23)


Adhesive:	Built-in, skin friendly non-woven polyester, acrylic adhesive
Needle safety:	Yes, passive (Needle not visible before, during and after insertion)
Insertion device:	Yes, incorporated in the product

Soft cannula lengths	Tubing lengths	Priming volumes
6 and 9 mm	12, 60, 80 and 110 cm	12 cm ~ 0.04 ml 60 cm ~ 0.10 ml 80 cm ~ 0.12 ml 110 cm ~ 0.15 ml


The neria™ guard infusion sets are sold in boxes containing 10 pieces. Separate cannulae and tubing parts are available in boxes containing 10pcs/box.



neria™ guard infusion set user steps

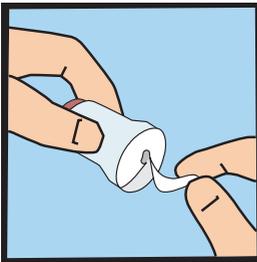


- 1.** Pull gently to remove the paper from the adhesive tape.

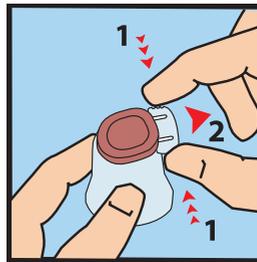


- 2.** Remove the safeguard from the insertion device by gently squeezing the sides of the safeguard and pulling it straight out.




- 3.** Position the insertion device on the skin and press the red activation button completely down to insert the infusion set.

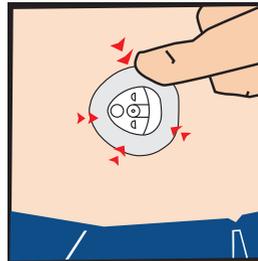

- 4.** Press the adhesive tape onto the skin.

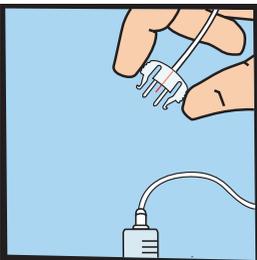

- 5.** Connect to the pump and prime the tubing according to instructions provided by the pump manufacturer.

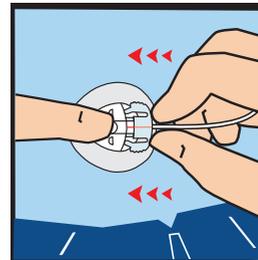

- 6.** Place a finger on the cannula housing while pushing the site connector straight in until you hear a "click".