


AN ADVANCE IN PATIENT CARE

From the leading innovators
in fecal management...

Flexi-Seal® SIGNAL™ FMS*

Signal Indicator Educator

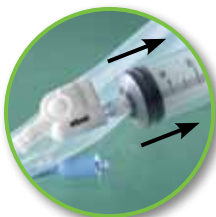

1 Attach the precisely labeled Flexi-Seal® FMS syringe to the white inflation port with the Signal fill indicator (marked 45 mL), and pull back the plunger to remove all excess air from the balloon.

2 Fill the empty syringe to inflate the retention balloon with 45 mL of tap water or saline. **The balloon should never be inflated with more than 45 mL of fluid.**



3 Slowly depress the syringe plunger.



4 The Signal indicator has an oval inflation chamber, or bubble, that will expand as fluid is injected. Once the balloon has reached the optimal fill level, the indicator bubble will pop.



5 Once the Signal indicator bubble has popped, a "best practice" approach is to pull back the syringe plunger to deflate the bubble, and then reinflate it just to the point of popping. This ensures the balloon is filled to the appropriate volume (up to 45 mL) to prevent overinflation.



Flexi-Seal SIGNAL™
The seal of innovation™

* Please see complete Instructions for Use in Package Insert.

*Clinical observation
has shown that
an **optimal fill level**
minimizes the risk
of leakage**

**Talk to your ConvaTec representative to learn more
about the benefits of *new* Flexi-Seal® SIGNAL™ FMS
for you and your patients!**

* The Signal indicator could pop before 45 mL if the space available for the balloon within the rectal vault is smaller than the size of the balloon. Filling should stop when the indicator pops out and stays out. Never add more than 45 mL total.



Flexi-Seal *SIGNAL*™
The seal of innovation™

TM indicates trademarks of ConvaTec Inc.
Flexi-Seal and the Flexi-Seal logo are registered
trademarks of ConvaTec Inc.

© 2010 ConvaTec Inc.

AP-XXXX-US

February 2010

