# Flexi-Seal™ SIGNAL™

## Fecal Management System

### Directions for Use

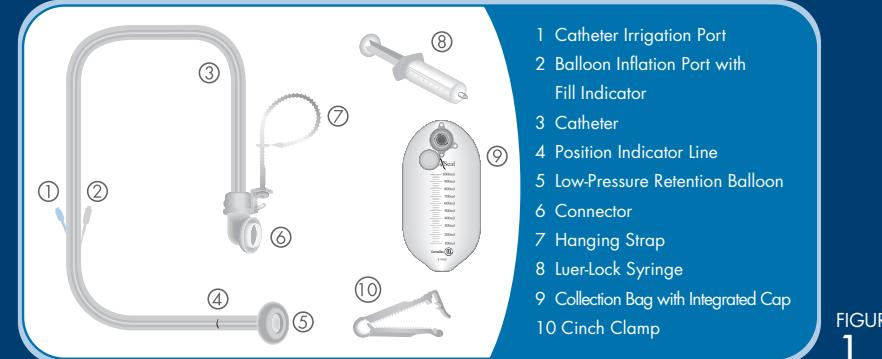
FIGURE 1

This system contains:

**1 soft catheter tube assembly**

**1 Luer-Lock Syringe**

**3 collection bags with filter**

**1 cinch clamp**

Non-Sterile | Single Patient Use Only | Does not contain natural rubber latex components  
Attention: See instructions for use | Keep dry. Avoid high humidity

**Caution:** United States federal law restricts this device to sale by or on the order of a physician.

## Flexi-Seal™ SIGNAL™ Fecal Management System

### PRODUCT DESCRIPTION

The Flexi-Seal™ SIGNAL™ Fecal Management System contains:

- 1 soft catheter tube assembly,**
- 1 Luer-Lock syringe,**
- 3 collection bags with filter** and
- 1 cinch clamp** (Figure 1)

The soft catheter is inserted into the rectum for fecal management to contain and divert fecal waste in order to protect the patient's skin and keep the bedding clean. There is a low-pressure retention balloon at the distal end and a connector for attaching the collection bag at the other end. There is a recess under the balloon for the clinician's finger allowing the device to be positioned digitally.

A blue and a white port are attached to the side of the catheter. The white port, marked " $\leq 45\text{ml}$ " is used to inflate the retention balloon after the device has been inserted into the patient's rectum. (Figure 2-1b) This white inflation port also provides a visual and tactile indication as to when the low pressure retention balloon is filled to its optimal volume. The blue port, marked "IRRIG." is used to flush the device if needed and administer medication, if prescribed. (Figure 2-1a)

### Indications

For use to manage fecal incontinence through the collection of liquid to semi-liquid stool and to provide access to administer medications.

### Contraindications

- 1. This product is not intended for use • for more than 29 consecutive days
- for pediatric patients

2. The Flexi-Seal™ SIGNAL™ Fecal Management System should not be used on individuals who • have suspected or confirmed rectal mucosal impairment, i.e. severe proctitis, ischemic proctitis, mucosal ulcerations

- have had rectal surgery within the last year
- have any rectal or anal injury
- have hemorrhoids of significant size and/or symptoms
- have a rectal or anal stricture or stenosis
- have a suspected or confirmed rectal/anal tumor
- have any in-dwelling rectal or anal device (e.g. thermometer) or delivery mechanism (e.g. suppositories or enemas) in place
- are sensitive to or who have had an allergic reaction to any component within the system

### Precautions and Observations

1. Close attention should be exercised with the use of the device in patients who have inflammatory bowel conditions or who have had rectal surgery. The physician should determine the degree and location of inflammation or extent of surgery (e.g. location of anastomosis) within the colon/rectum prior to considering use of this device in patients with such conditions.

2. Care should be exercised in using this device in patients who have a tendency to bleed from either anti-coagulant / antiplatelet therapy or underlying disease. If signs of rectal bleeding occur, remove the device immediately and notify a physician. Remove any indwelling or anal device prior to insertion of the Flexi-Seal™ SIGNAL™ FMS and do not insert any other devices into the rectum while the Flexi-Seal™ SIGNAL™ FMS is in place.

3. Notify a physician immediately if any of the following occur:

- rectal pain
- rectal bleeding
- abdominal symptoms such as distension/pain

4. Solid or soft-formed stool cannot pass through the catheter and will obstruct the opening. The use of the device is not indicated for solid or soft-formed stool.

5. Small amounts of moisture or seepage around the catheter is anticipated. To avoid skin irritation, initiate an appropriate institutional skin care protocol. At a minimum, the skin should be kept clean, dry and protected with a moisture barrier product.

6. If the catheter becomes blocked with feces, it can be rinsed with water using the irrigation port only (see Direction for Use "Irrigation of the Device"). Do not use the  $\leq 45\text{ml}$  white inflation port to irrigate. If obstruction of the catheter is due to solid stool, use of the device should be discontinued.

7. If the patient's bowel control, consistency and frequency of stool begin to return to normal, discontinue use of the device. As with the use of any rectal device, the following adverse events could occur:

- Leakage of stool around the device
- Rectal/anal bleeding due to pressure necrosis or ulceration of rectal or anal mucosa
- Perianal skin breakdown
- Temporary loss of anal sphincter muscle tone
- Infection
- Bowel obstruction
- Perforation of the bowel

8. This device is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

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## Directions for Use

### A. Preparation of Device

- In addition to the device system, gloves and lubricant will be required.
- Unfold the length of the catheter to lay it flat on the bed, extending the collection bag toward the foot of the bed.
- Securely attach the collection bag to the connector at the end of the catheter. (Figure 3)

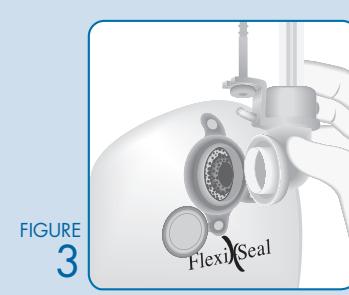
FIGURE 3

### B. Preparation of Patient

- Position the patient in left side-lying position; if unable to tolerate, position the patient so access to the rectum is possible.
- Remove any in-dwelling or anal device prior to insertion of the Flexi-Seal™ SIGNAL™ device.
- Perform a digital rectal exam to evaluate suitability for insertion of device.

### C. Insertion of Device

- Using the syringe provided, remove the air that is in the balloon by attaching the syringe to the white inflation port (marked " $\leq 45\text{ml}$ ") and withdrawing the plunger.

Remove the supplied syringe and fill it with 45ml of water or saline and connect the syringe to the white inflation port of the catheter.

Insert a lubricated gloved finger into the blue finger pocket for digital guidance during device insertion (the finger pocket is located above the position indicator line). (Figure 4)

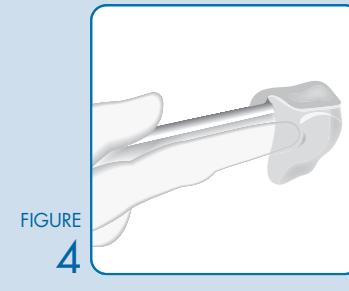
FIGURE 4

Coat the balloon end of the catheter with lubricant. Grasp the catheter and gently insert the balloon end through the anal sphincter until the balloon is beyond the external orifice and well inside the rectal vault. The finger may be removed or remain in place in the rectum during initial balloon inflation. (Figure 5)

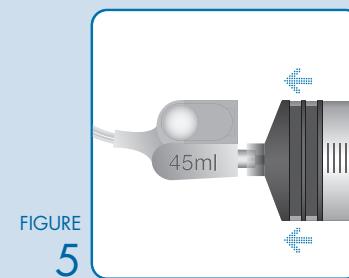
FIGURE 5

2. Inflate the balloon with up to 45ml of fluid by slowly depressing the syringe plunger. With the insertion finger removed, the SIGNAL™ dome will indicate once the balloon has reached the optimal fill level for the anatomy. (Figure 5) There may be cases where the SIGNAL™ dome will not indicate if the space in the rectum is large. **Under no circumstances should the balloon be inflated with more than 45ml of fluid.**

If the SIGNAL™ dome indicates at less than 30ml of fluid, withdraw the fluid and reposition the balloon in the rectal vault. After repositioning, fill the balloon as described above. Do not fill with more than 45ml of fluid.

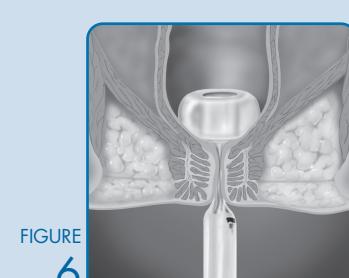
FIGURE 6

3. Remove the syringe from the inflation port, and gently pull on the soft catheter to check that the balloon is securely in the rectum and that it is positioned against the rectal floor. (Figure 6)

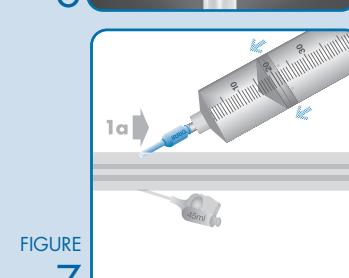
FIGURE 7

### D. Irrigation of the Device

To irrigate the device, fill the supplied syringe with water at room temperature. Attach the syringe to the blue irrigation port (marked "IRRIG.") and slowly depress the plunger. Do not irrigate through the white inflation port (marked " $\leq 45\text{ml}$ "). (Figure 7-1a)

Repeat the irrigation procedure as often as necessary to maintain proper functioning of the device. Flushing the device as described above is an optional procedure for use only when needed to maintain the unobstructed flow of stool into the collection bag. If repeated flushing with water does not return the flow of stool through the catheter, the device should be inspected to ascertain that there is no external obstruction (i.e., pressure from a body part, piece of equipment, or resolution of diarrhea). If no source of obstruction of the device is detected, use of the device should be discontinued.

### E. Maintenance of Device

Change the collection bag as needed. Discard used bags according to institutional protocol for disposal of medical waste. Observe the device frequently for obstructions from kinks, solid fecal particles or external pressure.

### F. Administration of Medication

- Attach the supplied syringe and flush the irrigation line with 10ml of water.

2. Prepare a new syringe with prescribed medication. Position the cinch clamp loosely on the catheter at the black indicator line. Connect syringe to the blue irrigation port (marked "IRRIG.") and administer medication.

3. To ensure delivery of medication into the rectum immediately flush the irrigation line with at least 50ml of water. Tighten the cinch clamp on the catheter to ensure no flow through the catheter. Allow the medication to dwell in the rectum for the desired amount of time as dictated by the prescribing physician.

4. Remove the cinch clamp. Attach a new syringe (not supplied) and flush the irrigation line with 10ml of water. Dispose of the syringe according to institutional policy.

### G. Sampling

To collect a sample from the catheter, open the sample port cap. Press the tip of a catheter syringe (not supplied) through the slit inside of the sampling port to access the interior of the catheter. Withdraw the syringe plunger to collect the sample. Withdraw the syringe and close the sampling port cap.

### H. Removal of Device

To remove the catheter from the rectum, the retention balloon must first be deflated. Attach the supplied syringe to the white inflation port (marked " $\leq 45\text{ml}$ ") and slowly withdraw all fluid from the retention balloon.

Disconnect the syringe and discard. Grasp the catheter as close to the patient as possible and slowly remove from the anus. Dispose of the device in accordance with institutional protocol for disposal of medical waste.

## General Guidelines

The device may be changed as needed to perform normal patient assessment.

The device is not intended for use for more than 29 consecutive days.

If the product packaging is damaged, do not use.

If further information or guidance is needed, please contact Convatec Inc. at the address below.

Convatec Inc.  
200 Headquarters Park Drive  
Skillman, NJ 08558  
1-800-422-8811

Convatec Canada Ltd.  
Montréal, Québec, Canada  
1-800-465-6302

[www.convatec.com](http://www.convatec.com)

© 2012 Convatec Inc.

TM indicates trademarks of Convatec Inc. Convatec, the Convatec Logo, Flexi-Seal, the Flexi-Seal Logo and Flexi-Seal SIGNAL are registered trademarks of Convatec Inc. in the U.S.

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ARTWORK SPECIFICATION	
<b>Artcode</b>	<b>SI2163</b>
<b>Artwork Description</b>	<b>INSERT Flexi-Seal SIGNAL FMS Rx (1x1PK) US</b>
<b>ICC</b>	<b>418000</b>
<b>Component Type</b>	<b>Insert</b>
<b>REQUIREMENT</b>	<b>SPECIFICATION</b>
Bucket	US
Local Code(s)	NA
Dieline	NA
Dimensions	5" x 7.75" (folded), 15" x 22.5" (flat)
Size Code	NA
Color(s) / Colour(s)	PMS 3015, PMS 368, PMS 541, Black
Specification Reference(s)	TF23-028
Bar Code: UPC(A)*	NA
Bar Code: PZN – Code 39*	NA
Bar Code: JAN-13*	NA
* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade "C" or better.	
<b>DATE / ARTCODE: HISTORY</b>	
31-OCT-2011 / SI2163 New Art - Insert	

Document ID : SI2163

Signed by:	Date: (mm/dd/yyyy hh:mm:ss AM/PM)	Justification:
------------	---	----------------

George R Danovsky      11/17/2011 02:25:15 PM      Change Control Approval

Peter Sauer      11/10/2011 11:21:53 AM      Final Quality Approval