
AutoSoft™ 90

Instructions for Use

Instrucciones de uso / Mode d'emploi / Anwendungshinweise
/ Istruzioni per l'uso / Gebruiksaanwijzing / Brugsanvisning /
Käyttöohje / Bruksanvisning / Bruksanvisning / Instrukcja użycia
/ Návod k použití / Οδηγίες χρήσης / Használati útmutató /
Navodila za uporabo / 사용 설명서 / 使用说明 / إرشادات الاستعمال /
Indicações de utilização / Инструкции по применению

TANDEM™
DIABETES CARE

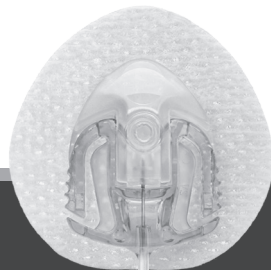EN	Do not re-use
ES	No reutilizar
FR	Ne pas réutiliser
DE	Nicht wiederverwenden
IT	Non riutilizzare
NL	Niet hergebruiken
DA	Må ikke genbruges
FI	Ei saa käyttää uudelleen
NO	Må ikke brukes om igjen
SV	Får ej återanvändas
PL	Nie używać ponownie
CS	Nepoužívejte opakovaně
EL	Μην επαναχρησιμοποιείτε
HU	Nem újrafelhasználható
SL	Samo za enkratno uporabo
KO	재사용하지 마십시오.
ZH	请勿重复使用
AR	لا تُعيد استخدامه
PT	Não reutilizar
RU	Для одноразового использования

EN	Caution: see instructions for use
ES	Precaución: ver instrucciones de uso
FR	Attention : voir notice d'utilisation
DE	Achtung: siehe Gebrauchsanweisung
IT	Attenzione: consultare le istruzioni per l'uso
NL	Let op: zie gebruiksaanwijzing
DA	Forsigtig: Se brugsanvisningen
FI	Vaara: katso käyttöohjeet
NO	OBS! se bruksanvisningen
SV	Försiktighet: se bruksanvisningen
PL	Przeostaga: patrz instrukcja użycia
CS	Upozornění: viz návod k použití
EL	Προσοχή: ανατρέξτε στις οδηγίες χρήσης
HU	Vigyázat: olvassa el a használati útmutatót
SL	Pozor: glejte navodila za uporabo
KO	주의: 사용 설명서 참조
ZH	注意事项: 见使用说明
AR	تحذير: راجع إرشادات الاستعمال
PT	Atenção: Ver as instruções de utilização
RU	Внимание: см. инструкцию по применению

EN	Consult instructions for use
ES	Consultar instrucciones de uso
FR	Consulter le mode d'emploi
DE	Anwendungshinweise beachten
IT	Consultare le istruzioni per l'uso
NL	Raadpleeg de gebruiksaanwijzing
DA	Se brugsanvisningen
FI	Lue käyttöohje
NO	Se bruksanvisningen
SV	Se bruksanvisningen
PL	Zapoznać się z instrukcją użycia
CS	Přečtete si návod k použití
EL	Συμβουλευτείτε τις οδηγίες χρήσης
HU	Olvassa el a használati útmutatót
SL	Glejte navodila za uporabo
KO	사용 설명서를 참고하십시오.
ZH	查阅使用说明
AR	راجع إلى إرشادات الاستعمال
PT	Consultar as instruções de utilização
RU	См. инструкцию по применению

Rx ONLY

EN	Prescription required, does not apply in all geographies
ES	La prescripción del producto no es aplicable en todas las áreas geográficas
FR	Soumis à prescription (ne concerne pas tous les marchés)
DE	Verschreibungspflichtig, gilt nicht für alle Regionen
IT	Prescrizione medica richiesta, non si applica in tutte le aree geografiche
NL	Uitsluitend op voorschrift, niet in alle landen van toepassing
DA	Receptpligtig; gælder ikke i alle lande
FI	Resepti vaaditaan; ei koske kaikkia alueita
NO	Resept påkrevd, gjelder ikke i alle geografiske områder
SV	Receptbelagt läkemedel, gäller inte i alla områden.
PL	Produkt dostępny na receptę; nie dotyczy niektórych lokalizacji
CS	Pouze na lékařský předpis. Neplatí ve všech zemích
EL	Απαιτείται συνταγή, δεν ισχύει σε όλες τις περιοχές
HU	Receptre kapható (nem minden országbán)
SL	Samo na recept, ne velja za vsa območja
KO	처방전이 필요하며, 모든 지역에서 사용할 수는 없습니다.
ZH	须在医生处方下使用，不适用于所有地区
AR	يُزَم وجود وصفة طبية، لا ينطبق في بعض المناطق الجغرافية
PT	Requer prescrição, não se aplica a todas as áreas geográficas
RU	Отпускается по рецепту (относится не ко всем регионам)

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Indications for use

The AutoSoft™ 90 infusion set is indicated for the subcutaneous infusion of insulin administered by an external pump. The infusion set is indicated for single use.

INTENDED USE

The infusion set is indicated for subcutaneous infusion of insulin in the treatment of diabetes. The infusion set has been tested to be compatible with insulin approved for subcutaneous infusion.

PRODUCT DESCRIPTION

AutoSoft™ 90 is an infusion device which combines an infusion set with an aid for insertion (serter). As the infusion set is an integrated part of the sserter, AutoSoft™ 90 is delivered assembled and ready for use as a single, sterile unit.

CONTRAINDICATIONS

AutoSoft™ 90 is neither intended nor indicated for use in intravenous infusion of blood or blood products.

WARNINGS

- Carefully examine the instructions for use

before inserting AutoSoft™ 90. Failure to follow instructions may result in pain or injury.

- AutoSoft™ 90 infusion sets with t:lock™ tubing connectors should only be used with Tandem cartridges featuring the t:lock™ connector.
- AutoSoft™ 90 is only sterile and non-pyrogenic if the insertion device is undamaged and unopened. Do not use if the insertion device is already opened or has been damaged. Ensure sterility by checking if the sterile paper and tamperproof seal are unbroken.
- When using AutoSoft™ 90 for the first time, do so in the presence of a healthcare provider. Since different individuals have varying amounts of subcutaneous layer, ensure that an appropriate cannula length is chosen to fit your needs.
- Inaccurate medication delivery, site irritation or infection may result from improper insertion or maintenance of your infusion site.
- Do not re-insert the introducer needle into the infusion set. This could cause a tearing or damage of the soft cannula and may lead to unpredictable medication flow.
- Reuse of the infusion set may cause site

irritation or infection and may lead to inaccurate medication delivery.

- Be sure that the needle guard is removed before insertion is made.
- If the soft cannula bends during insertion, apply a new AutoSoft™ 90 immediately at a new site.

PRECAUTIONS

- Never try to fill or free a clogged tubing while the infusion set is inserted. This may result in unpredictable medication flow.
- When unwinding, release the tubing with caution as a hard pull of the tubing can result in damage to the infusion device. Ensure that the infusion set is properly in place in the sserter when the tubing is fully released (see figure 8 on page 88).
- Do not leave air in the infusion set. Make sure to fill the tubing completely. Consult pump manufacturer's instructions for use.
- AutoSoft™ 90 is a single-use device and should be disposed of immediately after use. Do not clean or re-sterilize.
- Do not in any way put disinfectants, perfumes, deodorants, cosmetics or other products containing disinfectants in contact with the connector or the tubing. This may impact the integrity of the

infusion set.

- Replace the infusion set if the adhesive tape becomes loose or is displaced from its original site. Since the cannula is soft, it will not cause any pain if it becomes dislodged, and this may take place without notice. The soft cannula must always be completely inserted to receive the full amount of medication.
- Replace the infusion set every two to three days, or per your healthcare provider's instructions.
- If your insertion site becomes irritated or infected, replace infusion set and use new site until the first site has healed.
- Wash your hands before disconnecting and reconnecting the infusion set. Consult your healthcare provider on how to compensate for missed medication when disconnected.
- Never point a loaded insertion device towards any body part, where insertion is not desired.

RECOMMENDATIONS

- Check your blood glucose level one to three hours after insertion of the AutoSoft™ 90. Measure your blood glucose on a regular basis. Discuss this

with your healthcare provider.

- Do not change your infusion set just prior to bedtime, unless blood glucose can be checked one to two hours after insertion.
- If your blood glucose level becomes unexplainably high or an occlusion alarm occurs, check for clogs and leaks. If in doubt, change your infusion set since the soft cannula could be dislodged or partially clogged. Discuss plan to bring down your blood glucose level with your healthcare provider should any of these problems arise. Test your blood glucose level to make sure the problem is corrected.
- Carefully monitor your blood glucose level when disconnected and after reconnecting.
- If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

STORAGE & DISPOSAL

- Store infusion sets in a cool, dry location at room temperature. Do not store infusion sets in direct sunlight, or in high humidity.
- Store and handle insulin according to the

manufacturer's instructions.

- Put the lid back on the insertion device and dispose in a proper sharp container according to local laws, to avoid risk of needle stick.
- Dispose of a used infusion set according to local regulations for biohazardous waste.

Preparation and insertion

Read all instructions carefully before using the AutoSoft™ 90 infusion set. For information about pump therapy, possible errors, and potential risks and their causes, consult your pump manufacturer's instructions for use before connecting AutoSoft™ 90. When using AutoSoft™ 90, follow proper hygiene procedures. Please see the corresponding illustrations on page 88-90.

CONTENTS

- A. Lid
- B. Needle guard
- C. Soft cannula
- D. Adhesive tape
- E. Cannula housing
- F. Connector needle
- G. Tubing
- H. Introducer needle
- I. Insertion device
- J. Sterile paper
- K. Three raised dots – press here when lifting the lid off (see figure 6)
- L. Lined indentations – press here when loading AutoSoft™ 90 (see figure 10)
- M. Round indentations – press here simultaneously when inserting

- AutoSoft™ 90 (see figure 12)
- N. Disconnect cover (see figure B)

INSTRUCTIONS

Unfold the cover of this booklet to view images that correspond to the steps below.

1. Wash your hands before inserting AutoSoft™ 90.
2. Suggested insertion sites. Choose an insertion site as recommended by your healthcare provider. Do not use an area right next to your previous insertion site.
3. Clean the area of insertion with a disinfectant as directed by your healthcare provider. Be sure the area is air dry before inserting AutoSoft™ 90.
4. Pull the red tape to remove the seal.
5. Remove the sterile paper.
6. Press the three raised dots on each side of the lid firmly with one hand and lift up the lid with the other hand.
CAUTION: Be careful not to bend or touch the AutoSoft™ 90 introducer needle when preparing the inserter and infusion set.
IMPORTANT NOTE: If the infusion set is not securely placed in the insertion device with the needle pointing straight ahead prior to insertion, pain or minor injury may occur. To avoid this, please ensure that

the following steps are taken gently so the infusion set is not loosened / removed by accident.

7. **Unwinding the tubing:** Carefully twist the beginning of the tubing out of the slot. Pull up gently. Now unwind the tubing from the insertion device by pulling it carefully upwards.
CAUTION: Do not touch the AutoSoft™ 90 introducer needle when unwinding the tubing. Be careful not to pull too hard when the tubing is fully released. You could accidentally remove the infusion set from the inserter.
8. Connect AutoSoft™ 90 to a cartridge and the pump. Prime infusion set. Ensure there are no air bubbles in the tubing. Consult pump manufacturer's instructions for use.
CAUTION: When priming the infusion set, hold AutoSoft™ 90 with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.
9. Pull up gently to remove the adhesive backing paper.
10. **Prepare the inserter:** Place fingers on lined indentations. Press lined indentations on each side. Pull spring up until you hear a "click".
11. Carefully remove the needle guard by

gently twisting and then pulling it. Check that the soft cannula does not exceed the introducer needle.

CAUTION: Before positioning AutoSoft™ 90 over a clean insertion site, place the tubing in the slot to ensure that the tubing is not caught underneath the infusion set during insertion.

12. **Insert AutoSoft™ 90:** Position AutoSoft™ 90 over the insertion site. Simultaneously press the round indentations on both sides of the insertion device to insert AutoSoft™ 90.
13. Push gently on the center of the inserter to secure the adhesive tape on the skin.
14. Remove the inserter/introducer needle by grasping the center and pulling it gently back. Gently massage the tape onto the skin. Fill the empty space in the cannula with the following amount of insulin:

U-100 INSULIN:

6 mm: 0.3 units (0.003 ml)

9 mm: 0.5 units (0.005 ml)

15. Push the lid back in place until you hear a “click.”

DISCONNECT

- A. AutoSoft™ 90 allows you to temporarily disconnect your pump without having to change the infusion set. Hold the adhesive in place by placing a finger just in front of the cannula housing, and gently squeeze the sides of connector. Pull the connector needle straight out from the cannula housing.
- B. Insert the disconnect cover into the cannula housing until you hear a “click”.

RECONNECT

- C. If necessary, prime the infusion set until medication dispenses from the needle.
CAUTION: When priming the infusion set, hold AutoSoft™ 90 with the needle pointing down.
- D. Remove the disconnect cover from the cannula housing by pressing the grips. Place a finger on the adhesive tape just in front of the cannula housing.
- E. Push the connector needle straight in until you hear a “click”.

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© 2018 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, AutoSoft and t:lock are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. AW-1000184_D
Safety Service GmbH