





NATURA[®]


with Pre-Cut & Cut-To-Fit Skin Barriers

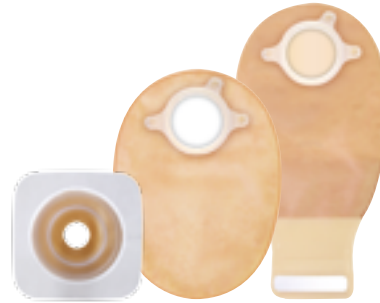


1 Skin barrier application

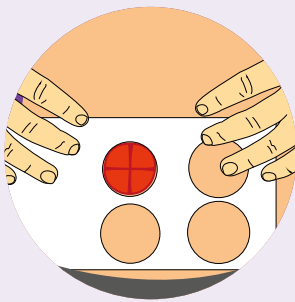


Measure the stoma and select the appropriate size skin barrier.

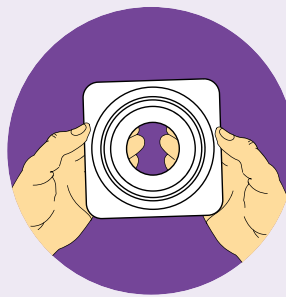

Pre-cut skin barriers
Select skin barrier with pre-cut opening slightly larger than your stoma.

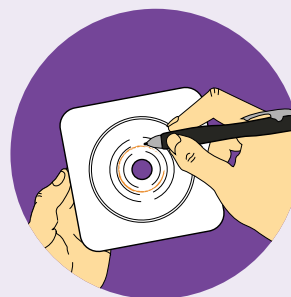


Cut-to-fit skin barriers
Trace the stoma size on the centre of the clear release line on the reverse of the barrier, then cut an opening about 3mm (1/8") larger than your tracing.

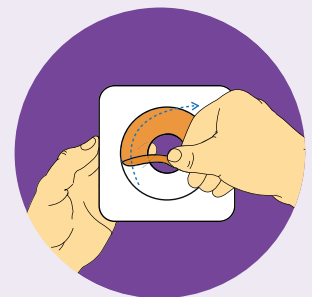


Peel off release liner and centre the opening over the stoma. Apply to clean dry skin.


2 Applying and wearing the pouch

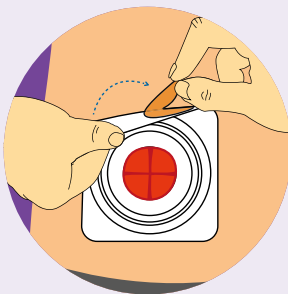

If present, peel off the white release liner from the tape border.

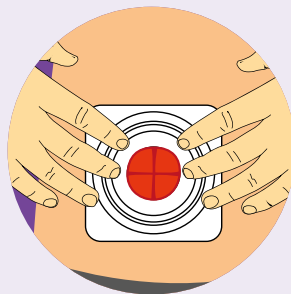

Press gently against skin, smoothing out any wrinkles.

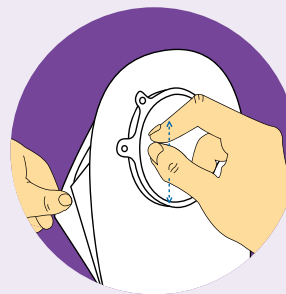


Separate the side of the pouch to allow some air to enter.




Position pouch over the skin barrier. Gently press the flanges together starting at the bottom until they "snap" in place and feel secure.
Always double check the coupling is completely closed, if not repeat the steps.

3


Closing the pouch

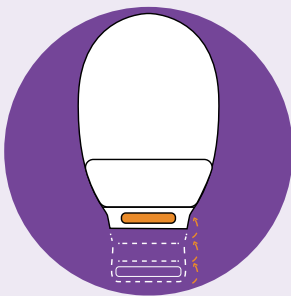

Fold the end up towards you until the interlocking closures line up.

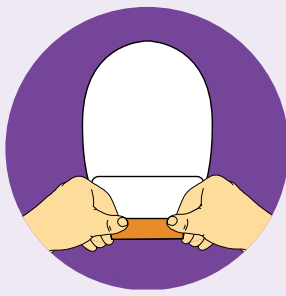


Press and pinch around the closure to secure.

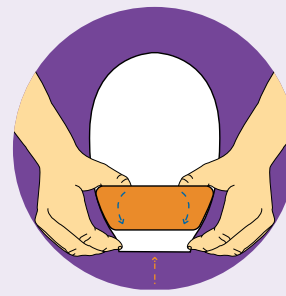


Turn the Lock-it-Pocket™ inside-out to tuck tail inside.

4


Emptying and cleaning the pouch

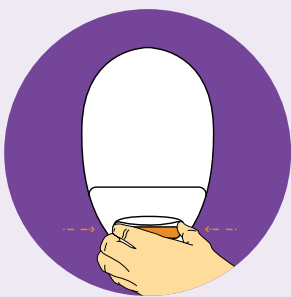

To empty pouch, tilt the tail up towards your body. Peel open the closure and unravel tail. Unlock pouch closure and open the tail by pushing in both ends of the outlet end-strips with fingers; drain pouch.

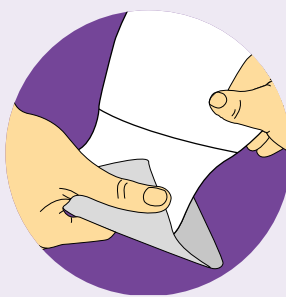


Tips for cleaning the pouch:
Support the pouch with one hand and wipe tail of pouch in firm downward motion with toilet tissue in other hand.

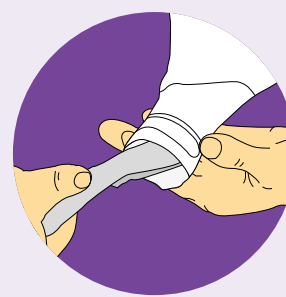


Clean in the inside surface of tail with toilet tissue or baby wipe.

5

Removing the pouch

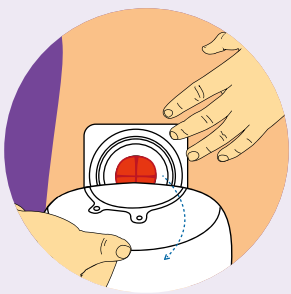

To remove pouch from the flange, hold the flange down gently with one hand and gently un-snap the pouch.

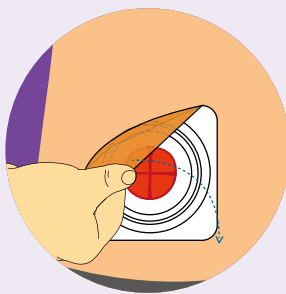

Gently pull the skin barrier down, away from the skin. While supporting the adjacent skin with your other hand.