


Application card

EsteemBody™


One-piece soft convex closed-end pouch

Before application, ensure your hands and the skin surrounding the stoma are clean, dry and free from oily substances.

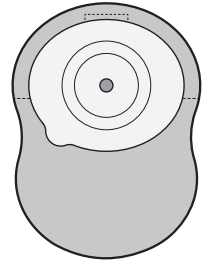


Preparation and application

1

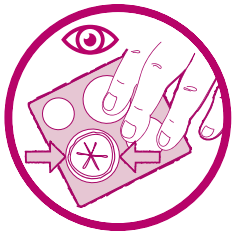


Place the stoma measuring guide over your stoma to confirm the correct size.

2

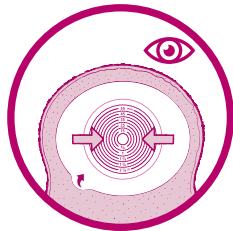


If using a pre-cut baseplate, select a baseplate with a pre-cut stoma opening slightly larger than your stoma.

3

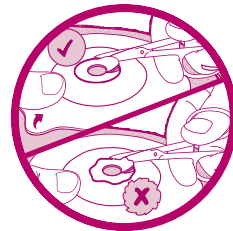

If using a cut-to-fit baseplate transfer the size to your baseplate and cut accordingly.

4

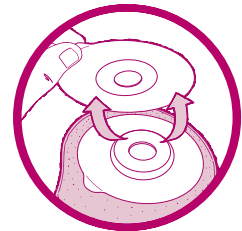


Remove baseplate release liner.

5



Hold the baseplate below your stoma and apply upwards against the skin around your stoma. Press the baseplate against the skin.

6

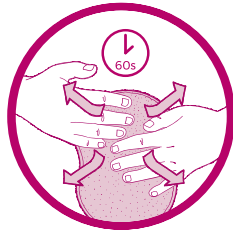

Smooth onto your skin. Maintain slight pressure on the baseplate for 60 seconds to ensure a good seal.

1



Start by gently lifting the top edge of the baseplate, then slowly and gently peel downward.

2



Dispose of the pouch according to local disposal regulations.

Scan to watch application video



For further support, call our Product Specialist team on 0800 282 254.

See package insert for complete instructions for use.