

FlexiSealTM PROTECT



**Designed to protect
patients and
clinicians against
over-inflation risk**



The Flexi-Seal™ PR

Fill Protection

The only fecal management system that senses when the retention balloon is correctly filled and visually indicates over inflation

Patient Protection

Blue finger pocket allows easy and accurate placement of the soft silicone retention balloon

Privacy and Odor Protection

Monitor volume without compromising patient privacy. Improved odor management*

*Compared to Flexi-Seal™ SIGNAL™ FMS

PROTECT Difference

STOP when indicator 'POPS'





GREEN indicator signals correct patient fill volume.*




RED indicator signals balloon over-inflation.



A white cap to help reduce the risk of misconnection between the blue IRIG./RX port and the white inflation port.

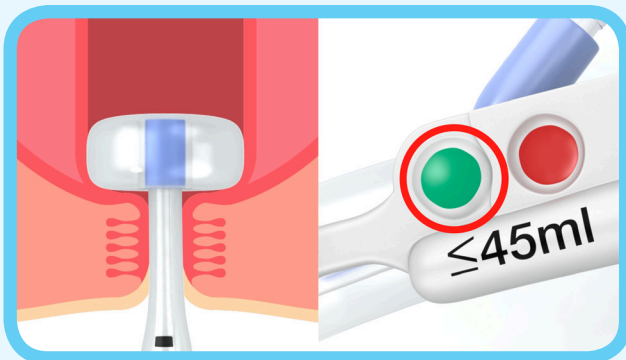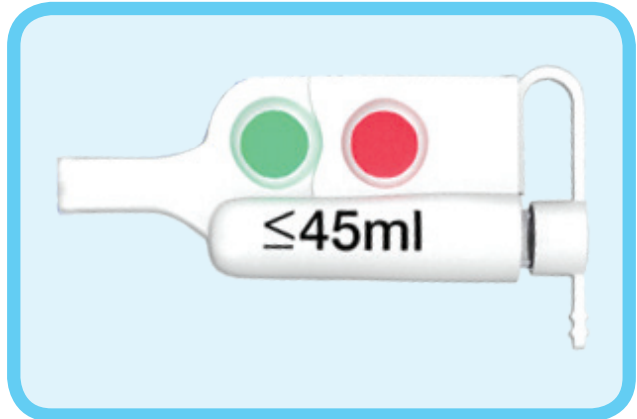

Inflate the balloon by slowly depressing syringe plunger. The green dome will indicate once the balloon has reached the patient's optimal fill volume.*

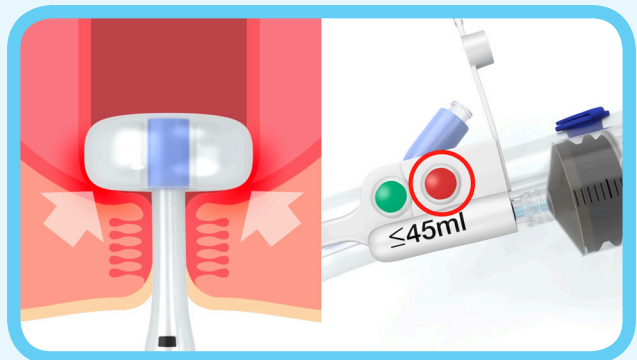

The red dome will indicate if the balloon is overfilled. Overinflation beyond the optimal fit for individual patient may lead to complications. If the red dome is triggered, fully deflate the balloon. The green and red indicators will deflate. Repeat the balloon inflation process and stop inflation once green dome has signaled optimal fill.

* Maximum fill volume is ≤45ml. Filling should stop when the PROTECT green indicator pops out and stays out which could happen before 45ml if the rectal space is smaller than maximum balloon size. Never add more than 45ml in total.

Flexi-Seal™ PROTECT FMS Additional Benefits



Infection Protection

Proven *in vitro* to contain *C. difficile* and its spread into the environment.¹



Total Odor Protection

Odor barrier runs the full length of the catheter, providing an end-to-end odor barrier.²



Sampling Protection

The blue sampling port allows safe and easy stool collection helping minimise the risk of fecal exposure.



Privacy™ Collection Bag

With 6x more odor-capturing compared to the charcoal filtered collection bags³. This filter also eliminates the need of manually releasing the excess gas from the collection bag.⁴

Product Ordering Codes

Flexi-Seal™ PROTECT FMS Kit (1 kit/box, 1bag)	421630
Privacy™ Collection Bag with APS Filter (10/box)	411108

The Flexi-Seal™ Fecal Collector with Hydrocolloid Adhesive:

- An alternative when the use of Flexi-Seal™ PROTECT FMS is contraindicated
- Skin-friendly hydrocolloid adhesive with tape border
- Tapered pouch with drainage port & deodorizing filter
- Not made with natural rubber latex

Flexi-Seal™ Fecal Collector

Flexi-Seal™ Fecal Collector with Hydrocolloid Adhesive (10 Collectors per box)	650078
--	--------

Skin Protectants should be used before and during the use of Flexi-Seal™ PROTECT FMS or Flexi-Seal™ Fecal Collectors.

Skin Protectants

Sensi-Care® Clear Zinc (24x 2 oz tubes/case) <i>Protects minor skin irritation, Helps seal out wetness</i>	413586
Sensi-Care® Protective Barrier (24x 4 oz tubes/case) <i>Adheres to and helps treat denuded skin</i>	325614

REFERENCES: 1. Jones S, Towers V, Welsby S, et al. Clostridium difficile Containment Properties of a Fecal Management System: An In Vitro Investigation. Ostomy Wound Management 2011;57(10):38–49.
2. Odor Barrier Testing. 130124-001. Data on file, ConvaTec Inc. 3. Flexi-Seal™ Privacy Bag Filter Evaluation. 121412-001. Data on file, ConvaTec Inc. 4. Minimizing the spread of *C. difficile* spores from the release of gas. February 19, 2013. Data on file, ConvaTec Inc.

To learn more, call:

1-800-422-8811

Mon-Fri, 8:30 AM-7:00 PM

www.convatec.com