
en Oxygen Mask, High Concentration, Non-Rebreather
da Ilmtaske, Høj koncentration uden genånding
sv Syrgasmask, hög koncentration, icke-återändningsbar
no Oksygenmaske, Høy koncentrasjon uten gjenånding
fi Happimaski, korkea pitoisuus, ei-takaisinhengitys
ru Кислородная маска, высокой концентрации, нереверсивная
de Sauerstoffmaske, hohe Konzentration, ohne Rückatmung
nl Zuurstof masker,hoge concentratie,non-rebreather
fr Masque à oxygène, Haute Concentration, Sans Réinspiration
pl Maska tlenowa do wysokich stężeń bez oddechu zwrotnego
es Mascarilla de oxígeno, alta concentración, sin reinhalación
pt Máscara de oxigénio, de alta concentração, sem reinalação
it Maschera ossigeno, Alta concentrazione, senza rいrespirazione
cs Kysliková maska, vysoká koncentrace bez recyklace vzduchu
tr Yüksek Konsantrasyonlu Oksijen Maskesi, Gerisolumasız
gr Μάσκα Οξυγόνου, Ύψη λήξης Συγκέντρωσης, Μη Επανεύστροφη

(en) Oxygen Mask, High Concentration, Non-Rebreather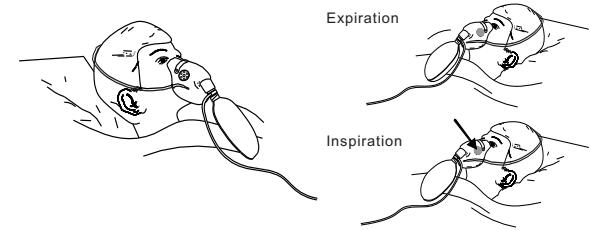

Description

Oxygen Mask, High Concentration, Non-Rebreather is a disposable oxygen mask which can be used as a high concentration oxygen mask or as a partial rebreather mask.

Oxygen tubing (if included) can be connected to the Oxygen flow-meter via Fits-All connector without requiring an additional adapter.

Contraindications

Severe Hyperoxia, Apnea

Warnings

- Oxygen, while not flammable, supports combustion. Do not use near excessive heat, open flames, or other sources of ignition.
- This device is intended for single-use only and should not be reused. Re-use may lead to increased risk of infection or cross contamination and physical properties of the device may no longer be optimal for intended use.
- Use of low Oxygen flow rates may result in excessive rebreathing of CO₂.
- Sterilization or use of cleaning solutions may result in retention of harmful residues.
- Always ensure tight connection before use. Monitor proper connection during use.

Instructions for Use

- Unpack and connect to Oxygen supply.
- Inflate reservoir bag completely prior to application.
- Place mask over patient's face and adjust for best fit.
- Set Oxygen flow to a rate preventing the bag from fully collapsing, when patient inhales. Observe bag for sufficient replenishment.
- To use as a partial rebreather mask: Remove rubber discs on mask sides and bottom.

Symbols and explanations

REF	Catalogue number	Use-by date	LOT	Batch code
	Do not re-use			Use-by date
	Manufacturer			Batch code
	Do not use if package is damaged			
EC REP	Authorised representative in the European Community			

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