

THREE PLATEAU SIZES

The Esteem™+ Flex Convex system is available in 3 plateau sizes (V1, V2, V3) to accommodate a range of stomas and abdomens.



1. POUCH TYPE	2. STOMA SIZE (MM)	CONVEXITY OPTION					
		CUT-TO-FIT			PRE-CUT		
		V1	V2	V3	V1	V2	V3
Closed	20-25	421612	421613	421614			
	25-35	421612	421613				
	35-43	421612					
Drainable	20-25	421615	421619	421622			
	25	421615	421619	421622	421616	421620	
	25-30	421615	421619				
	30	421615	421619		421617	421621	
	30-35	421615	421619				
	35	421615	421619		421618		
	35-43	421615					
Urostomy	10-25	421623	421626	421628			
	15	421623	421626	421628	421624	421627	421629
	20	421623	421626	421628	421625		
	25-35	421623	421626				
	35-43	421623					

CONVEXITY PLATEAU SELECTION GUIDELINES/ CONSIDERATIONS

Full assessment of the stoma and abdomen should be made when selecting the convexity plateau size. A healthcare professional should assess the need for skin barriers with convexity prior to use.

Different convexity plateau widths can be selected to provide pressure where needed. Here are some guidelines to aid your choice of convexity plateau:

Stoma Protrusion

Protruding: use widest option available – pressure distributed over a wider area around the stoma

Flush: use narrowest option available - also assess the abdomen type and where pressure is required

Retracted: use narrowest option available – this gives pressure closest to the stoma to help the stoma protrude

Abdomen

Skin folds/ creases: use widest option available - to help to move any skin folds out of the way

Rigid/ rounded: use widest option available – pressure distributed over a wider area and can aid adhesion

Flaccid - Average tone: use option most suitable to stoma protrusion

Stoma Shape

Circular/round: use pre-cut if available

Non-circular or oval: use cut-to-fit